

ForeSight Jr™ Sensor Sizing Guide

Clean



Apply



Connect




Patient	Cerebral Sensor	Non-Cerebral Sensor (ie. Flank)
Neonate	 Cerebral < 8 kg Small Sensor	 Non-cerebral < 5 kg Small Sensor
Cerebral Pediatric Overlap ≥ 3 kg to < 8 kg	 Small or Medium Sensor Consider size which allows for bilateral StO ₂ monitoring	
Pediatric	 Cerebral ≥ 3 kg Medium Sensor	 Non-cerebral ≥ 3 kg Medium Sensor
Adolescents	 Cerebral ≥ 40 kg Large Sensor	 Non-cerebral ≥ 40 kg Large Sensor


*Use clinical judgement when choosing sensor size for available forehead space

ForeSight Jr™ Sensor Locations

ForeSight Jr™ Small Sensor (FSESS)

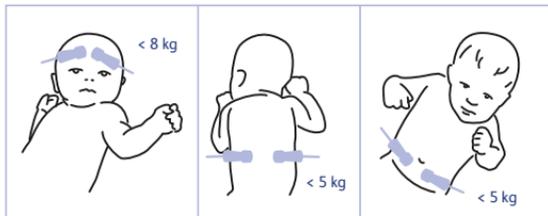

ForeSight Jr™ Medium Sensor (FSESM) ≥ 3 kg

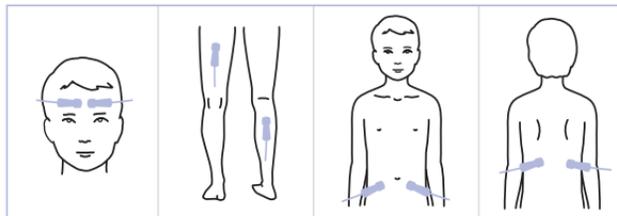


A unique nonadhesive solution is available for a ForeSight Jr™ Small Sensor (FSESNS)



Cerebral sensor size overlap considerations

≥ 3 kg → < 8 kg

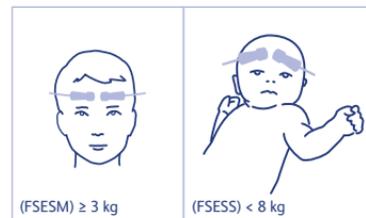

Consider size which allows for bilateral StO₂ monitoring

CAUTION: Federal (United States) law restricts these devices to sale by or on the order of a physician. See instructions for use for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

BD, Irvine, CA 92618 USA

bd.com



BD, the BD Logo, ForeSight and ForeSight Jr are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2026 BD. All rights reserved. BD-165695 v1.0 US