


BD Nexiva™ Closed IV Catheter System—Dual Port

Tips for success

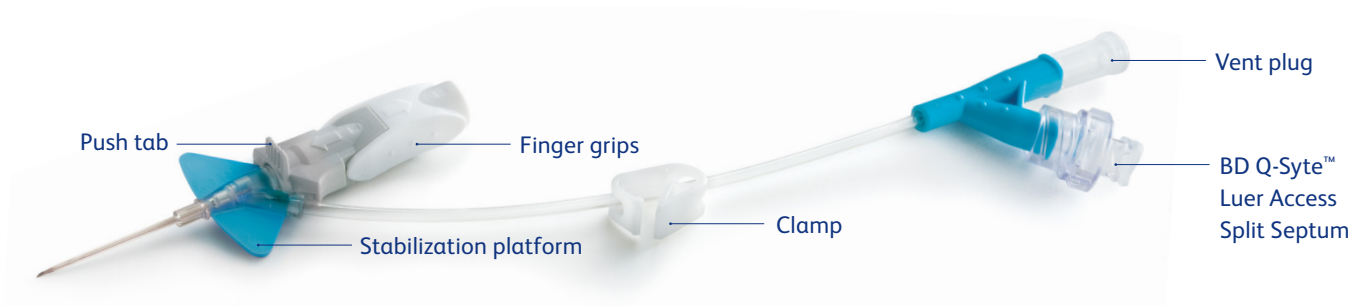

Sharp insertion:

- Keep the stabilization platform and finger grips pushed together
- **Do not** insert while holding onto the stabilization platform as this will move the needle back into the catheter
- Avoid holding in a manner that pushes against the stabilization platform

Easy vessel access:

- Look for initial blood return in the catheter near the insertion site
- Lower and slightly advance the entire system to ensure the catheter tip is within the vessel
- Look for continuous blood flow along the extension tubing to confirm the catheter is still in the vessel

Smooth catheter advancement:

- Push the catheter forward off the needle, rather than pulling the needle back
- Use the pad of your index finger, not your fingernail, behind the push tab
- Push down and forward against the push tab

Simple needle removal:

- Push down gently on the stabilization platform, and pull back on the finger grips until the needle shield releases
- **Do not** touch the push tab during needle removal

Blood containment:

- Secure the vent plug and the needle-free connector before insertion
- Clamp the extension tubing before replacing the vent plug

System access:

- Insert the male luer into the needle-free connector using a straight-on approach
- Clamp **before** disconnecting the flush syringe
- Hold the bottom of the needle-free connector when removing the syringe or tubing during disconnection
- Use a straight-off approach when removing the male luer

Please consult product labels and inserts for instructions for use, indications, contraindications, warnings, and precautions.

BD, Sandy, UT 84070 USA

bd.com

