



BD Nexiva[™]

Closed IV Catheter System—Single Port

Catheter + high-pressure extension set* + stabilization platform

*24-gauge catheter systems should not be used with power injectors.






Secure vent plug

> Must remain in place during insertion for:

- Flashback
- Blood containment

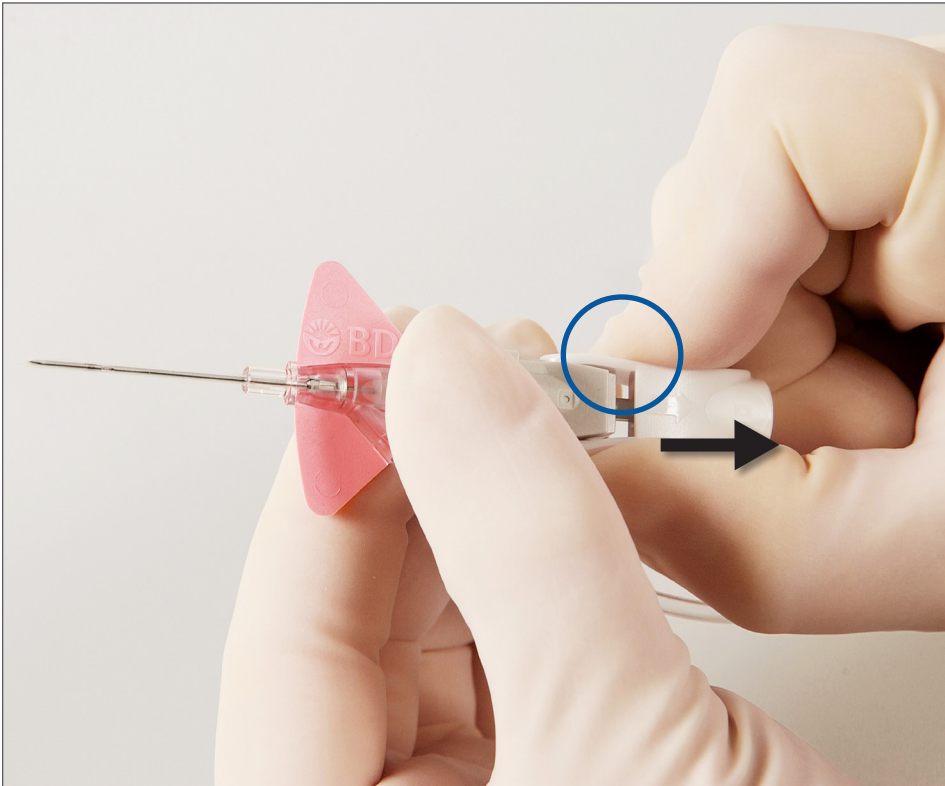


Release seal



- > Loosens catheter for easier advancement
- > Pull back ~ 1/8 inch

Colour to white
everything tight

- > Push coloured platform and white grip back together

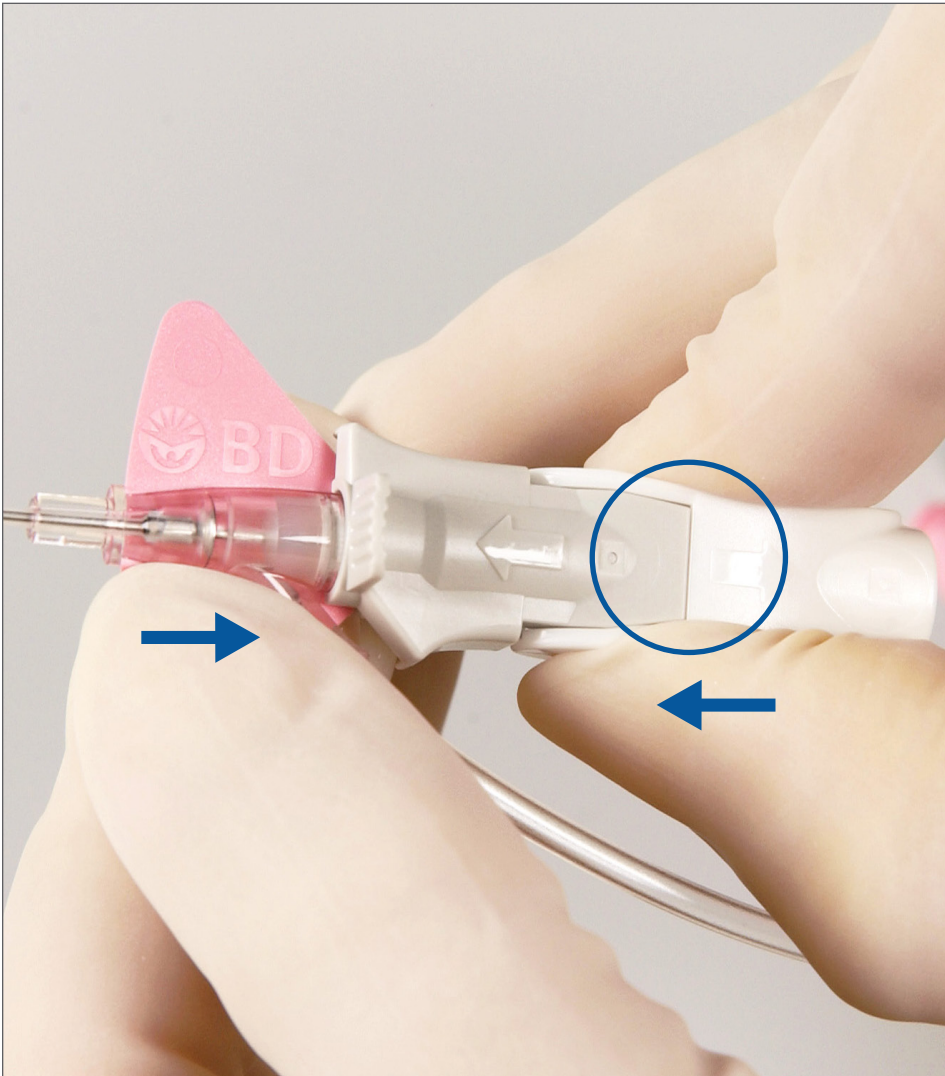



Keep it **sharp**

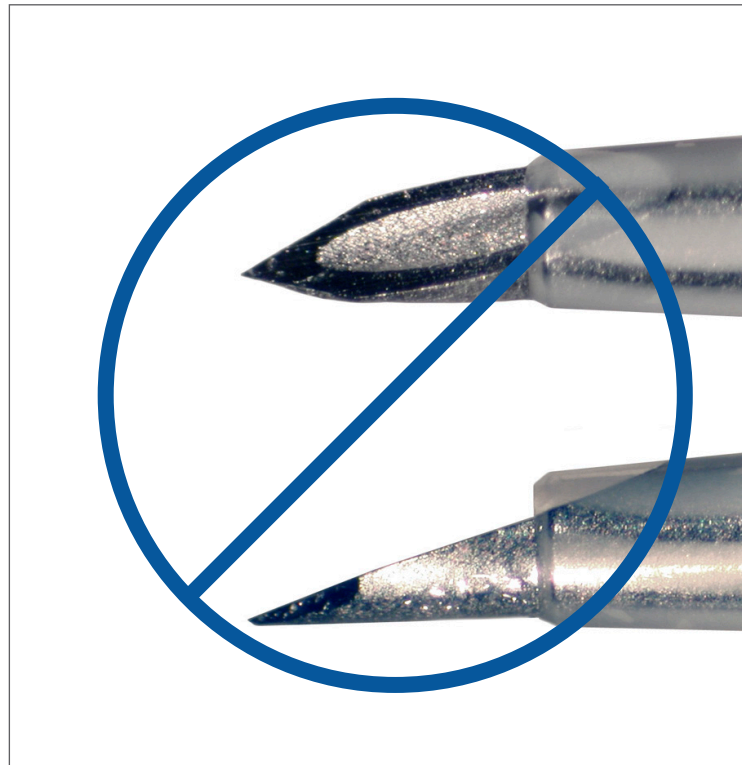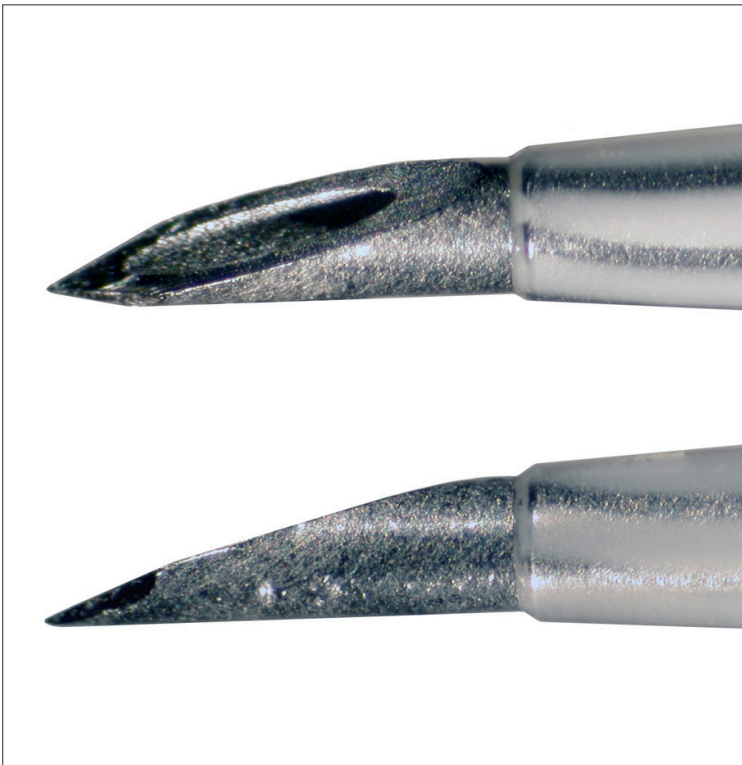

BD Instaflash™ Needle Technology

- > Immediate blood return is seen along the catheter

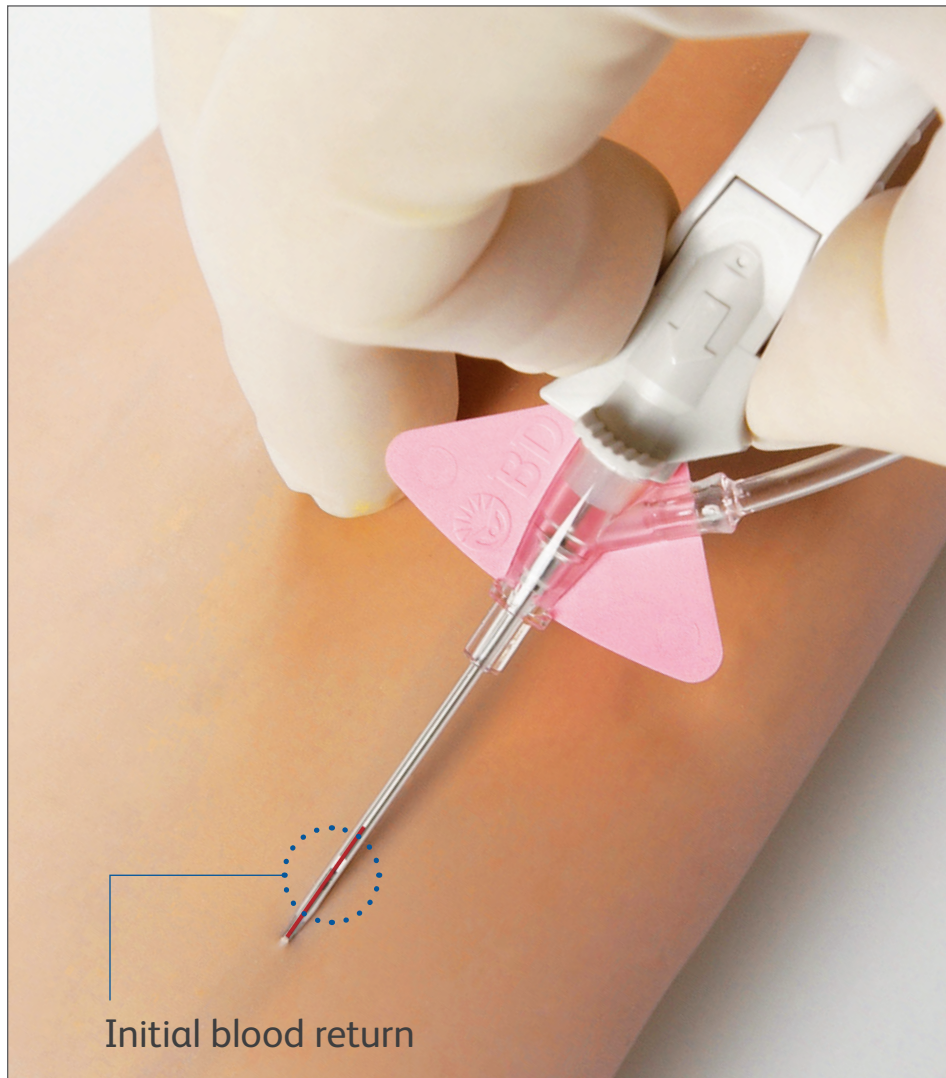


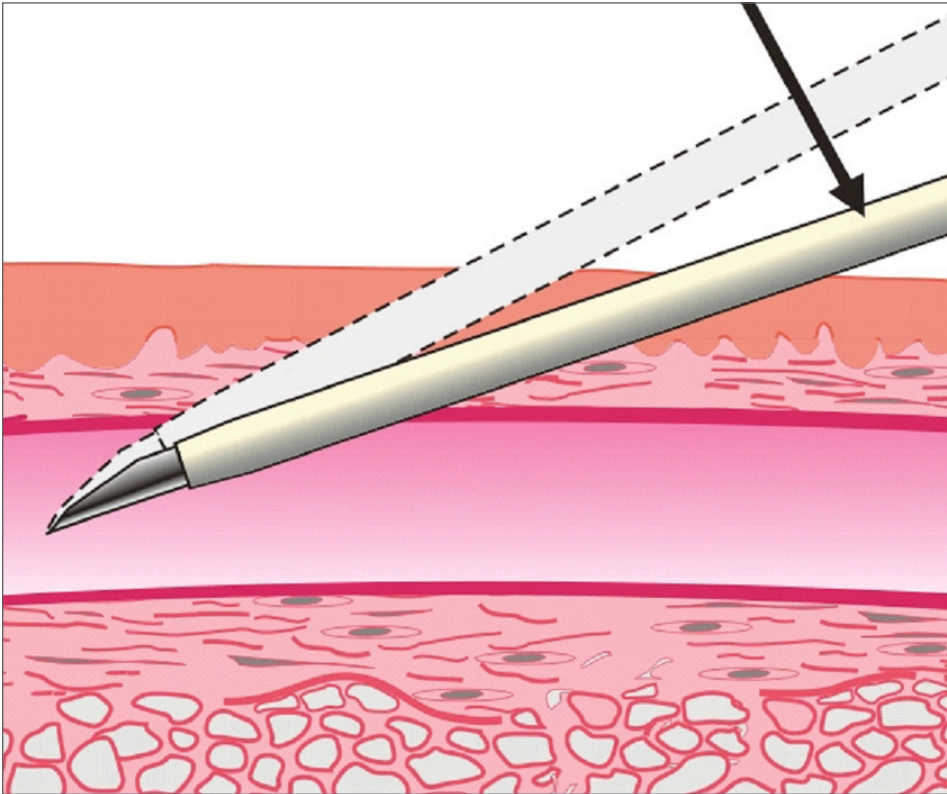


Lower and advance

- > Lower catheter almost parallel to skin
- > Slightly advance entire unit to ensure catheter tip is within vessel


Forward first

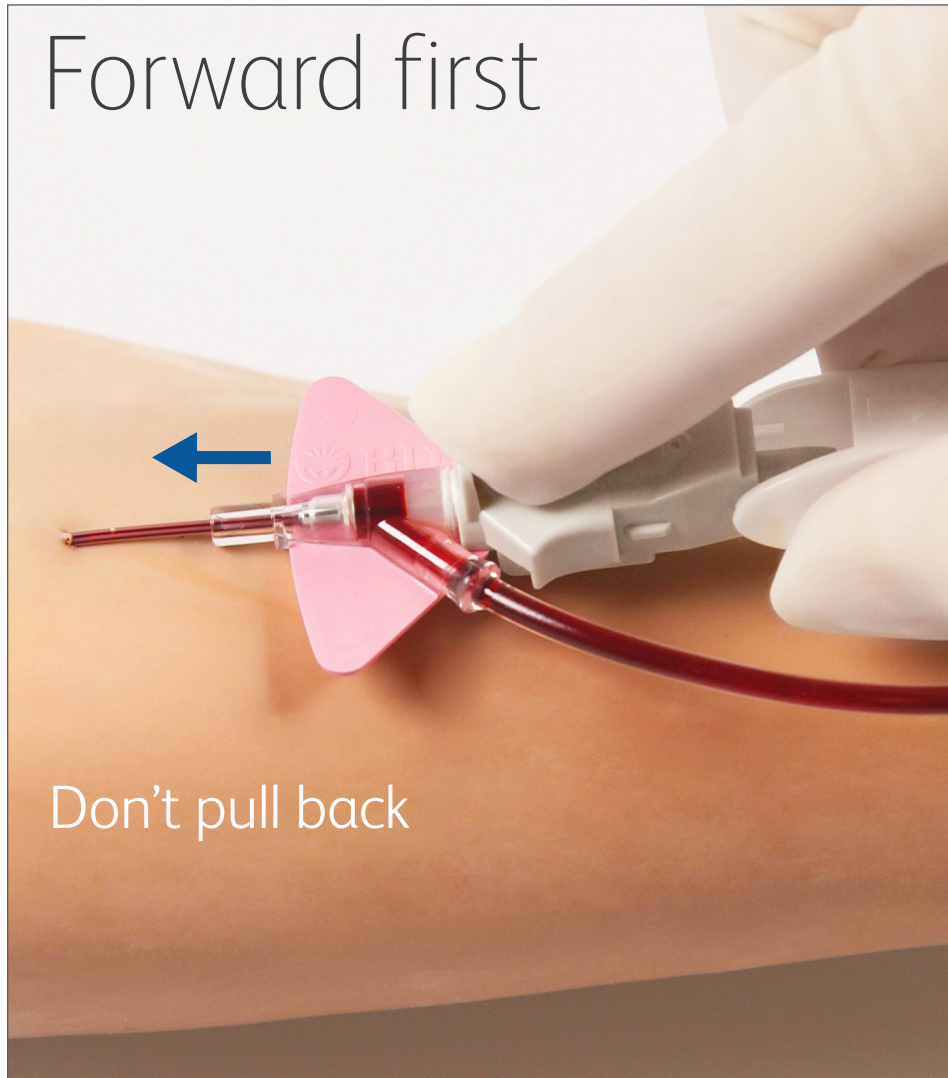


Pad to tab

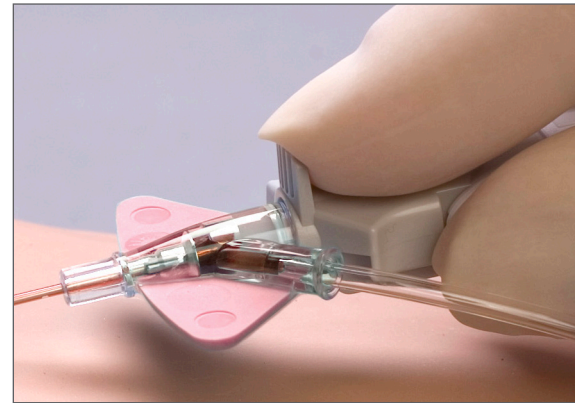


Not fingernail

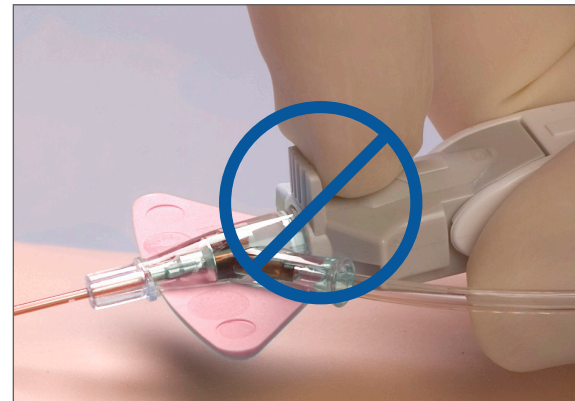


Forward first

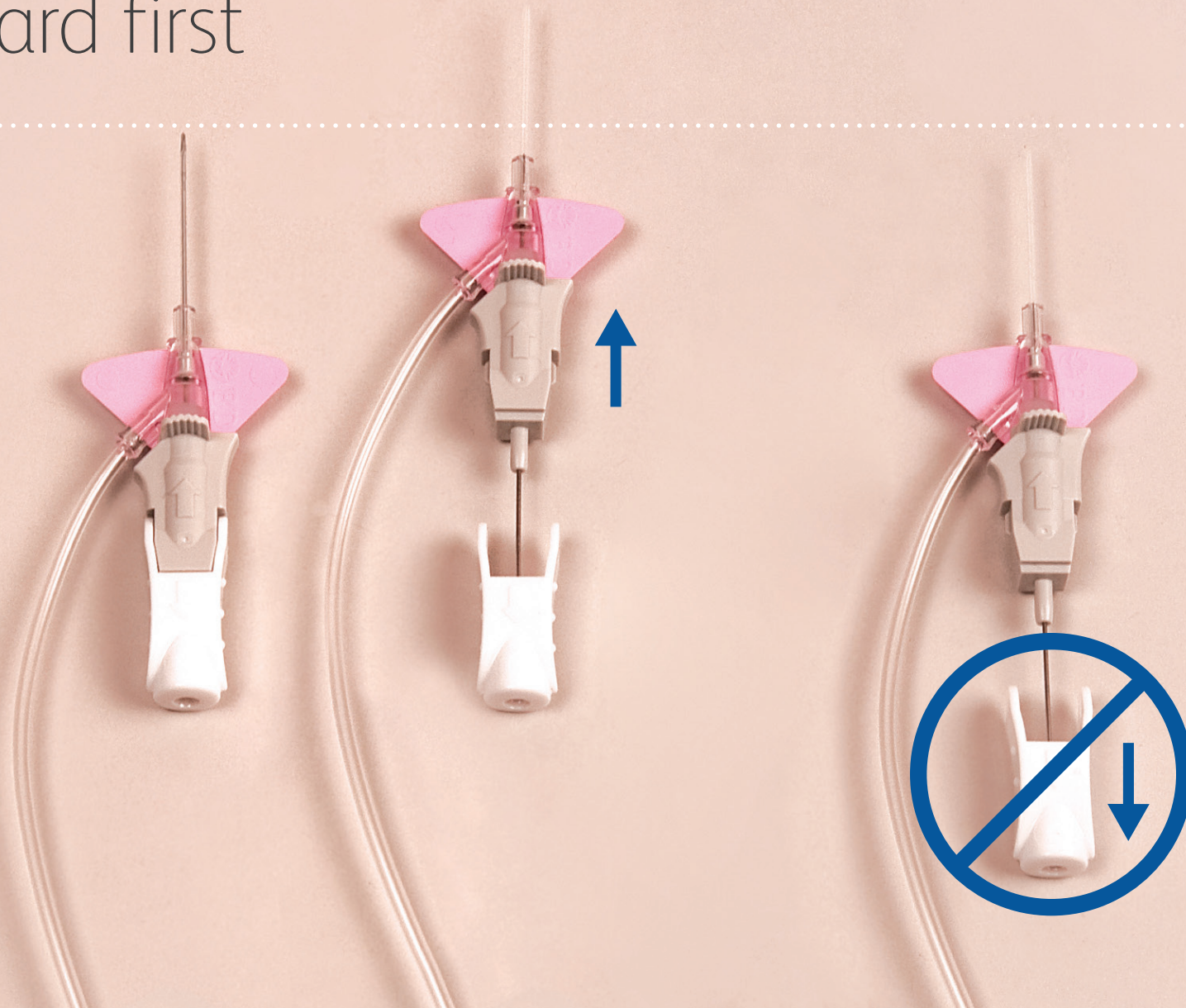

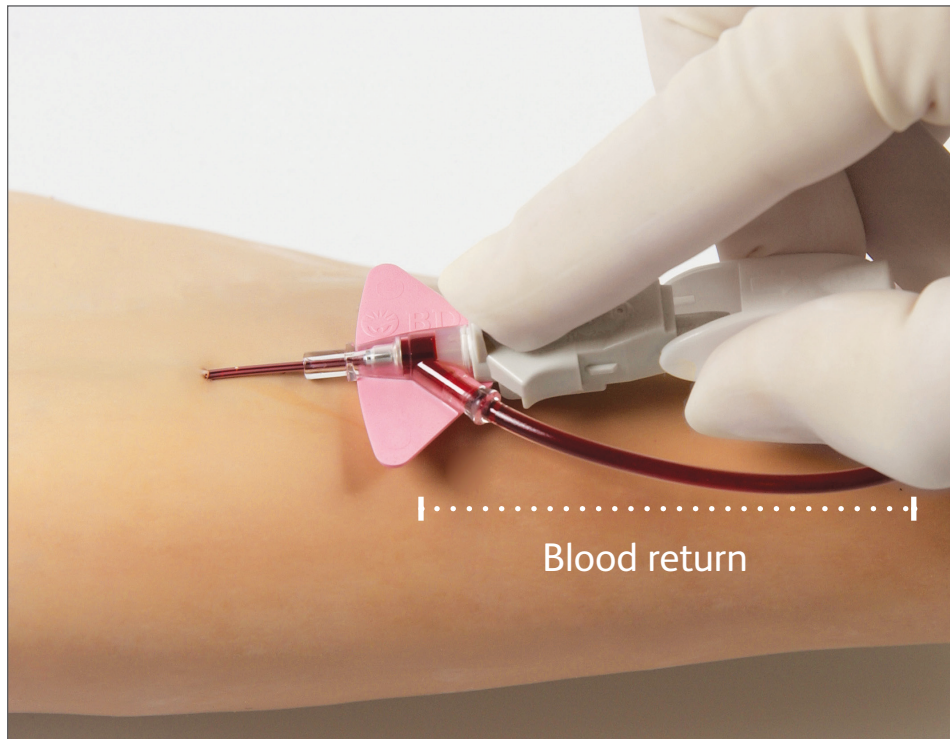

Secondary flash

- > Blood return continues up the extension
- > Continuous flow confirms the catheter tip is within the vessel lumen
- > Quickens as needle is withdrawn

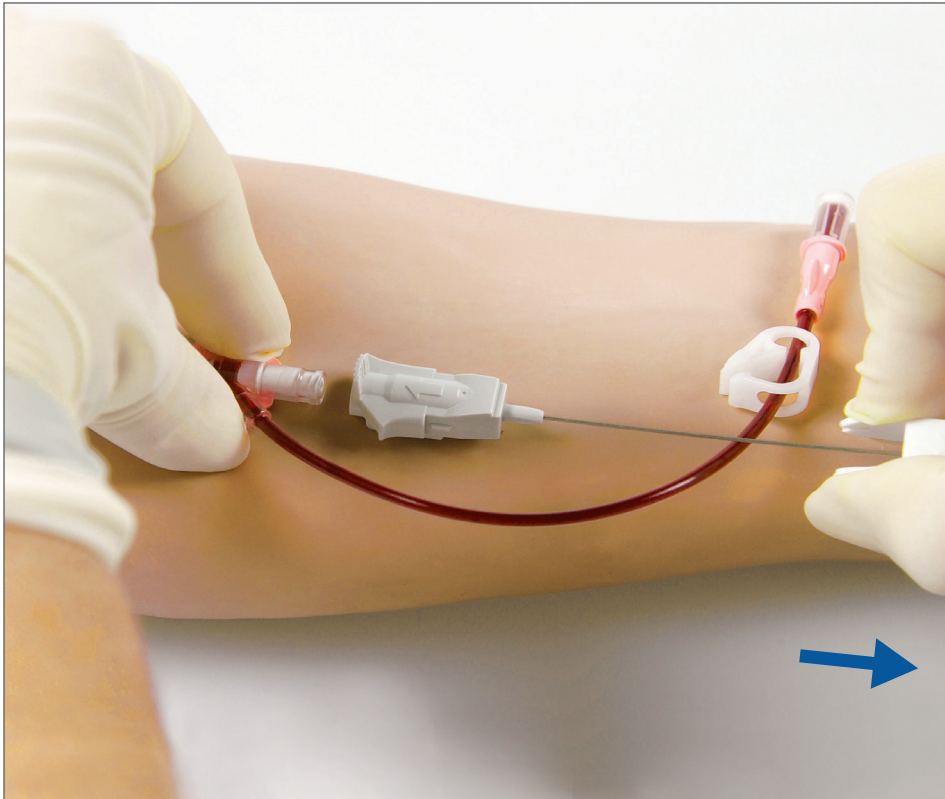


Stabilize catheter
and remove needle



Stabilize system

- > Transparent dressing should cover the septum

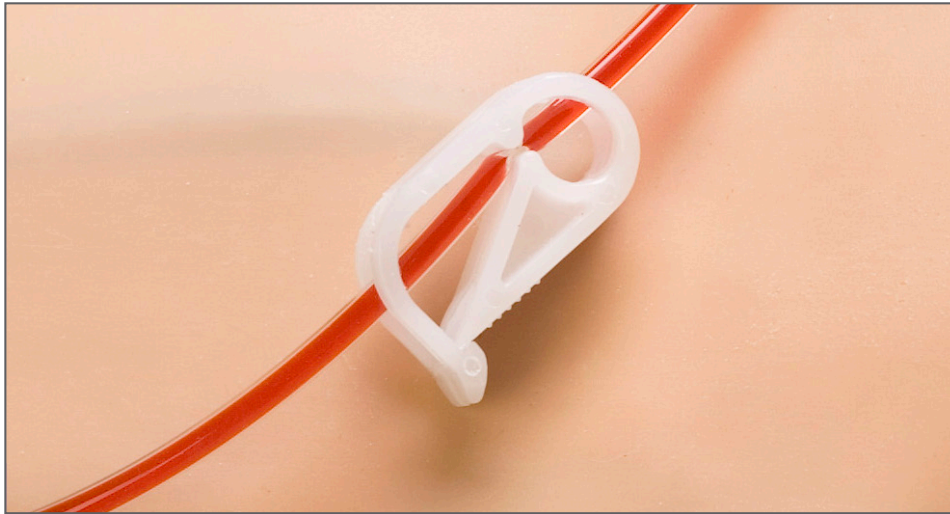

Clamp and replace vent plug

- > Clamp extension set
 - Prevents blood leakage
- > Remove vent plug and attach administration tubing or access port
- > Release the clamp to flush or infuse
- > Complete securement of extension tubing



BD–Canada, Mississauga, ON, L5N 0B3

bd.com

BD, the BD logo, Instaflash and Nexiva are trademarks of Becton, Dickinson and Company or its affiliates.
© 2021 BD. All rights reserved. BD-45910 (10/21)

