

BD Nexiva™

Closed IV Catheter System—Single Port


Points to practice



1

Preparation

- Secure the vent plug.
- The clamp should not be engaged.
- Twist to remove the needle cover.
- Holding as shown, pull back approximately 1/8" on the finger grips (Fig. 1a).
- Push the finger grips back to their original position so the stabilization platform and finger grips are snugly together (Fig. 1b).

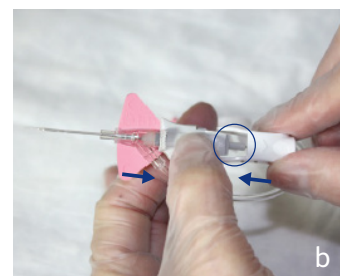

b



2

Insertion and flashback


- Hold the system as shown and access the vessel (Fig. 2).
- Initial blood return is along the catheter, then up the extension tube. Look at the catheter for initial blood return.
- Lower and advance the entire catheter and needle unit slightly to ensure the catheter tip is within the vessel.



3

Advancement

- Place the index finger pad behind the push-tab and push the catheter off the needle into the vessel (Fig. 3).
- Tip: Do not pull back on the needle during advancement.*

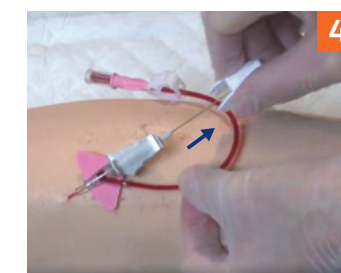


4

Needle removal

- Stabilize the catheter system, by holding down the extension set, and pull back on the finger grips until the push-tab component releases from the stabilization platform (Fig. 4).
- Discard the shielded needle into a puncture-resistant, leak-proof sharps container.

Tip: Do not hold onto the push-tab component of the device as this will prevent the release of the needle shield.

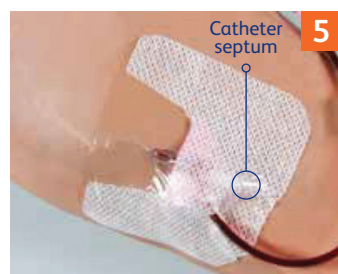

5

Stabilization

- Apply a transparent dressing to cover the septum, allowing maximum use of the extension tubing (Fig. 5).



6

Preparation for use

- Engage the clamp.
- Remove the vent plug and attach the administration tubing or needle-free connector (Fig. 6).
- Release the clamp to flush or infuse.
- Complete securement of the extension tubing.

Consult the product insert for complete instructions, warnings and cautions.

BD—Canada, Mississauga, ON, L5N 0B3

bd.com

BD, the BD Logo and Nexiva are trademarks of Becton, Dickinson and Company or its affiliates.
© 2022 BD. All rights reserved. BD-51904 (01/22)

