

BD[®] SiteRite[™] 9

Ultrasound System

Voice Recognition



Voice Commands (if enabled)

General Voice Commands:

- SiteRite[™] Nine Show Voice Commands
- SiteRite[™] Nine Close Menu
- SiteRite[™] Nine Freeze Image
- SiteRite[™] Nine Unfreeze Image
- SiteRite[™] Nine Save Image
- SiteRite[™] Nine Volume Up
- SiteRite[™] Nine Volume Down
- SiteRite[™] Nine Arm Circumference (0 - 99)
- SiteRite[™] Nine Delete Arm Circumference
- SiteRite[™] Nine Exit Site (0 - 99)
- SiteRite[™] Nine Delete Exit Site
- SiteRite[™] Nine Trim Length (0 - 99)
- SiteRite[™] Nine Delete Trim Length
- SiteRite[™] Nine Start Thirty Second Timer
- SiteRite[™] Nine Start Sixty Second Timer
- SiteRite[™] Nine Stop Timer

Ultrasound Voice Commands:

- SiteRite[™] Nine Brightness Up
- SiteRite[™] Nine Brightness Down
- SiteRite[™] Nine Depth Up
- SiteRite[™] Nine Depth Down
- SiteRite[™] Nine Doppler
- SiteRite[™] Nine Exit Doppler
- SiteRite[™] Nine Evaluate Vessel
- SiteRite[™] Nine Accept
- SiteRite[™] Nine Reject

Sherlock Voice Commands:

- SiteRite[™] Nine Calibrate
- SiteRite[™] Nine Freeze ECG
- SiteRite[™] Nine Mute Sherlock
- SiteRite[™] Nine Unmute Sherlock



For example, you could say: “SiteRite[™] Nine Save Image”