

Sherlock 3CG+™

Tip Confirmation System

Voice Recognition



Complete list of commands

- Sherlock calibrate sensor
- Sherlock freeze signal
- Sherlock print record
- Sherlock audio on/off
- Sherlock volume up/down
- Sherlock trim length (0 – 99)
- Sherlock exit site (0 – 99)
- Sherlock delete trim length
- Sherlock delete exit site
- Sherlock maximize sensor
- Sherlock navigation on/off
- Sherlock R-peak on/off
- Sherlock help menu
- Sherlock close menu



For example, you could say: “Sherlock trim length forty three”