


BD® Blunt Fill and Blunt Filter Needles

Important usage guidelines




1. Attachment

Attach the BD® Blunt Fill or Filter Needle to any standard Luer-Lok or Luer-Slip hypodermic syringe. Twist until firmly seated to connect to a luer-lock syringe. Pull shield straight off the needle.

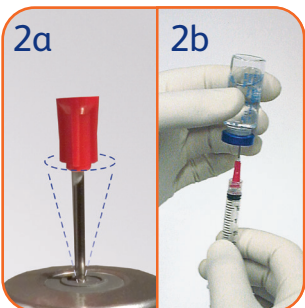

2. Aspiration

- 2a. Puncture stopper at 90° with the BD® Blunt Fill Needle to minimize risk of catching internal wall of vial stopper.
Note: For glass ampoules use the BD® Blunt Filter Needle.
- 2b. Draw up medication in accordance with your institution's protocol.

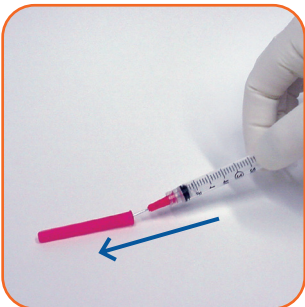

3. Recapping

Passively recap BD® Blunt Fill or Blunt Filter Needle.



4. Disposal

Detach needle from syringe and immediately discard into an approved sharps collector after single use.

Remember

- The BD® Blunt Fill and Blunt Filter Needles are not to be used for skin injections.
- Follow your institution's protocol for aseptic technique.
- Follow your institution's policy for safe disposal of all medical waste.
- Use the BD® Blunt Filter Needle with glass ampoules.
- The BD® Blunt Filter Needle is a one-way filter and should be removed once fluid is filtered into the syringe.
- To connect the BD® Blunt Fill or Blunt Filter Needle to a Luer-Slip syringe, push firmly.

BD® Blunt Fill Needle



BD® Blunt Filter Needle



Note hub color difference between blunt filter vs blunt fill needle

This guide is not intended as a replacement for the detailed information contained in the IFU. Please consult BD product labels and package inserts for any indications, contraindications, hazards, warnings, cautions and instructions for use.

1 Becton Drive, Franklin Lakes, NJ, 07417, USA

[bd.com](https://www.bd.com)

Scan this code to watch a video on how to use the BD® Blunt Fill and Blunt Filter Needle



BD and the BD Logo are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2023 BD. All rights reserved. BD-1656 (05/23)