

# Nursing administration using BD PhaSeal™ Optima

BD PhaSeal™ Optima  
Connector C35-O ▶



Obtain the BD PhaSeal™ Optima Connector for step 1 from your unit's supply closet/med room



**Luer lock** the Connector onto the patient's IV line or access port per facility protocol.



**Engage** the Injector on the IV administration set to the Connector by pushing the Injector straight onto the Connector until the two pieces snap together. **Release** clamp and administer infusion per facility protocol.



Once the infusion is complete, **stop** the pump, **clamp** the IV lines and **disengage** the Injector by pulling straight back.



BD PhaSeal Optima  
Backprime Video



BD PhaSeal Optima  
IV Push Video



BD PhaSeal Optima  
Vial Adaptor Video



**Note:** The BD PhaSeal™ Optima Connector should remain connected to the IV line to maintain a closed system and may be used for additional IV administration or IV push procedures.

[bd.com](http://bd.com)

BD, the BD Logo, Optima and PhaSeal are trademarks of Becton, Dickinson and Company. All other trademarks are the property of their respective owners. © 2021 BD and its subsidiaries. All rights reserved. BD-30333 (04/21)

