


BD® PosiFlush™ SafeScrub Prefilled Saline Syringe

With disinfection unit, 0.9% sodium chloride injection, USP, and 70% isopropyl alcohol


Points to Practice

To ensure safe medication preparation and administration, clinicians should practice the “7 rights” of medication administration: right patient, right drug, right dose, right time, right route, right reason, and right documentation.

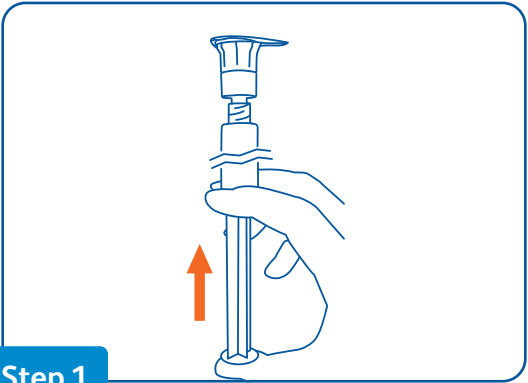


Step 1

Tear open flow wrap packaging at either end and remove device. Inspect device for product integrity. Depress plunger with disinfection unit on to relieve the resistance between the stopper and the barrel.

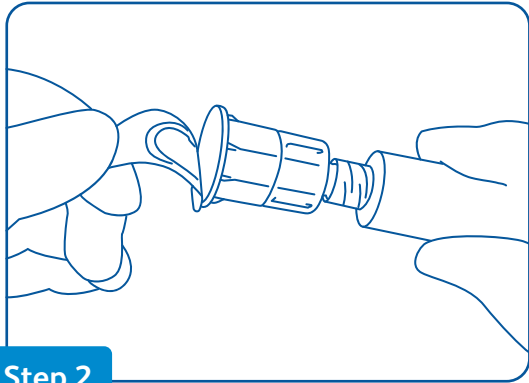


Step 2

Using aseptic technique, peel off foil cover of the disinfection unit. Do not touch the interior foam pad. Use device immediately after removal of lid.

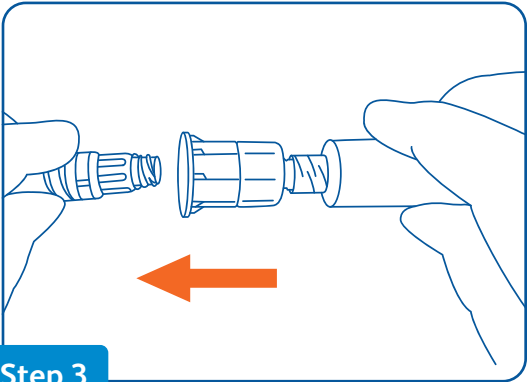


Step 3

Center the disinfection unit onto the needleless access device. **Press the disinfection unit/device** until the foam encapsulates the surface of the needleless access device.

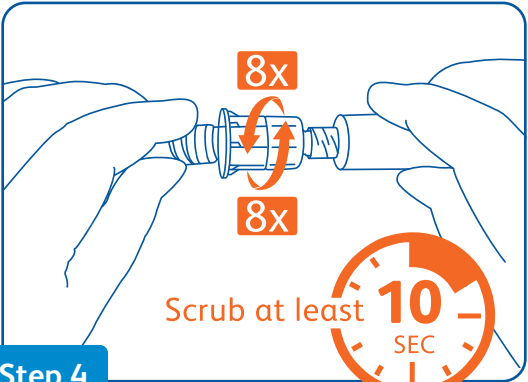


Step 4

Scrub by twisting back and forth for at least **10 seconds**. Do this for a minimum of **8 repetitions clockwise** alternating with **8 repetitions counterclockwise**.

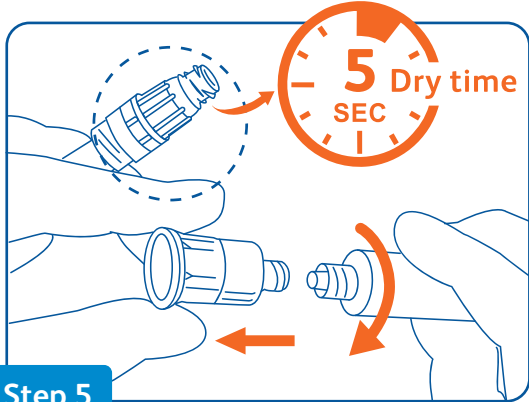


Step 5

Allow the needleless access device to **dry for a minimum of 5 seconds**. Aseptically remove disinfection unit/syringe cap by twisting counterclockwise and discard as per facility protocol, including potentially unused disinfection unit. **DO NOT REUSE** disinfection unit/syringe cap. **WARNING:** Do not leave the disinfection unit on the needleless access device after disinfecting.

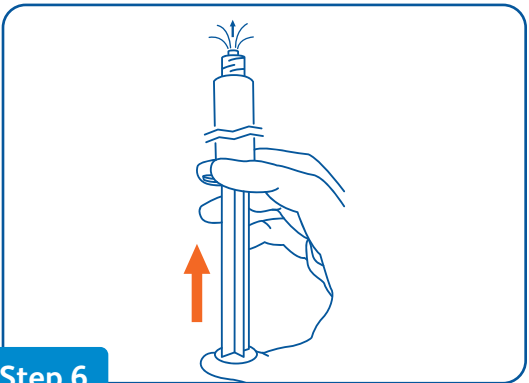


Step 6

Hold the syringe upright and expel the air in the syringe.

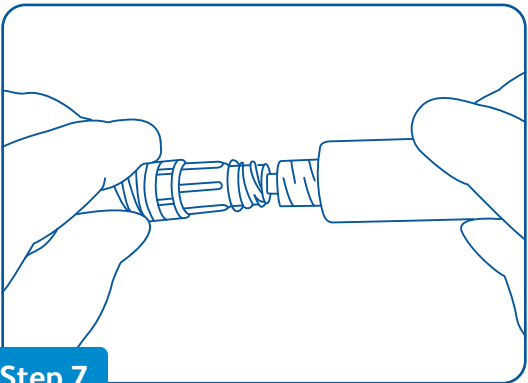

Step 7

Attach the syringe to the needleless access device then flush system following institution's policy and indwelling device manufacturer's recommendations.

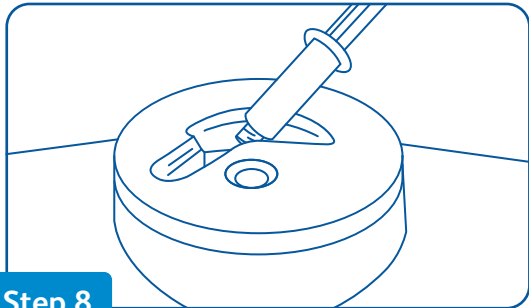

Step 8

Discard used syringe and any unused portion of the solution according to institution policy. **DO NOT REUSE.**

- This product is not made with natural rubber latex or preservatives.
- Sterile isopropyl alcohol solution & sterile sodium chloride solution.
- **CAUTION:** Federal (USA) law restricts this device to sale by or on the order of a physician.

Description: PosiFlush™ SafeScrub is a prefilled flush syringe with an integrated disinfection unit. It comprises a flush syringe and a disinfection unit that is permanently attached to the distal end of the syringe cap. The prefilled flush syringe contains 0.9% sodium chloride injection, USP and performs as a delivery device at the time of administration. The disinfection unit is a disinfection cleaner that contains 70% isopropyl alcohol (IPA).

Intended Use: The 0.9% sodium chloride injection, USP, PosiFlush™ SafeScrub prefilled flush syringe with an integrated disinfection unit is intended to be used as a disinfection cleaner for needleless access devices attached to indwelling vascular access devices (VADs) and flushing of these VADs.

Contraindications: Do not use this device on patients weighing less than 2.1 kg (4.6 lbs).

This guide is not intended as a replacement for the detailed information contained in the IFU. Please consult BD product labels and package inserts for any indications, contraindications, hazards, warnings, cautions and instructions for use.

