

BD PhaSeal™ Optima System for intravesicular administration

Equipment necessary:

1. BD PhaSeal™ Optima Connector (C35-0)
2. Syringe unit prepared with drug
3. Luer lock Foley tip adapter for administration (brand per facility's protocol)

Procedure:

NOTE: Ensure all luer lock connections are securely tightened.

1. Luer lock the connector to the Foley tip adapter.



2. Insert the Foley tip adapter with the connector attached into the patient's Foley catheter instillation port.



3. Engage the injector on the syringe unit to the connector assembly by pushing the injector straight onto the connector until the two pieces snap together. Administer the drug.



4. Disengage the injector from the connector assembly by pulling straight back.



NOTE: Follow aseptic technique and local guidelines for safe handling of hazardous drugs and facility protocol for drug administration, including disinfection of all IV access ports and BD PhaSeal™ System components. The connector/Foley tip adapter assembly may be left in place after instillation per facility protocol.

Dispose of the syringe unit and Foley tip adapter connector assembly without disassembling and in accordance with facility protocol.

