


BD PhaSeal™ Optima Infusion Clamp Instructions




1. Position the BD PhaSeal™ Optima Infusion Clamp around the BD PhaSeal™ Optima Injector/Connector assembly as shown in diagram on inside of Clamp.

Helpful tip: Note the Y-site with the BD PhaSeal™ Optima Components is placed at the smaller opening of the infusion clamp.

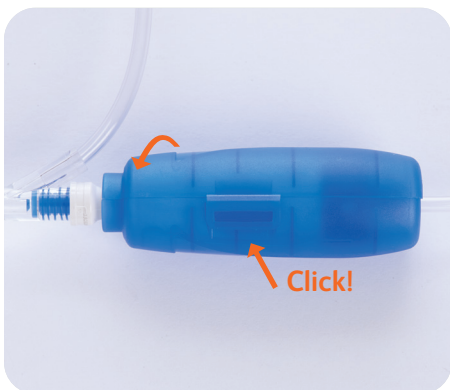

2. Close the BD PhaSeal™ Optima Infusion Clamp around the BD PhaSeal™ Optima Injector/Connector assembly. Snap the latch to close.

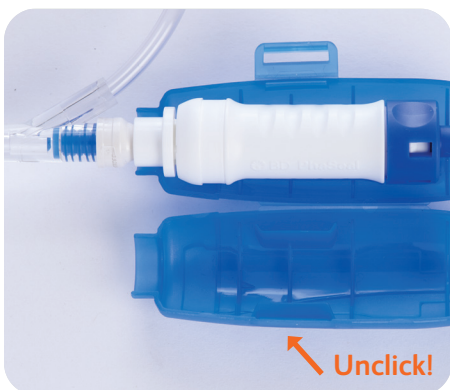

3. To open the BD PhaSeal™ Optima Infusion Clamp, apply pressure to the latch until it unclicks, then open the clamp.

Note: Infusion clamp is for single patient use only and can be used to secure injector/connector throughout patient's hospital/clinic visit. Clamp should be discarded in accordance with facility protocol after single patient use.

bd.com

BD, the BD Logo and PhaSeal are trademarks of Becton, Dickinson and Company or its affiliates.
© 2023 BD. All rights reserved. BD-16659 (07/23)

