


BD Nexiva™

Closed IV Catheter System—Dual Port with BD MaxZero™ Needle-free Connector

Tips for success

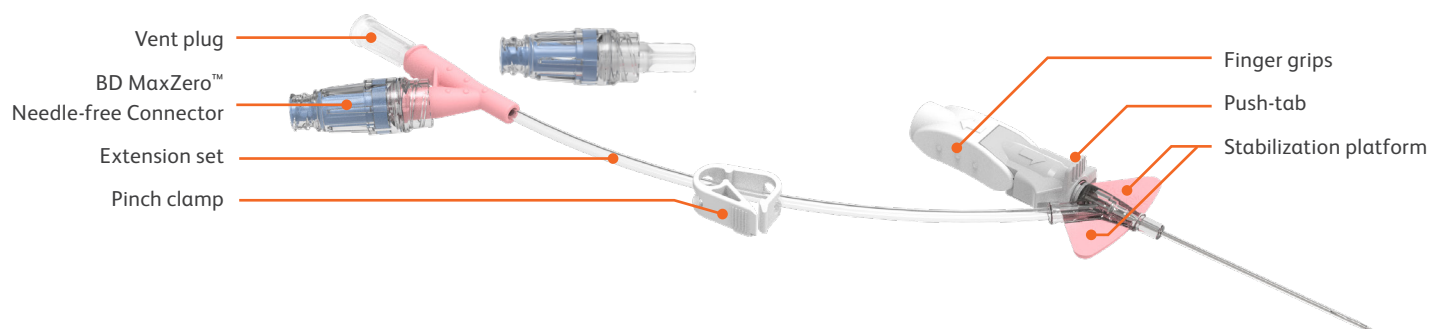

Sharp insertion:

- Keep the stabilization platform and finger grips pushed together.
- **Do not** insert holding only onto the stabilization platform as this will push the needle back into the catheter.
- Avoid holding in a manner that pushes against the stabilization platform.

Vessel access:

- Look for initial blood return in the catheter near the insertion site.
- Lower and slightly advance the entire system to ensure the catheter tip is within the vessel.
- Look for continuous blood flow along the extension tubing to confirm you are still in the vessel.

Catheter advancement:

- Push the catheter forward off the needle rather than pulling the needle back.
- Use the pad of your index finger, not your fingernail, behind the push tab.
- Push down and forward against the push tab.

Needle removal:

- Pushing down gently on the stabilization platform, pull back on the finger grips until the needle shield releases.
- **Do not** touch the push tab during needle removal.

Blood containment:

- Secure the vent plug and the BD MaxZero™ Needle-free Connector before insertion.
- Clamp the extension tubing before replacing the vent plug.

Please consults product labels and inserts for instructions for use, indications, contraindications, warnings and precautions.

BD, Sandy, UT, 84070, U.S.

[bd.com](https://www.bd.com)

