

BD SafetyGlide™ Syringe

For TB and allergy

Important usage guidelines



1. Preparation

Pull the shield straight off the needle.

Rotate the safety arm as needed to read the scale.




2. Aspiration

Draw up medication in accordance with your institution's protocol.



3. Injection

To facilitate a low-angle injection, rotate the safety arm so that it is oriented to the needle bevel. Administer medication in accordance with your institution's protocol.

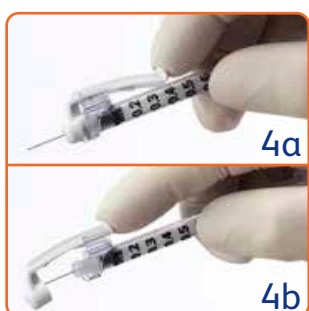

4. Activation

- 4a. After injection, apply a single finger stroke to activate the safety arm by moving it completely forward until the safety arm is fully extended and the needle tip is covered.
- 4b. Visually confirm that the safety arm has fully advanced and the needle tip is covered.

Use one-handed technique and activate away from self and others. Visually confirm that the needle tip is covered. After activation or if unable to activate, immediately discard into an approved sharps collector after single use.

Remember

- Follow your institution's protocol for aseptic technique and skin preparation.
- Follow your institution's policy for safe disposal of all medical waste.

This guide is not intended as a replacement for the detailed information contained in the IFU. Please consult BD product labels and package inserts for any indications, contraindications, hazards, warnings, cautions and instructions for use.

1 Becton Drive, Franklin Lakes, NJ, 07417, USA

[bd.com](https://www.bd.com)

BD, the BD Logo and SafetyGlide are trademarks of Becton, Dickinson and Company or its affiliates. © 2023 BD. All rights reserved. BD-1280 (05/23)

Scan this code to watch a video on how to use BD SafetyGlide™ Syringes for TB and allergy

