



November 2022

Cleaning of BD Pyxis™ Products

This document covers the updated cleaning agent guidelines for the following BD Pyxis system products: BD Pyxis™ MedStation®, BD Pyxis™ Anesthesia Station, BD Pyxis™ Supply Technologies, and BD Pyxis™ CII Safe.

Acceptable cleaning agents and hand sanitizers are listed in the tables below. BD recommends immediately discontinuing use of any agents not included.

Standard Cleaning Agents and Hand Sanitizers

Cleaning Agents	Dilution
10% Bleach	1 part bleach to 9 parts water
70% isopropyl alcohol (IPA)	No dilution required
Aseptizyme	1 oz/gallon water
Clorox Wipes	Not applicable
Clorox Healthcare© Bleach Germicidal Cleaner	No dilution required
Hibiclens	25.6 oz/gallon water
LpH Disinfectant Cleaner	0.5 oz/gallon water
Maxima 128	1 oz/gallon water
Metrizyme	1 oz/gallon water
Mild detergent	Per manufacturer's recommendation, as needed
Super Sani-Cloth	Not applicable
Thymocide	Not applicable
Virkon	Per manufacturer's recommendation
Warm water	Not applicable
Wexcide	1 oz/gallon water
Wexcide-Ready-To-Use	No dilution required

The following hand sanitizers are acceptable for use when working with the equipment. The equipment will not be damaged if excess sanitizer is transferred to the equipment during use. **Note:** Do not use the following hand sanitizers to clean the device.

Hand sanitizers	
Purell w/ 62% Alcohol	No dilution required
3M Avagard D	No dilution required
EcoLab Quik-Care	No dilution required

For safety information about the BD Pyxis™ system not covered in this document, please call the Technical Support Center for BD Pyxis™ Products at 1.800.727.6102 or 858.617.2000.

Manufactured by CareFusion 303, Inc. 10020 Pacific Mesa Blvd. San Diego, CA 92121 United States

©2022 BD. BD, the BD Logo and Pyxis are trademarks of Becton, Dickinson and Company. All other trademarks are the property of their respective owners. BD-66532