

Implementing BD Pyxis™ IV Prep

Katie Cimmino, PharmD, BCSCP
Senior Pharmacist
Supervisor of IV Compounding and Research
NewYork-Presbyterian Queens

June 18, 2020


Speaker

Katie Cimmino, PharmD, BCSCP
Senior Pharmacist
Supervisor of IV Compounding and Research
NewYork-Presbyterian Queens

Disclosure: The data, views and claims within this presentation are that of the presenter.
No portion of this content may be reproduced without explicit written permission.

This program is sponsored by BD.

Agenda

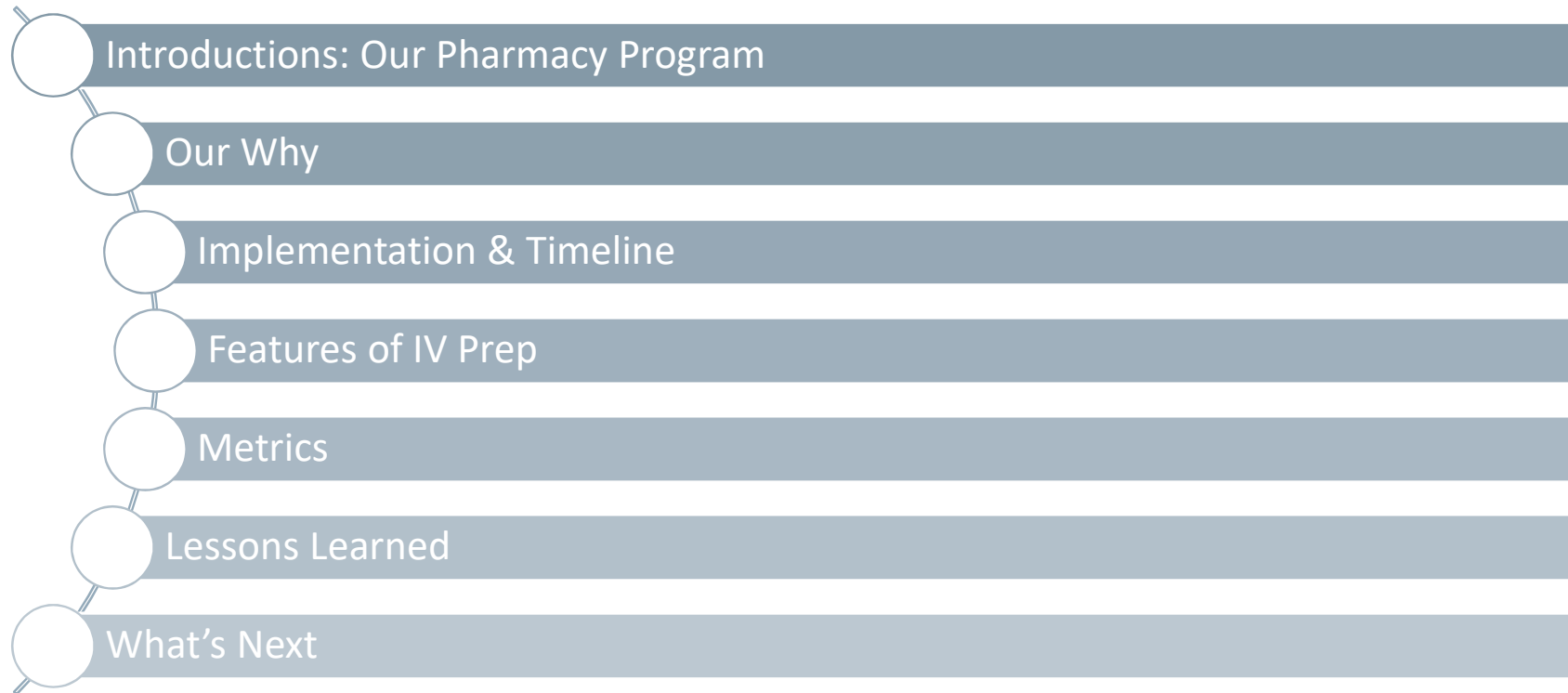

NewYork-Presbyterian Queens


- 535-Bed Acute Care Facility
- Community hospital
- Affiliated with Weill Cornell medicine



NYPQ Pharmacy

- Pharmacy provides services 24/7
- Pharmacy staff includes 69.8 FTE (with per diems have 90 staff members)
 - All staff members: understanding of USP <797> and training in USP <800>
 - 60 staff members: understanding/ training clean room practices
 - 40 staff members: compounding competency
 - 25 staff members: hazardous drug compounding competency
- Highly automated
 - 90% of doses are dispensed utilizing technology
 - Robots/ automatic dispensing cabinets, IV workflow system
- Over 320,000 IV doses prepared yearly
 - 10% of total doses dispensed
 - Includes pre-mixed, point of care, and compounded items
 - Compounded items account for 1/3 of IV doses prepared

NYPQ Pharmacy Operations & Strategy

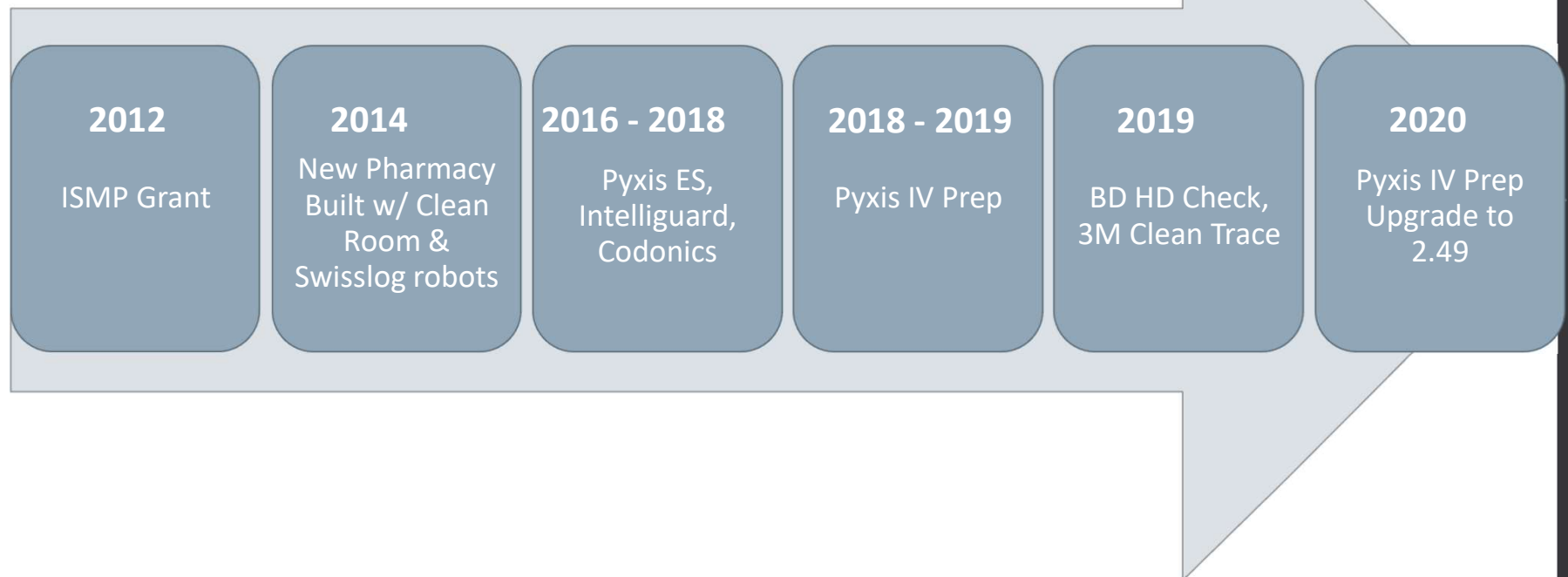

Journey to Zero Harm: For patients and employees

Implementation of
technology for patient safety
& employee confidence

BD Pyxis IV Prep helps us close the
gap in barcode scanning
compliance & uses gravimetrics



Why Implement an IV Workflow Management System?



- Best practice recommended by ISMP¹
 - Studies show manual compounding can have error rates of up to 9%²
- Standardizes process and documentation
 - Integrates barcode scanning, image capture, gravimetric analysis
- Pyxis IV Prep provides metrics and performance insights for IV workflow efficiencies

Why implement BD IV Prep?

Safety: hard stops during compounding to correct potential errors


Technology: gravimetric verification, barcode scanning, image capture

Documentation: audit trail of compounding process

Inventory Management: remainders, stock batch, returns

Compliance: USP 797 and USP 800



Implementation Timeline

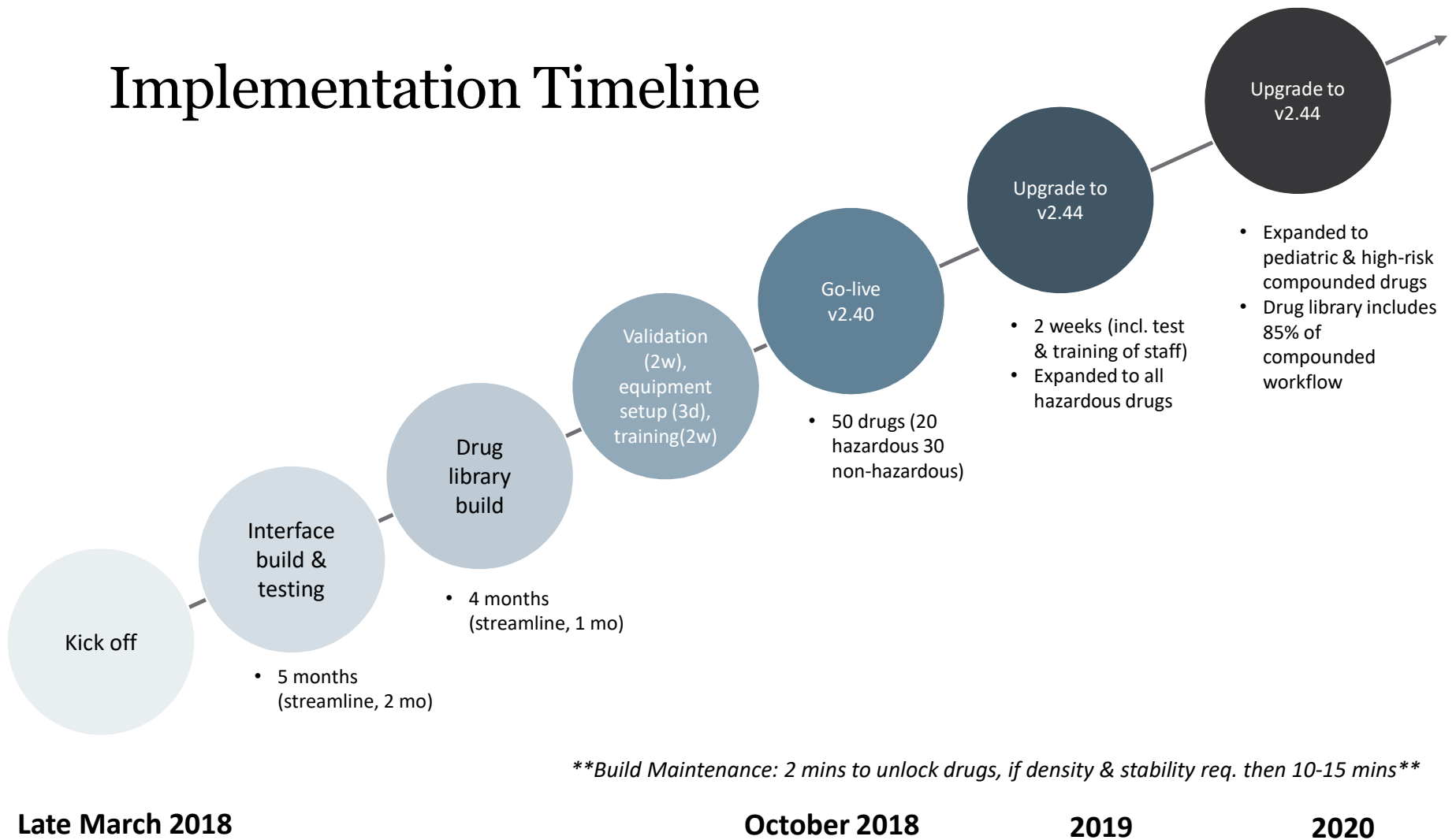

Compounding



Hazardous Compounding

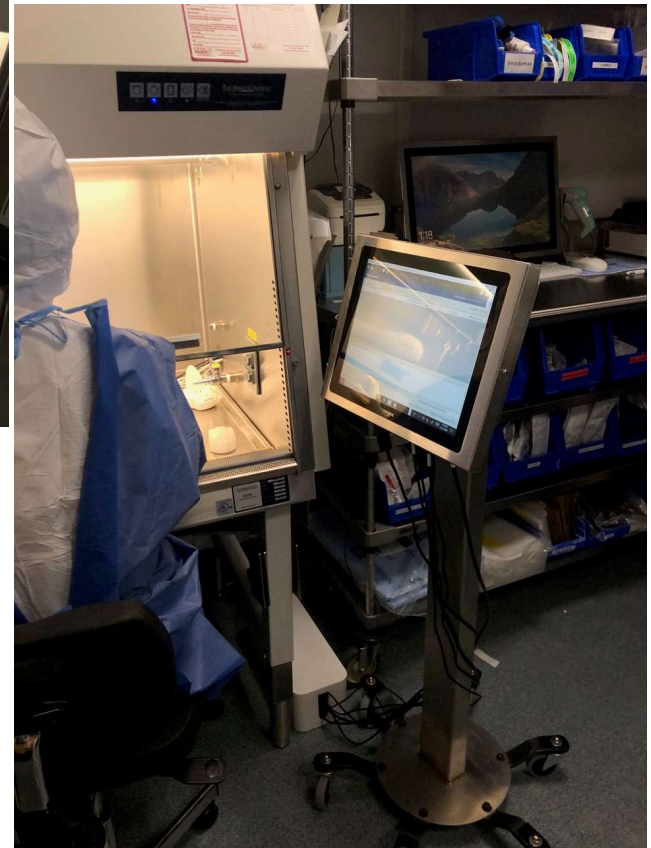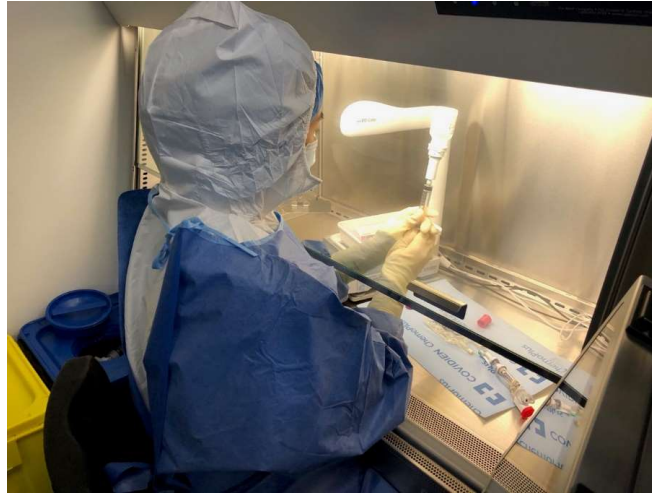

IV Prep Example

1 - Transfer of thiamine injection 100mg



Check of container after injection

Action within tolerance

Container: dextrose 5% 50mL bag VIAFLEX Baxter

Active ingredient quantity to transfer: 95.05mg

Nominal: 0.95mL; Actual: 0.96mL; Deviation: +1%

Label Set Up

One Step

NewYork-Presbyterian Queens Pharmacy
MCKESSON, PATIENT15 ED - Trauma T01B-
 DOB: 6/30/1971 47 y MRN: 3368615
 LATEX ALLERGY
valproate sodium 1750 mg
dextrose 5% 50 mL

Freq: Daily Route: IV PiggyBack
 Infuse over: 60 min Rate: 67.5 mL/hr



This barcode is for pharmacy use only

Hang By: 10/21/18 09:00

"Hang By" refers to the point at which drug administration must begin

Storage: Room Temp.

Prep By: 10/20/18 0900
 Checked By: _____
 Order ID: M001BBN963



Patient specific barcode used for EMR documentation

This corresponds to the administration time in EMR

Two Step Label


NewYork-Presbyterian Queens Pharmacy
[REDACTED] Chemo Clinic - **[REDACTED]**
CARBOplatin 358 mg
sodium chloride 0.9% 250 mL
 Freq: Once Total Volume: 285.80 mL
 Infuse over: 30 min Route: IV PiggyBack
HAZARDOUS - ANTINEOPLASTIC
 Rate: 571.6 mL/hr
Hang By: 06/02/20 10:15
 RX USE ONLY: 154500

Storage: Room Temp.

NOT CHECKED- DO NOT ADMINISTER

Storage: Room Temp.
 Prep By: JaiKoc RPh 06/01/20 1015 Rx Deliver By: 06/01/20 09:42
 Checked By: n032 RPh Order ID: M001FFJPZR
CAUTION CHEMOTHERAPY DRUG | Hazardous
Antineoplastic Drug | Group A: Hazardous-Antineoplastic

Emergency Management with IV Prep

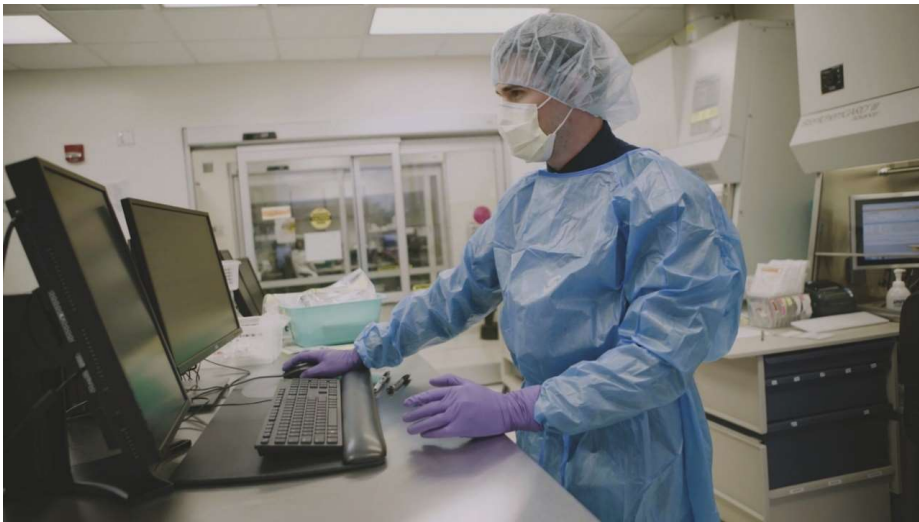

- Out of range room certifications
 - IV Prep quickly allows you to set global settings if beyond use date must quickly change
- Clinical Trial
 - NYPQ remdesivir site under emergency use authorization supply
- Management of important medication usage


Implementing now: Pyxis IV Prep & USP 797 task tracking

New task management function with *workflow hard stops* enforce compliance & documentation of tasks

- Recording USP compliant related tasks such as:
 - Environmental activities (pressure, temperature, humidity)
 - Cleaning activities
 - Room Certifications
 - Media fill/ fingertip sampling (competency checklists)
- Can notify manger/ supervisor of overdue tasks
- Reporting elements

Expected Outcomes:

- Actual documentation of completed tasks
- Dashboard of USP compliance across system
- Standardizes tasks
- Competency checklist
- Increase USP Compliance

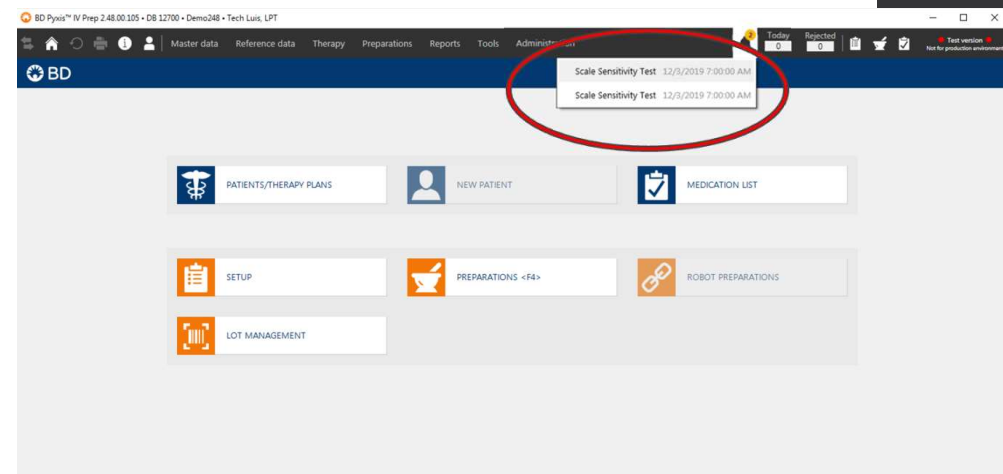


Benefits of Pyxis IV Prep - Data




Big Picture Metrics


Average number of preparations made per day

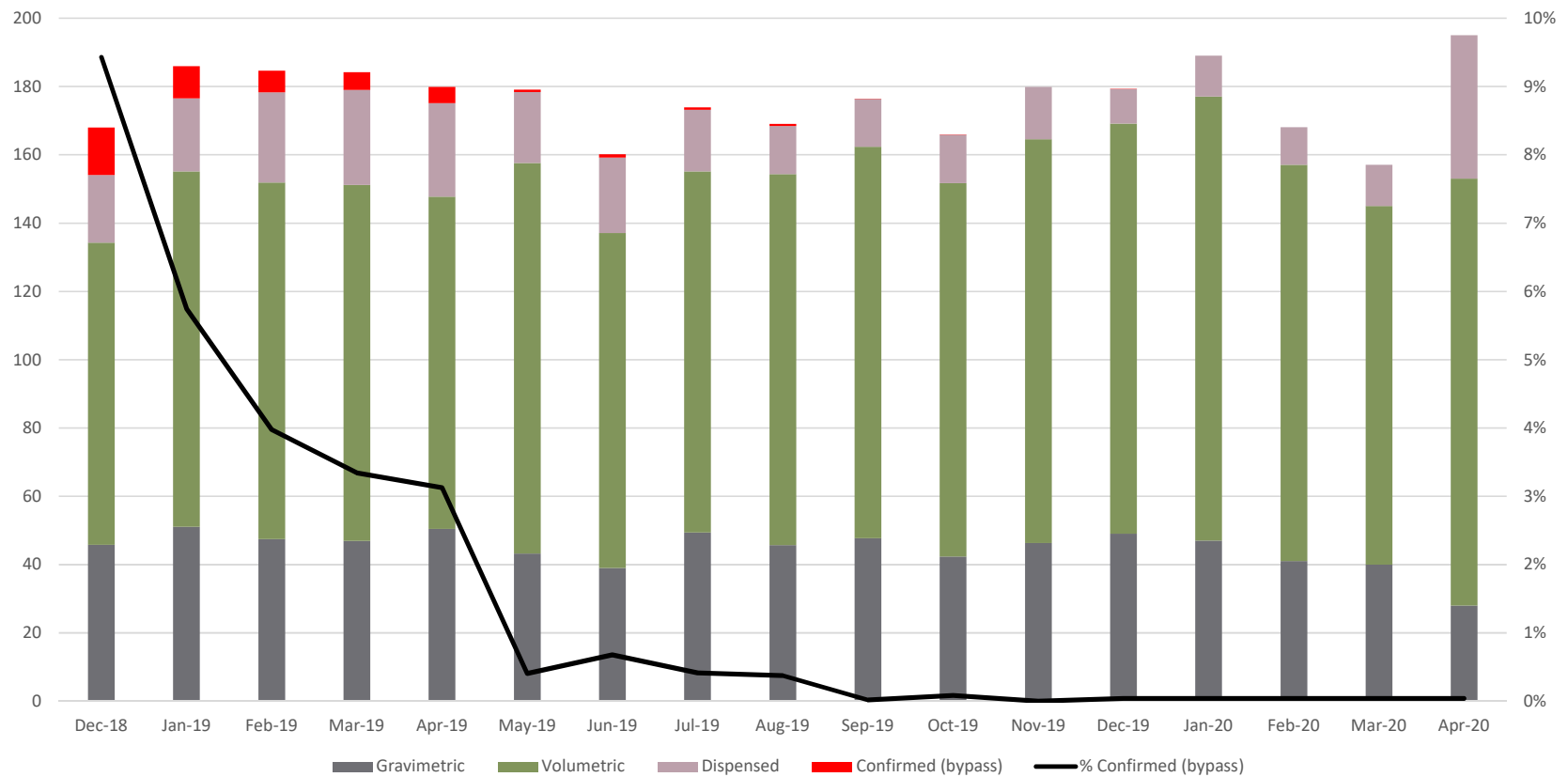

Who is becoming a natural super user?

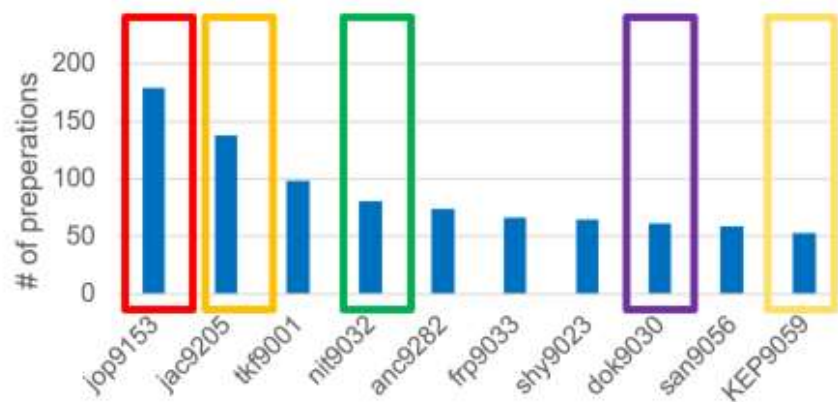


Who is utilizing the system?

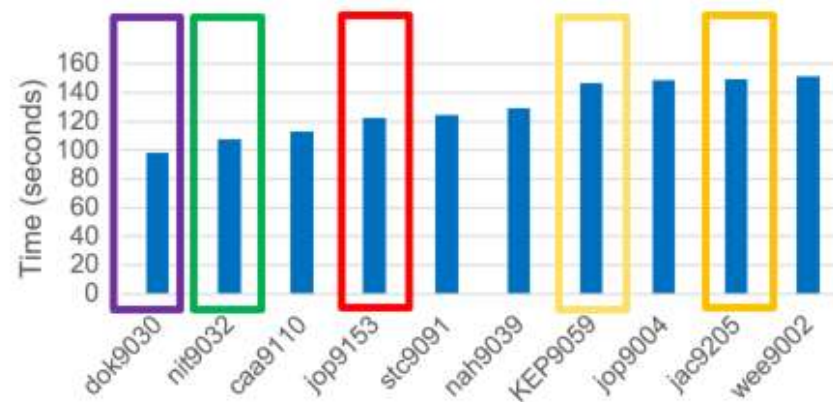


Who is compounding quickly?

Metrics by Drug Level

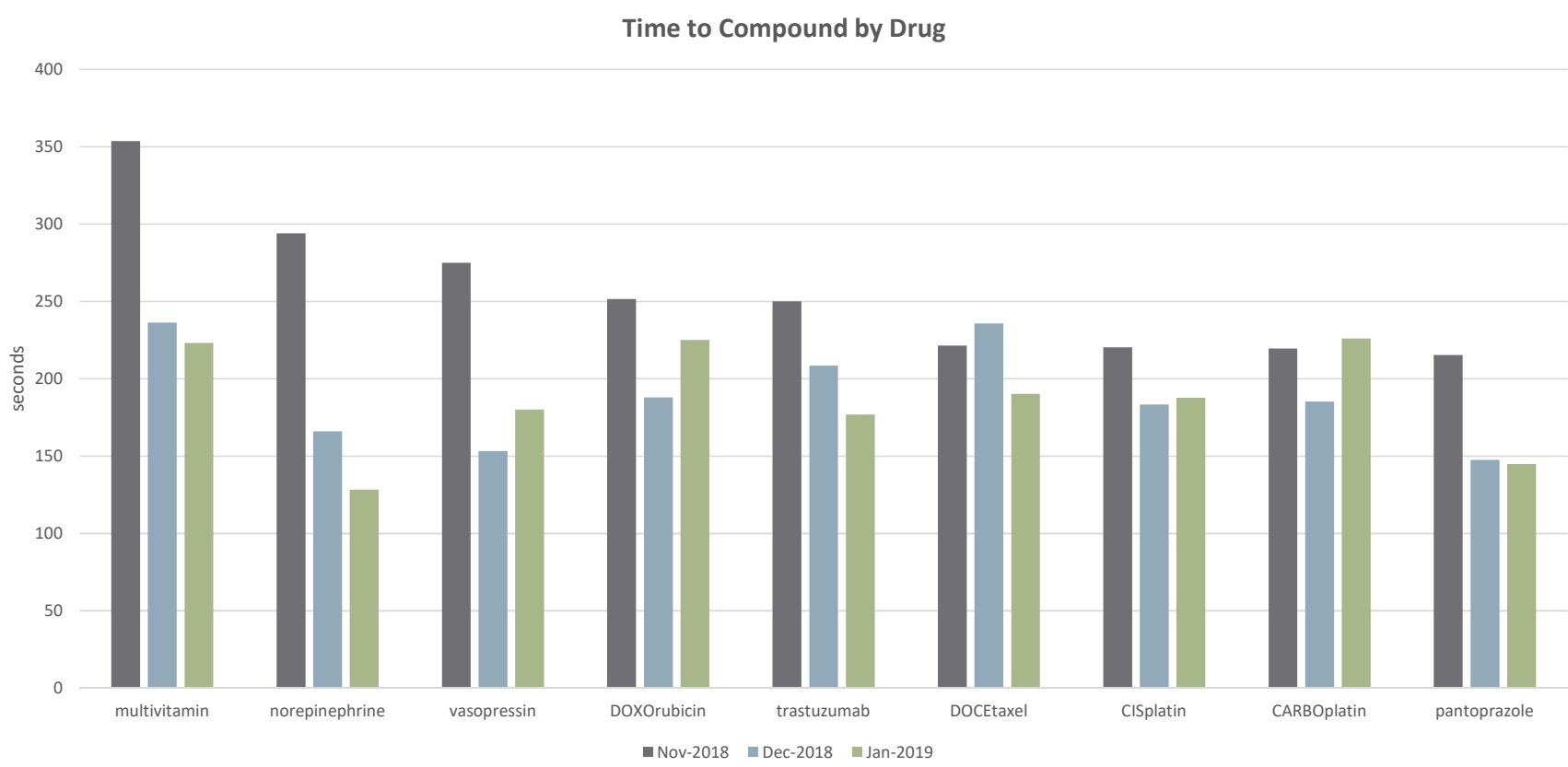


Can workflow be adjusted to optimize system?

November 2018 to April 2019

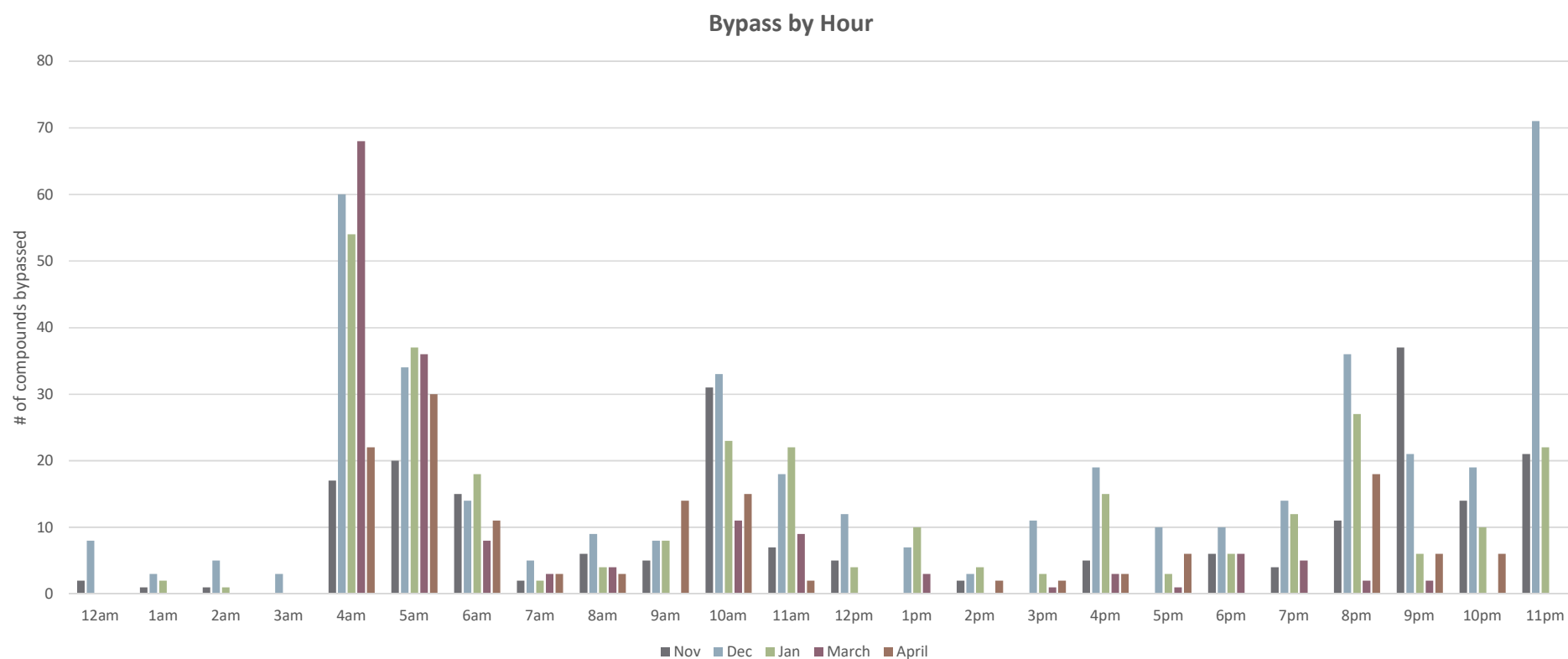


Percent Bypass by Drug (Dec 2018)

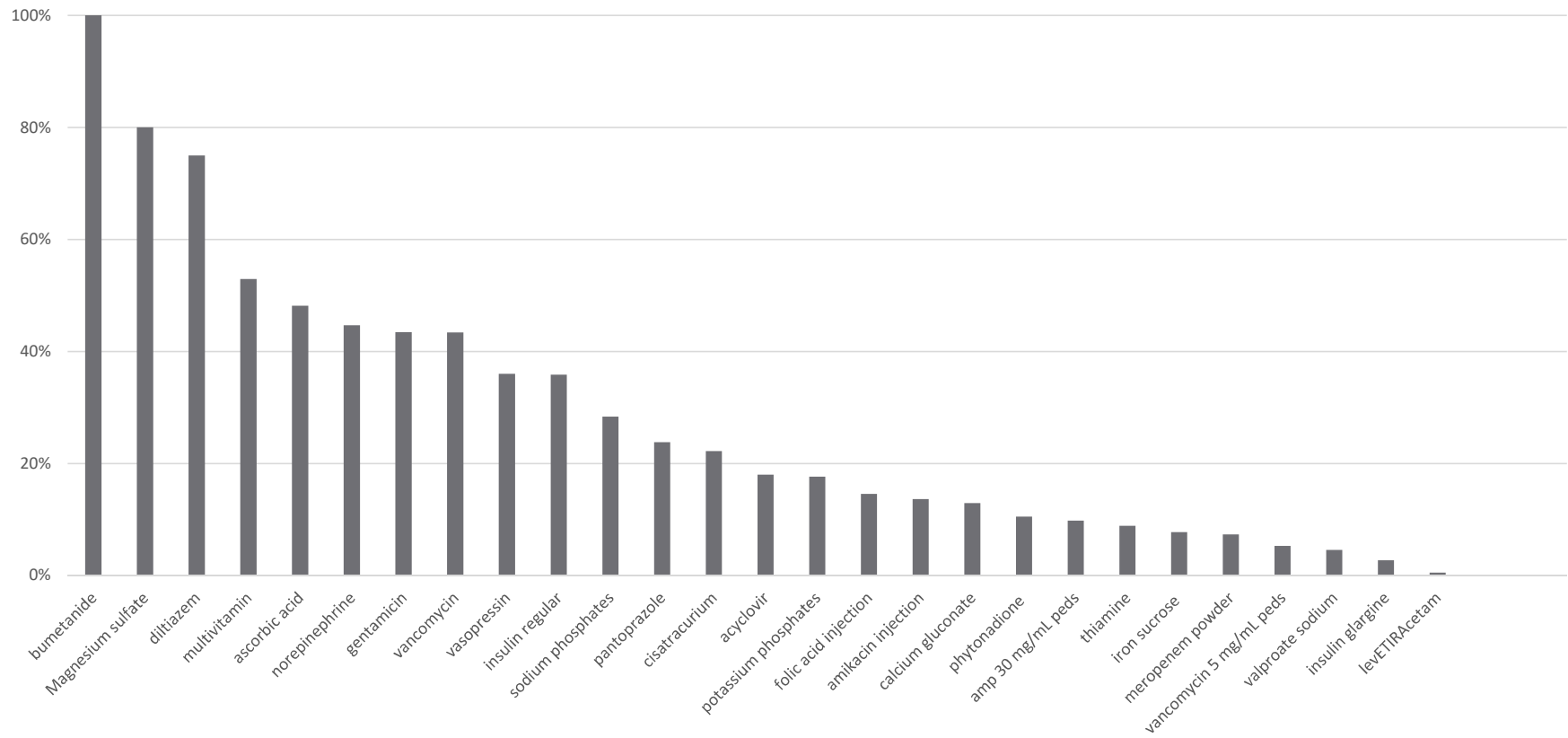

Productivity

➤ From initial go-live

- Time spent compounding has decreased (**average 132 seconds per gravimetric compound for 2019**)
- Accuracy of gravimetric compounds has increased
 - **Review of all gravimetric preparations for time period showed an average dose deviation of 0.50% per compound.**

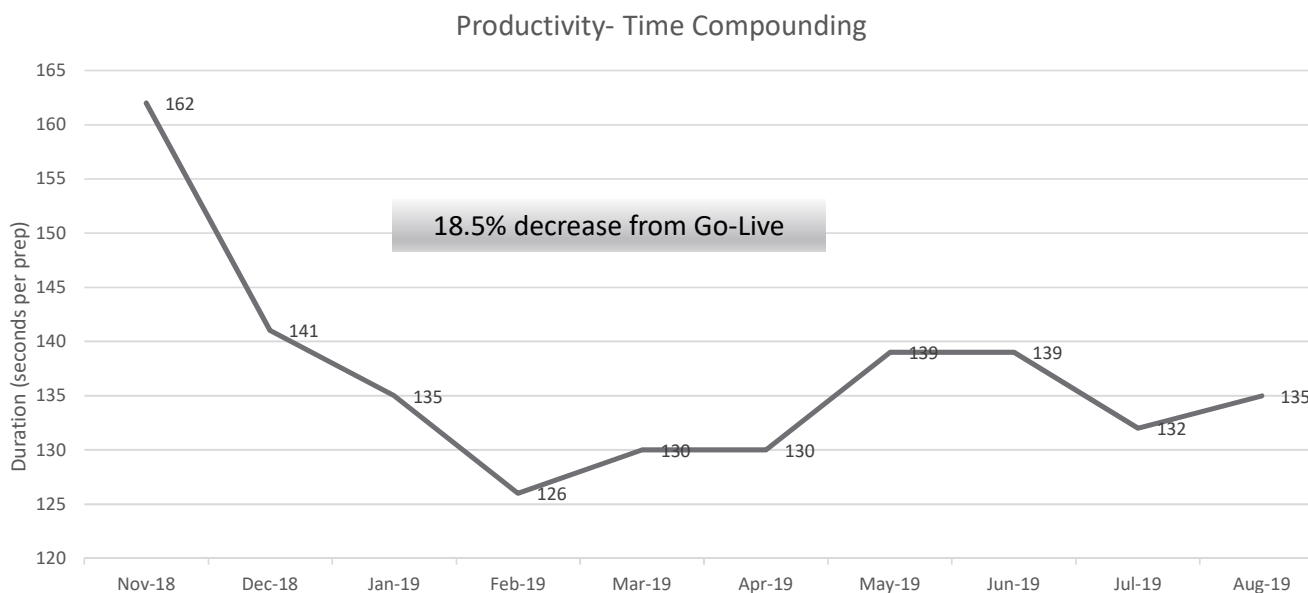

Streamlining workflow


➤ Streamlined pharmacy workflow and thus reduced pharmacy compounding waste

- 2019- Average \$3,500 waste reduction per month
- Saved at-least 25 minutes compounding time per day
- Downward trend in cost-savings is due to the fact that metrics have allowed us to change workflow to reduce overall waste



Safety Features

- In 2019, out of 62,265 preparations IVWMS detected and stopped 7.5% errors.
 - This is lower than the national average estimate of 9%²
- Of those errors, only 21.9% could have been detected prior to IVWMS.
 - Not all errors have significant clinical impact

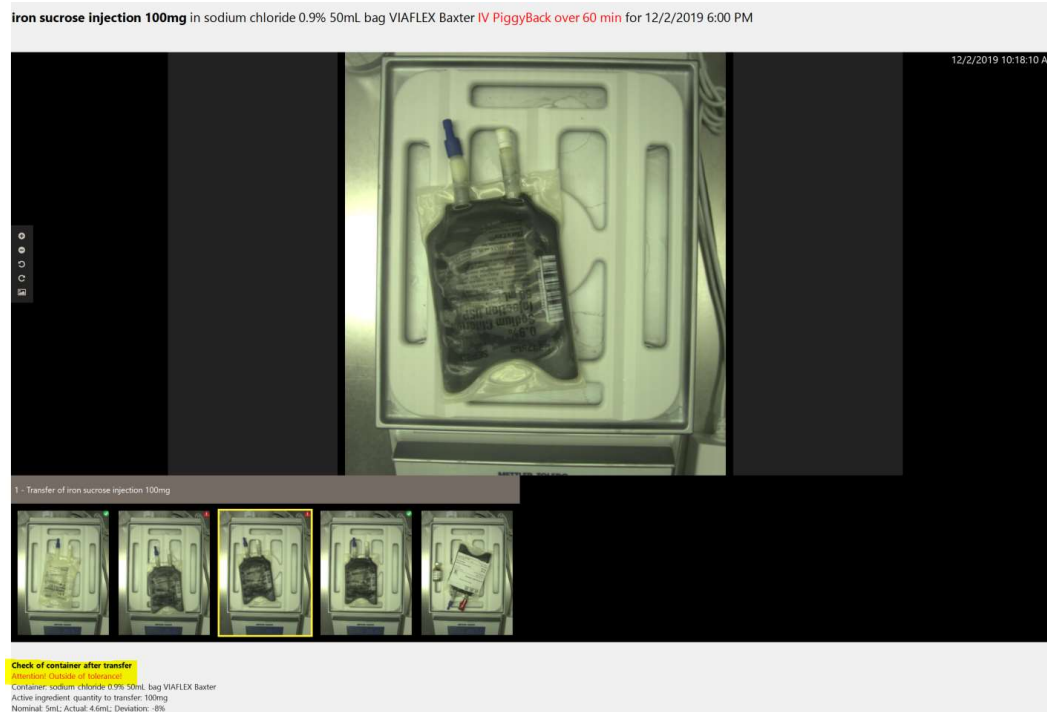

Key takeaways on how to develop metrics

➤ Post Go-Live examine metrics:


- by user-compounding speed, compounding accuracy and bypass rate
- by product-speed, accuracy, and bypass rate

➤ Continuous Metrics


- Begin to set department goals as a whole
- If you are not meeting those metrics see if it is due workflow or users
- Create goals/ metrics surrounding productivity, safety, and cost-savings

Share metrics data with staff to reinforce the positives of the error reduction!

Lessons Learned

- 
- Rollout implementation vs. Big Bang
 - Soft go-live with equipment
 - Getting the right equipment
 - Importance of Super Users
 - Continuous education beyond Pharmacy & sharing our success internally

What's next for NYPQ Pharmacy

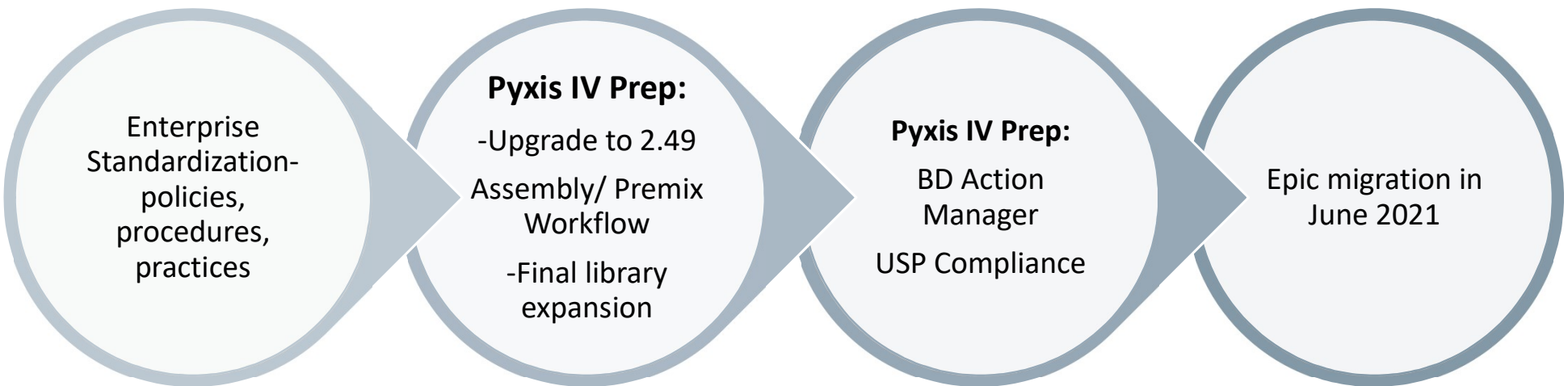

References

1. ISMP. Maximize benefits of IV workflow management systems by addressing workarounds and errors. 2017 Sept. <https://www.ismp.org/resources/maximize-benefits-iv-workflow-management-systems-addressing-workarounds-and-errors>
2. Flynn, EA, Pearson, RE, Barker, KN. Observational study of accuracy in compounding IV admixtures at five hospitals. *Am J Health-Syst Pharm*. 1997 Apr 15; 54: 904–912.

Thank you!

Replay: BD.com/BDInstitute

BD, the BD Logo, BD HealthSight, and the BD HealthSight Logo are trademarks of Becton, Dickinson and Company. © 2020 BD and its subsidiaries. All rights reserved.

