

Interrupting the power down sequence on the BD Alaris™ PC Unit

Quick start guide

To power down the BD Alaris™ PC Unit:


Press and hold the **CHANNEL OFF** key on each attached module. Do not release it until you hear a beep (approximately 1.5 seconds).

Another method: Press the **OPTIONS** key on the PC unit and then press **Power Down All Channels**.

During the power off sequence, the main display flashes **Powering Down**.

To interrupt the power down sequence, quickly press any key (except *SYSTEM ON*) on the PC unit.

Note: This places the PC unit in an idle state for the same patient it was previously used on. The previous patient's parameters, such as weight, will remain stored on the device.

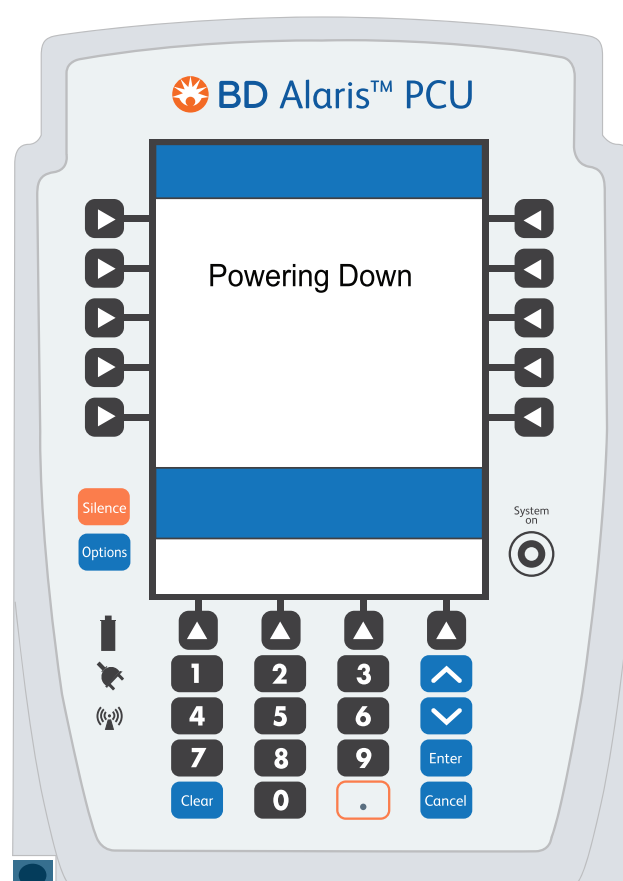

If the unit needs to be used on a new patient, completely power down the PC unit and restart it by pressing the **SYSTEM ON** key.

To start programming for a new patient: Press **Yes** at the **NEW PATIENT** prompt.

For product support, contact Customer Advocacy at **888.812.3266** or customerfeedback@bd.com

For technical support, contact Instrument Technical Support at **866.488.1408**.

For product orders, contact Customer Order Management at **800.482.4822**.

⚠ See reverse side for applicable warnings and cautions.

📖 For complete instructions, refer to the BD Alaris™ System User Manual at bd.com

Interrupting the power down sequence on the BD Alaris™ PC Unit

Warnings and cautions

WARNING: The BD Alaris™ PC unit performs a **self check during power up**. The PC unit should beep, no errors should occur, and if a module is connected all LED segments should flash. If the BD Alaris™ PC unit fails the self check, remove the failing PC unit or module from use.

