



# PleurX™ Catheter System app

Quick start guide




Scan the code to download the app.  
Available in the Google Play Store  
and the Apple App Store.



# Add an entry




Add a drainage entry by tapping the floating **+** button.

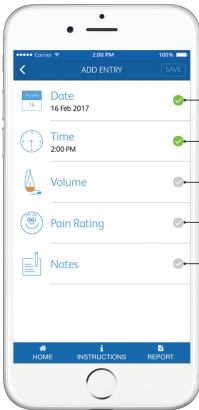

The date and the time are prepopulated. Edit them by tapping **Date** or **Time**.


Add the drainage volume by tapping **Volume**.

A pain rating and notes are not mandatory, but additional information can help you and your physician with your drainage data.

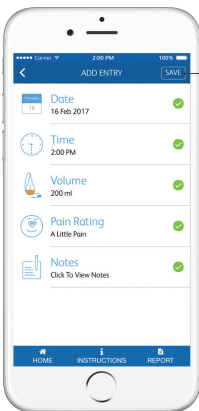

Save an entry by tapping the **SAVE** button.

# View logs

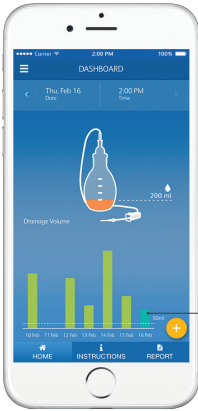

View drainage details by tapping the entry on the bar graph.




Toggle between the **Drainage Volume** graph and **Pain Rating** graph.

View drainage data by swiping left and right.


View drainage data in detail by scrolling up or down.



Edit the entry by tapping it.

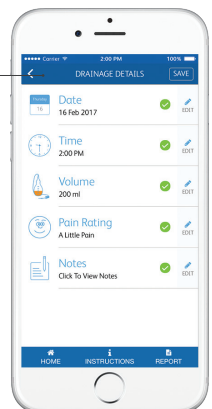

# Instructions

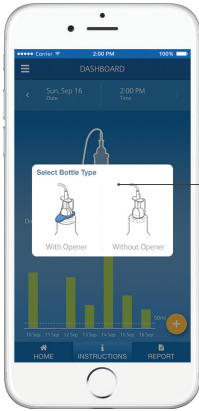


Tap **INSTRUCTIONS** and select your bottle type.

**Bottle with opener**




**Bottle without opener**

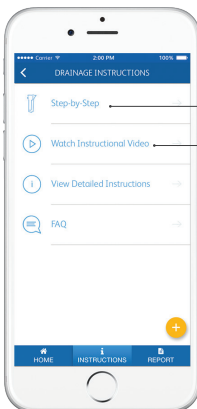



View step-by-step instructions on drainage by tapping on **Step-by-Step**. You may use the **Next** and **Back** voice commands to navigate between pages.

View drainage procedure instructional videos by tapping on **Watch Instructional Video**.


# Set drainage schedule

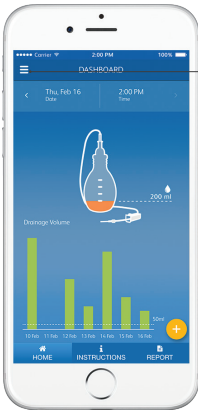

Open the menu and tap **View Calendar**.

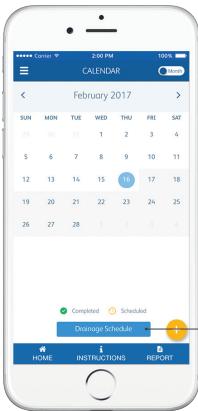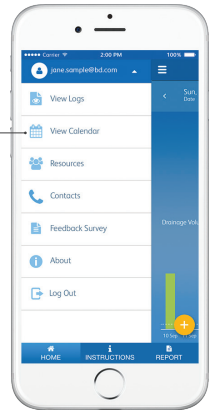


Tap **Drainage Schedule**.

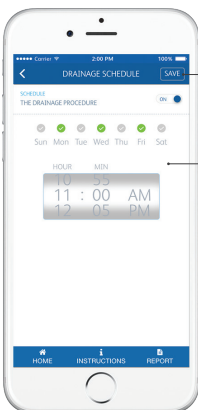

Set a drainage schedule.

Tap the **SAVE** button when you are finished.



# Send a report

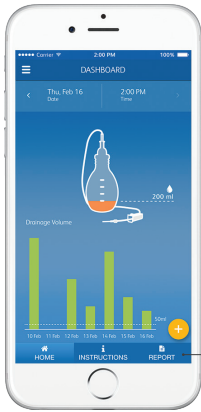


Tap the **REPORT** button.

Select the report type.

Tap the **SEND** button.

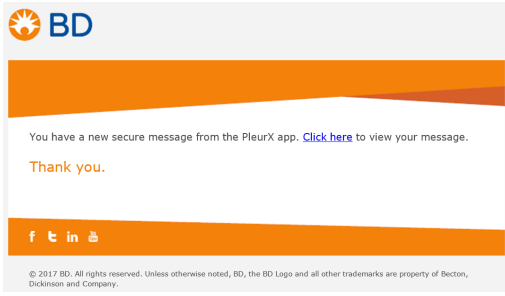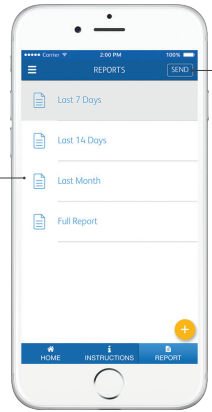

You will receive an email from **donotreply@bd.com** with your drainage report. Download the report from the email.

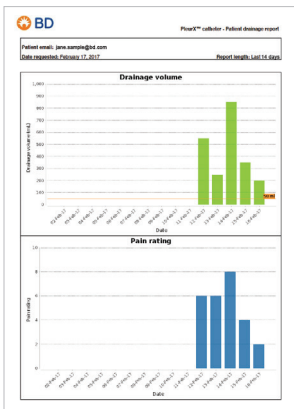

**Notes for physician**

12-Feb-17 Much less pain than before. Progress  
 15-Feb-17 Feeding much better due to regular draining

**Drainage details**

Date	Fluid volume drained (mL)	Pain rating	Where was the drainage performed (ambulatory, inpatient)	Who performed the drainage procedure?	How long did the drainage procedure take?
15-Feb-17	300	8	Outpatient	Self	20-30 minutes
15-Feb-17	350	6	Inpatient	Self	20-30 minutes
15-Feb-17	200	6	Inpatient	Self	20-30 minutes
16-Feb-17	350	8	Hospital	Centenar	20-40 minutes
16-Feb-17	300	4	Hospital	Family or friend	20-30 minutes
16-Feb-17	200	2			

All data used to generate this report was entered by the user through the PleurX App.

You may print the drainage report and bring it to your next appointment.





Scan the code to download the app.

Available in the Google Play Store and the Apple App Store.



Questions about the PleurX™ Catheter System?

Call a PleurX™ Patient Navigator at **833.549.7677**.

BD, Vernon Hills, IL, 60061, U.S.

[bd.com](http://bd.com)



BD, the BD Logo and PleurX are trademarks of Becton, Dickinson and Company or its affiliates. © 2019 BD. All rights reserved. PPA190504 (0919)