



BD[®] Surgiphor[™] Antimicrobial Irrigation System: The only pre-mixed, terminally sterile dilute povidone-iodine (PVP-I) wound irrigation system

Surgiphor[™] Irrigation System may help to reduce reliance on IV solutions used for irrigation purposes*

Indication: Surgiphor[™] Antimicrobial Irrigation System is intended to mechanically loosen and remove debris, and foreign materials, including microorganisms, from wounds.

Facilities may consider Surgiphor[™] Irrigation System may be an alternative in certain processes where you currently utilize IV Fluids, such as:

- Irrigation mixing in the OR (back table) of PVP-I or antibiotics
 - Supports reduced fluid usage and fluid inventory consolidation
- Pharmacy mixing of irrigation solutions
 - Ready-to-use solution can reduce compounding workload and support conservation of fluids
- Pulsatile lavage with IV solutions
 - May offer an alternative to pulsed lavage, if appropriate

The BD Surgiphor[™] Antimicrobial Irrigation System is not on allocation.

For more information on the Surgiphor[™] Irrigation System, please visit go.bd.com/Surgiphor or contact your BD Surgical Solutions Sales Representative

*For Irrigation only.

Product description: Surgiphor[™] Antimicrobial Irrigation System is an antimicrobial irrigation system containing 0.5% povidone iodine (PVP-I) in phosphate-buffered saline, potassium iodide and Vitamin E TPGS. PVP-I acts as a preservative to help inhibit microbial growth in the irrigation solution. **Indication for use:** BD Surgiphor[™] Antimicrobial Irrigation System is intended to mechanically loosen and remove debris, and foreign materials, including microorganisms, from wounds. **Contraindications:** BD Surgiphor[™] Antimicrobial Irrigation System should not be used in patients with known allergic reaction to any of the ingredients in the solutions. BD Surgiphor[™] Antimicrobial Irrigation System should also not be combined with other irrigation or antiseptic solutions due to potential reactions and reduction in the effectiveness of the system. Not for use in neonates. **Warnings:** Do not use or mix with other cleansers, soaps, lotions, or ointments. Do not use for injection or infusion. Do not swallow. Do not use in eyes or ear canals. Discontinue use immediately if irritation or an allergic reaction occurs. Do not use if packaging is damaged or if seal integrity is compromised. Do not reuse BD Surgiphor[™] solution after 24 hours. **Precautions:** BD Surgiphor[™] solution may cause a temporary irritation and/or burning sensation on exposed skin in very rare cases. BD Surgiphor[™] solution may cause allergic reactions such as rash or skin irritation in patients with iodine allergy. Anaphylaxis with the use of BD Surgiphor[™] solution may occur in patients with severe iodine allergy. Federal law restricts this device to sale by or on the order of a licensed physician. Single patient use only. Not for at-home use. Please consult product insert for complete indications, contraindications, warnings, precautions, safety information and instructions for use.



Distributed by BD, 1 Becton Drive, Franklin Lakes, NJ 07417

© 2024 BD. All rights reserved. Unless otherwise noted, BD, the BD Logo and all other trademarks are the property of Becton, Dickinson and Company or its affiliates. [Terms of Use](#) | [Privacy Policy](#)