REF 50-7245A

Lockable Drainage Line

For use with the PleurX™ or PeritX™ Catheter Systems

- en** Lockable Drainage Line
For use with the PleurX™ or PeritX™ Catheter Systems
- fr** Ligne de drainage verrouillable
À utiliser avec les systèmes de cathéter PleurX™ ou PeritX™
- de** Arretierbarer Drainageschlauch
Zur Verwendung mit den Kathetersystemen PleurX™ oder PeritX™
- it** Tubo di drenaggio bloccabile
Per l'uso con cateteri PleurX™ o PeritX™
- es** Tubo de drenaje bloqueable
Para su uso con los sistemas de catéteres PleurX™ o PeritX™
- pt** Tubo de drenagem bloqueável
Para utilização com os sistemas de cateter PleurX™ ou PeritX™
- nl** Vergrendelbare drainageslijn
Voor gebruik met PleurX™ of PeritX™ kathetersystemen
- sv** Låsbar dränageslang
För användning med katetersystemen PleurX™ och PeritX™
- fi** Lukittava dreniletku
Käytettäväksi PleurX™ tai PeritX™ katetrijärjestelmien kanssa
- no** Låsbar drenasjeslange
For bruk sammen med PleurX™ eller PeritX™ katetersystemene
- da** Aflåselig drænslange
Til brug sammen med PleurX™ eller PeritX™ katetersystemer
- el** Γραμμή παροχέτευσης με δυνατότητα ασφάλισης
Για χρήση με τα συστήματα Καθετήρα PleurX™ ή Καθετήρα PeritX™
- tr** Kilitlenebilir Drenaj Hattı
PleurX™ veya PeritX™ Kateter Sistemleri ile kullanılmak üzere tasarlanmıştır

CareFusion

 75 North Fairway Drive
Vernon Hills, IL 60061 USA

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CE
0123

Made in Dominican Republic by
CareFusion DR 203 Ltd.

Becton Dickinson Pty Ltd.
Level 5, 66 Waterloo Road
Macquarie Park, NSW 2113, Australia

Becton Dickinson Limited
14B George Bourke Drive
Mt. Wellington, Auckland, 1060 New Zealand
362-24427 • 2020-11

© 2020 CareFusion Corporation or one of its affiliates.
All rights reserved.

For product inquiries: bd.com/drainage
PleurX, PeritX, CareFusion and the CareFusion logo are
trademarks or registered trademarks of CareFusion
Corporation, or one of its affiliates.

STERILE EO

Not made
with natural
rubber latex.

USA Rx Only

For Single
Use Only

Do not use if
package is damaged.

DEHP
Contains
phthalates

 **CareFusion**

en

Table of Contents

Product Information	1
How to Drain Using the Lockable Drainage Line	2

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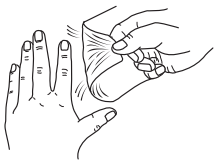
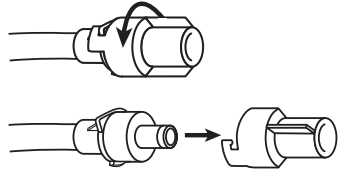
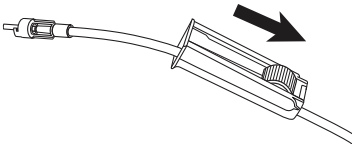
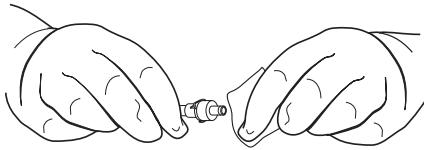
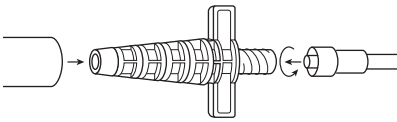
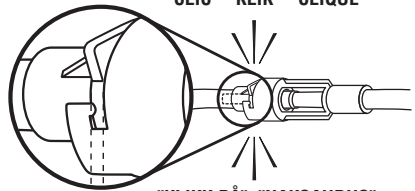
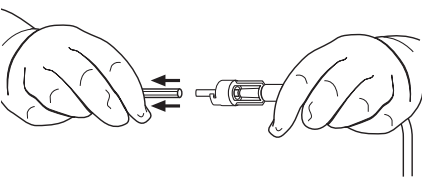
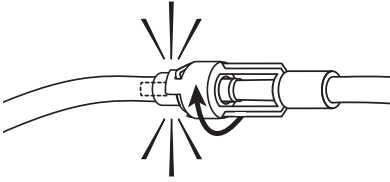
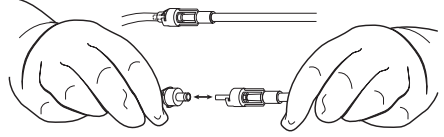
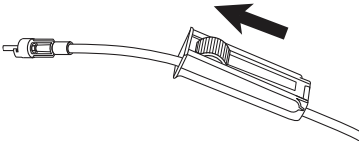
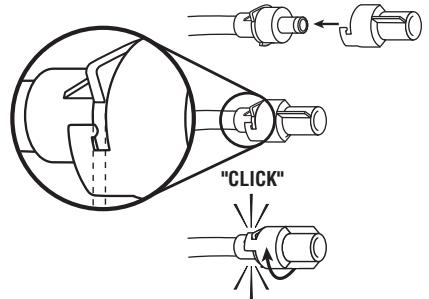
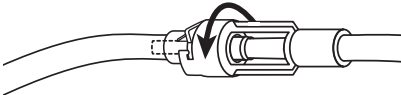
Lockable Drainage Line Procedure

Product Information

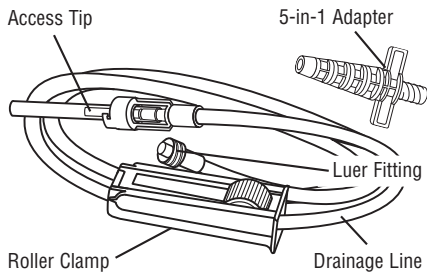
The Lockable Drainage Line is designed to be used exclusively with the PleurX™ Pleural or PeritX™ Peritoneal Catheter for drainage of the pleural or peritoneal cavity, respectively. (Figure 1)

Read this entire pamphlet carefully before draining fluid.

Indications for Use

The Lockable Drainage Line is indicated for use only with the PleurX™ Catheter and PeritX™ Catheter for intermittent drainage. The Lockable Drainage Line is used to drain fluid using standard wall suction, water seal drainage system, glass vacuum bottle, or other appropriate method (portable suction).

Lockable Drainage Line

(Figure 1)

Warnings

The patient should not be left unattended during drainage.

Cautions

For single use only. Re-use may result in a nonfunctional product or contribute to cross contamination.

Contains Phthalates. The benefit of treatment outweighs the remote possibility of exposure to phthalates.

Chest: Potential complications of draining the pleural space include, but may not be limited to, pneumothorax, re-expansion pulmonary edema, hypotension, circulatory collapse and infection.

Abdomen: Potential complications of draining fluid from the abdomen include, but may not be limited to, hypotension, circulatory collapse, electrolyte imbalance, protein depletion, ascites leakage, peritonitis, wound infection, intraperitoneal adhesion, and loculations of the peritoneal space.

If fluid spills, use soap and water to clean skin and use appropriate cleaning agent for all other surfaces.

Contact the patient's physician if:

- The patient develops fever (body temperature above 100.5°F (38°C), or you notice any redness, swelling, oozing or pain at the exit site.
- These may be signs of infection that may require treatment.
- Shortness of breath is not relieved after draining 1000 ml from the chest or 2000 ml from the abdomen at one time.
- The patient continues to experience symptoms, but little or no fluid drains from the catheter.
- Less than 50 ml drainage in three (3) drainage procedures in a row.
- The appearance (color, thickness, etc.) of the fluid changes significantly between drainages.

Sterility

This product has been sterilized and is for single use only. Do not re-sterilize. CareFusion will not be responsible for any product that is re-sterilized, nor accept for credit or exchange any product that has been opened but not used.

Do not use if package is damaged.

Drainage Line Supplies

1 - Drainage Line with Lockable Access Tip

1 - 5-in-1 Drainage Line Adapter

Note: -60 cm H₂O = -1.7 in Hg = -44 mm Hg = -0.8 psi

General Guidelines

1. These instructions are provided as a guide to the clinician to perform a drainage procedure and are intended to supplement any hospital developed protocols, clinical drainage protocols or appropriate drainage device instructions for vacuum sources other than the PleurX™ bottle. Ensure that you are familiar with these instructions prior to beginning the procedure.
2. When using the drainage line, it may be used to drain at a suction level of -60 cm H₂O or less for as long as it takes to drain 1000 ml of fluid from the chest or 2000 ml of fluid from the abdomen per day. The volume of fluid removed should be based on the individual patient's status and the risks for over draining, including re-expansion pulmonary edema in the chest or hypotension for ascites patients.

How to Drain Using the Lockable Drainage Line

The drainage procedure should be performed using aseptic technique. The patient should be drained as directed by their doctor, usually every one to two days. Do not change frequency or drain more fluid than their doctor has recommended without first consulting their doctor. Make sure you are familiar with the suction method prior to draining.

Prepare to Drain

Warning: Do not use scissors or other sharp objects near the catheter.

Warning: The patient should not be left unattended during drainage.

1. Set up a clean, clear workspace on a table or counter.
2. Thoroughly wash your hands with soap and water for at least 1 minute.
3. Remove the self-adhesive dressing from over the catheter. Be sure not to tug on the catheter: support the skin while removing the self-adhesive dressing, gently grasp one edge and slowly peel the self-adhesive dressing from the skin (**Figure 2a**). **Note:** Avoid skin trauma by peeling the self-adhesive dressing back, rather than pulling it from the skin. (**Figure 2b**). If you see any swelling or fluid around the catheter, finish this drainage procedure and contact your doctor or nurse.

Caution: If the patient develops a fever (body temperature above 100.5°F (38°C), or you see any redness, swelling or fluid around the catheter, finish this drainage procedure and contact the patient's doctor or nurse.

4. Thoroughly wash your hands again with soap and water for at least 1 minute.
5. Open the Lockable Drainage Line kit.

Connect the Drainage Line to Wall/Portable Suction

Warning: Keep the valve on the catheter and the access tip on the drainage line clean. Keep them away from other objects to help avoid contamination.

Warning: Do not put anything except the access tip of the Lockable Drainage Line into the catheter valve since any other device could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

Warning: If wall suction or portable suction is used, it must be regulated to -60 cm H₂O or less.

Warning: Some portable suction units are not capable of regulating to -60 cm H₂O or less. Read the portable suction unit instructions for use prior to draining.

1. Close the roller clamp completely by rolling the wheel on the roller clamp toward the suction source. (**Figure 3**)

Caution: The roller clamp must be fully closed to occlude the drainage line. When not connected to a suction source, make sure the roller clamp is fully closed; otherwise the drainage line may allow air into the body or let fluid leak out.

2. Attach the 5-in-1 adapter to the Luer fitting on the drainage line. (**Figure 4**)
3. Connect the 5-in-1 adapter to the suction source.
4. Remove the protective cover by twisting it and pulling gently. Discard the cover. (**Figure 5**)
5. Remove the cap by twisting it counterclockwise and pulling gently. Discard the cap. (**Figure 6**)
6. Clean around the valve opening with a new alcohol pad. (**Figure 7**)
7. Insert the access tip into the catheter valve and advance it completely into the valve. You will feel and hear a click when the access tip and valve are securely connected. (**Figure 8**)
8. If desired, lock the access tip to the catheter valve by twisting the access tip until you feel and hear a second click. (**Figure 9**)

Caution: Make sure that the valve and the access tip are fully connected when draining. If they are accidentally separated, they may become contaminated. If this occurs, clean the valve with an alcohol pad and use a new drainage line to avoid potential contamination.

Warning: Precautions should be taken to ensure the drainage line is not tugged or pulled.

Connect the Drainage Line to Glass Vacuum Bottle(s)

Warning: Keep the valve on the catheter and the access tip on the drainage line clean. Keep them away from other objects to help avoid contamination.

Warning: Do not put anything except the access tip of the Lockable Drainage Line into the catheter valve since any other device could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

Warning: When draining with glass vacuum bottles, do not use a needle larger than 17 G.

1. Close the roller clamp completely by rolling the wheel on the roller clamp toward the glass vacuum bottle. (**Figure 3**)

Caution: Before connecting to a glass vacuum bottle, close the roller clamp completely by rolling the wheel on the roller clamp towards the glass vacuum bottle; otherwise it is possible for some or all of the vacuum in the bottle to be lost.

Caution: The roller clamp must be fully closed to occlude the drainage line. When not connected to a glass vacuum bottle, make sure the roller clamp is fully closed; otherwise the drainage line may allow air into the body or let fluid leak out.

2. Attach a 17 G needle to the Luer fitting on the drainage line. **(Figure 4)**
3. Connect the 17 G needle to the glass vacuum bottle.
4. Remove the protective cover by twisting it and pulling gently. Discard the cover. **(Figure 5)**
5. Remove the cap by twisting it counterclockwise and pulling gently. Discard the cap. **(Figure 6)**
6. Clean around the valve opening with a new alcohol pad. **(Figure 7)**
7. Insert the access tip into the catheter valve and advance it completely into the valve. You will feel and hear a click when the access tip and valve are securely connected. **(Figure 8)**
8. If desired, lock the access tip to the catheter valve by twisting the access tip until you feel and hear a second click. **(Figure 9)**

Caution: Make sure that the valve and the access tip are fully connected when draining. If they are accidentally separated, they may become contaminated. If this occurs, clean the valve with an alcohol pad and use a new drainage line to avoid potential contamination.

Warning: Precautions should be taken to ensure the drainage line is not tugged or pulled.

Drain Fluid

Warning: It is normal to feel some discomfort or pain when draining fluid. If discomfort or pain is experienced when draining, roll the wheel on the roller clamp towards the suction source to slow or stop the flow of fluid for a few minutes. If severe pain or discomfort persists, contact the physician. Pain may be an indication of infection.

Warning: Do not drain more than 1000 ml of fluid from the chest or 2000 ml of fluid from the abdomen at any one time.

1. Roll the wheel on the roller clamp away from the suction source to begin drainage. **(Figure 10)**
When fluid begins to flow into the drainage line you may partially close the roller clamp to slow the flow of fluid by rolling the wheel on the roller clamp toward the suction source.
2. If you need to change to a new bottle / canister / water seal device for any reason, close the roller clamp completely by rolling the wheel on the roller clamp toward the suction source. Remove the drainage line from the suction source and connect to a new bottle / canister / water seal device. Roll the wheel on the roller clamp away from the suction source to resume draining.
3. When flow stops or the desired amount of fluid has been removed, completely close the roller clamp by rolling the wheel on the roller clamp toward the suction source. **(Figure 3)**

Finish Drainage

4. If locked, twist the access tip to unlock it from the catheter valve. Set the used drainage line down. **(Figure 11)**
5. Pull the access tip out of the valve in a firm, smooth motion. **(Figure 12)**
6. Clean the catheter valve with a new alcohol pad. Discard the alcohol pad.
7. Place the new cap over the catheter valve and twist it clockwise until it clicks into its locked position. Cap might feel loose until it is locked into place. **(Figure 13)**
8. If required, record the drainage volume.
9. Disconnect the suction source and dispose of the used drainage line and bottle / canister / water seal device in accordance with applicable local, state, and federal regulations.
10. Clean around the catheter site and apply the appropriate dressing.

Caution: If fluid spills, use soap and water to clean skin and use appropriate cleaning agent for all other surfaces.

Note: Not made with natural rubber latex.

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Becton Dickinson İth. İhr. Ltd.