REF **50-7262**

PleurX™ Supplemental Insertion Kit

en Supplemental Insertion Kit

fr Kit d’insertion supplémentaire

de Zusatz-Einführhilfe

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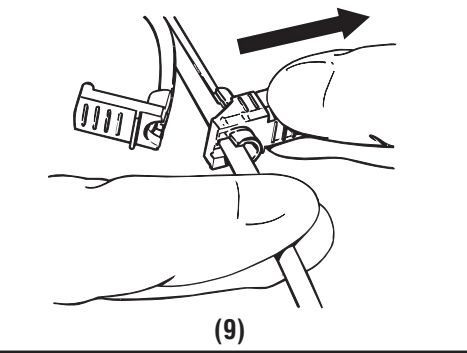
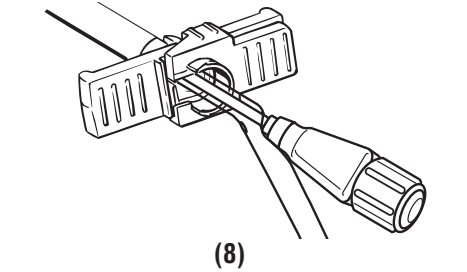
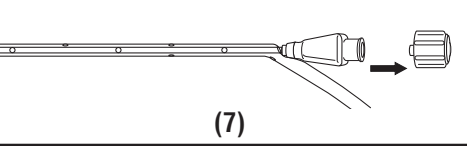
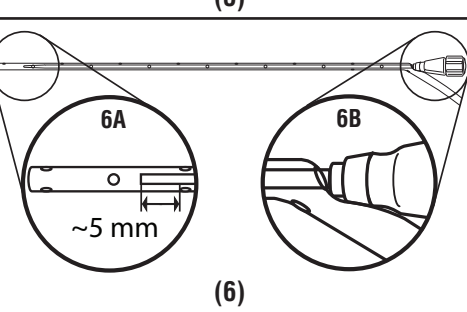
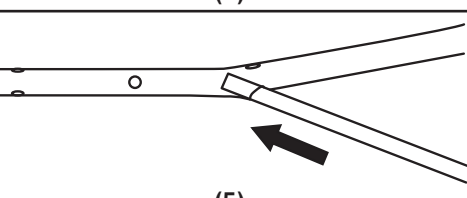
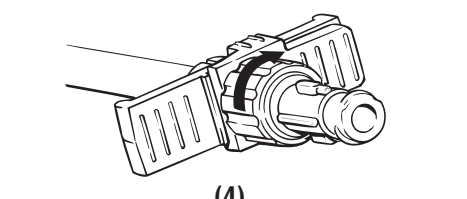
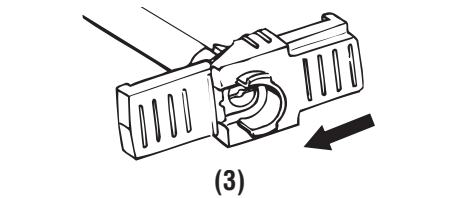
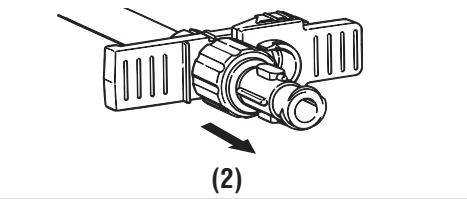
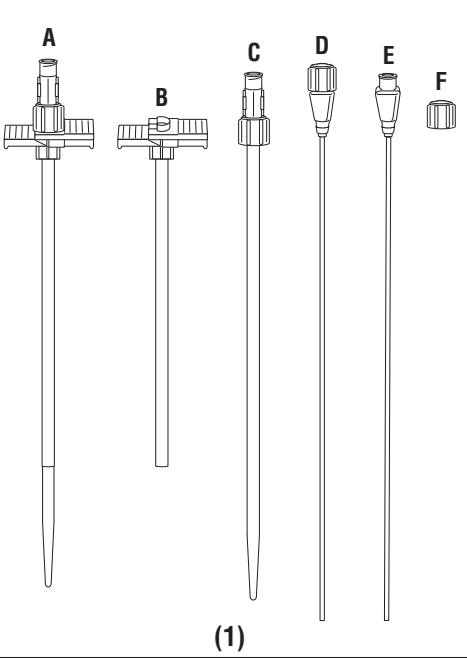
Contents

1 Valved Peel-Away Introducer (Figure 1A)

1 Sheath (Figure 1B)

1 Dilator (Figure 1C)

1 Catheter Insertion Stylet (Figure 1D)

1 Stylet, silicone coated (Figure 1E)

1 Yellow Cap (Figure 1F)

Indications for Use

The PleurX™ Supplemental Insertion Kit is intended to aid in the percutaneous insertion of a PleurX™ Catheter into the pleural space and PeritX™ Catheter into the peritoneal space.

Contraindication

The valved peel-away introducer and catheter insertion stylet are not designed for use in the arterial system or as a hemostatic device.

Cautions

The valved peel-away introducer is designed to reduce fluid loss and the risk of air intake, but is not a hemostasis valve. It is not intended to create a complete two-way seal, nor for arterial use. The valve will substantially reduce the rate of fluid flow, but some fluid loss through the valve may occur. Do not use if package is damaged.

For single use only. Re-use may result in non-functional product or contribute to cross-contamination.

Cleaning, disinfection, and sterilization may compromise essential material and design characteristics leading to device failure.

Potential complications of access and drainage of the pleural cavity include, but may not be limited to, the following: re-expansion pulmonary edema, pneumothorax, laceration of lung or liver, hypotension/circulatory collapse, wound infection, empyema and infection in the pleural cavity.

Potential complications of access and drainage of the peritoneal cavity include, but may not be limited to, the following: laceration of liver or bowel, hypotension/circulatory collapse, electrolyte imbalance, protein depletion, ascites leakage, peritonitis, wound infection, intraperitoneal adhesion, tumor growth in the catheter tunnel, and loculation of the peritoneal cavity.

Do not use catheter insertion stylet if the catheter has been trimmed to accommodate patient anatomy.

Do not use forceps on the introducer to break the handle and/or peel the sheath.

Insertion Procedure

Refer to the PleurX™ Catheter or PeritX™ Catheter insertion instructions provided with the catheter. After the guidewire has been placed, follow the steps below to use the valved peel-away introducer and the catheter insertion stylet. The valved peel-away introducer will be used in place of the non-valved peel-away introducer found in the PleurX™ Catheter or PeritX™ Catheter kit. The catheter insertion stylet will aid in insertion of the catheter found in the PleurX™ Catheter or PeritX™ Catheter kit.

- Remove the dilator from the sheath. (Figure 2)
- Slide the valve over the sheath opening, insert the dilator through the valve, and lock in place using the rotating collar. (Figures 3 and 4)
- Insert the tip of the catheter insertion stylet through the first fenestration of the catheter (closest to the cuff) and advance it into the catheter toward the tip (away from the cuff). (Figure 5)

- Ensure that the base of the stylet hub is pushed all the way up against the catheter. (Figure 6B), such that the tip of the catheter insertion stylet is just past the 3rd to last fenestration near the tip of the catheter. (Figure 6A)
- After percutaneous access has been achieved, leave the guidewire in place and advance the valved peel-away introducer over the guidewire and into the space. (Figure 6A)
- Remove the dilator and guidewire from the sheath by unlocking the rotating collar and gently withdrawing the dilator and guidewire as a unit from the sheath. **Note:** If placing the catheter over the guidewire, only remove the dilator. Turn the yellow cap counter clockwise to remove it from the stylet. (Figure 7) Thread the catheter with the stylet over the guidewire and ensure the guidewire is coming out of the stylet hub prior to placing the catheter into the introducer.
- Advance the catheter with stylet through the valve all the way until the hub of the stylet is at the top of the sheath. (Figure 8)

- Pull the stylet out of the catheter. Continue pushing the catheter into the sheath advancing it until all the fenestrations are within the pleural or peritoneal space. This can be verified using fluoroscopy as the fenestrations are located along the barium sulfate stripe. **Note:** If placing the catheter over the guidewire, remove the stylet and the guidewire as a unit out of the catheter.
- After the catheter has been positioned, crack the sheath handle in half.
- Peel the non-valved side of the sheath handle partially away from the catheter. (Figure 9)
- Near the valve, hold the catheter firmly in position and pull the valve off of the catheter. **Note:** It is normal to experience some resistance when pulling the catheter through the slit on the valve.
- Remove the sheath from the patient.
- Refer again to the instructions provided with the PleurX™ Catheter or PeritX™ Catheter after the sheath is peeled-away.

Note: Not made with natural rubber latex.

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