**REF 50-7265A**

# Lockable Drainage Line Kit

**For use with the PleurX™ or PeritX™ Catheter Systems**

- en** Lockable Drainage Line Kit  
For use with the PleurX™ or PeritX™ Catheter Systems
- fr** Kit de ligne de drainage verrouillable  
À utiliser avec les systèmes de cathéter PleurX™ ou PeritX™
- de** Arretierbares Drainageschlauch-Set  
Zur Verwendung mit den Kathetersystemen PleurX™ oder PeritX™
- it** Kit per Tubo di Drenaggio Bloccabile  
Per l'uso con cateteri PleurX™ o PeritX™
- es** Kit de Línea de Drenaje Bloqueable  
Para su uso con los sistemas de catéteres PleurX™ o PeritX™
- pt** Kit de Tubos de Drenagem Bloqueáveis  
Para utilização com os sistemas de cateter PleurX™ ou PeritX™
- nl** Kit voor vergrendelbare drainageleiding  
Voor gebruik met PleurX™ of PeritX™ kathetersystemen
- sv** Kit för låsbar dränageslang  
För användning med katetersystemen PleurX™ och PeritX™
- fi** Lukittava dreניתkusarja  
Käytettäväksi PleurX™ tai PeritX™ katetrijärjestelmien kanssa
- no** Låsbart drenasjeslangesett  
For bruk sammen med PleurX™ eller PeritX™ katetersystemene
- da** Låsbar drænslangekit  
Til brug sammen med PleurX™ eller PeritX™ katetersystemer
- el** Κιτ γραμμής παροχέτευσης με δυνατότητα ασφάλισης  
Για χρήση με τα συστήματα Καθετήρα PleurX™ ή Καθετήρα PeritX™
- tr** Kilitlenebilir Drenaj Hattı Kiti  
PleurX™ veya PeritX™ Kateter Sistemleri ile kullanılmak üzere tasarlanmıştır

## CareFusion

 75 North Fairway Drive  
Vernon Hills, IL 60061 USA

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**Made in Dominican Republic by  
CareFusion DR 203 Ltd.**

**Becton Dickinson Pty Ltd.**  
Level 5, 66 Waterloo Road  
Macquarie Park, NSW 2113, Australia

**Becton Dickinson Limited**  
14B George Bourke Drive  
Mt. Wellington, Auckland, 1060 New Zealand  
362-24428 • 2020-11

© 2020 CareFusion Corporation or one of its affiliates.  
All rights reserved.

For product inquiries: [bd.com/drainage](http://bd.com/drainage)  
PleurX, PeritX, CareFusion and the CareFusion logo are  
trademarks or registered trademarks of CareFusion  
Corporation, or one of its affiliates.

**STERILE EO**

Not made  
with natural  
rubber latex.

**USA Rx Only**

For Single  
Use Only

Do not use if  
package is damaged.

**DEHP**  
Contains  
phthalates

en

**Table of Contents**

Product Information .....1  
 How to Drain Using the  
 Lockable Drainage Line Kit .....2  
 Place a New Self-Adhesive Dressing .....4

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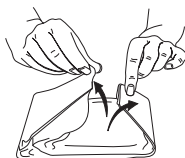
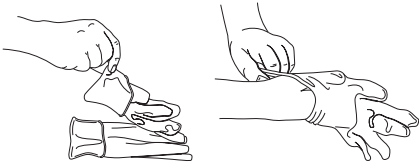
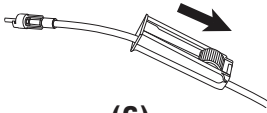
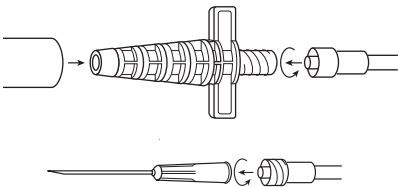
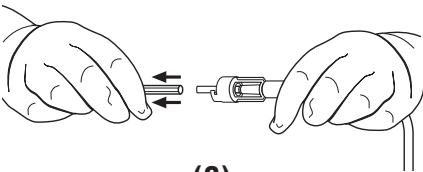
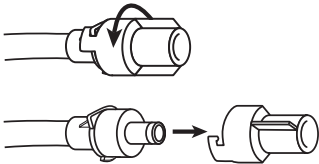
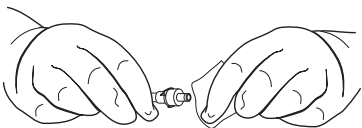
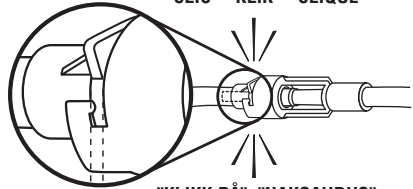
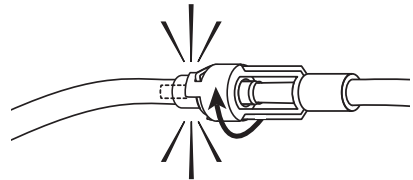
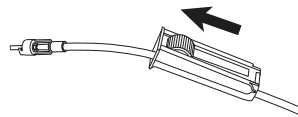
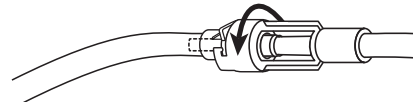
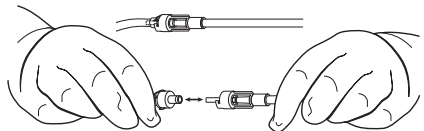
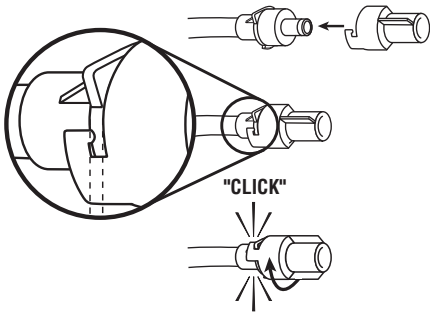
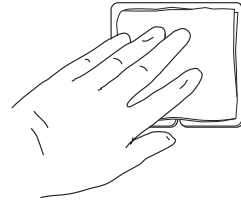
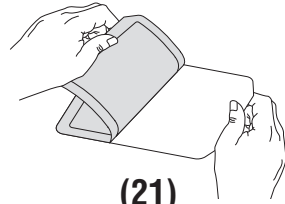
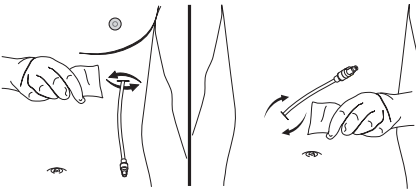
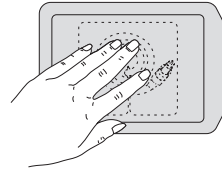
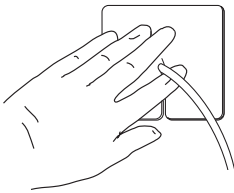
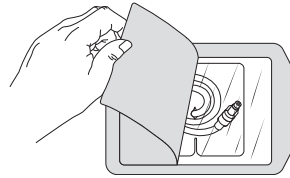
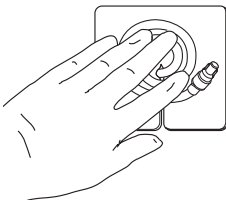
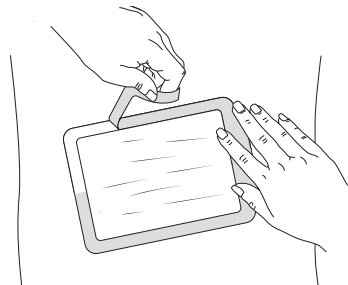
(19)

(24)

## Lockable Drainage Line Kit Procedure

### Product Information

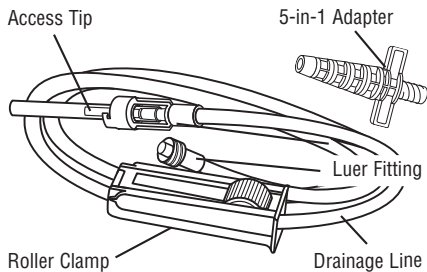
The Lockable Drainage Line Kit is designed to be used exclusively with the PleurX™ Pleural or PeritX™ Peritoneal Catheter for drainage of the pleural or peritoneal cavity, respectively.

Read this entire pamphlet carefully before draining fluid.

### Indications for Use

The Lockable Drainage Line Kit is indicated for use only with the PleurX™ Catheter and PeritX™ Catheter for intermittent drainage. The Lockable Drainage Line Kit is used to drain fluid using standard wall suction, water seal drainage system, glass vacuum bottle, or other appropriate method (portable suction).

### Lockable Drainage Line


(Figure 1)

### Warnings

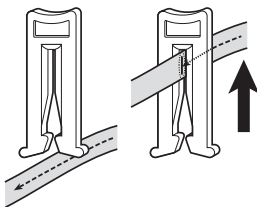If you accidentally cut the catheter or damage the valve, follow these steps:

- Pinch the catheter closed between your fingers.
- Slip the blue **emergency** slide clamp over the catheter and push the catheter completely into the small end of the clamp. This will close the catheter. (Figure 2)
- Notify the doctor immediately.

(Figure 2)

## Cautions

### Rx Only

For single use only. Re-use may result in a nonfunctional product or contribute to cross contamination.

Contains Phthalates. The benefit of treatment outweighs the remote possibility of exposure to phthalates.

**Chest:** Potential complications of draining the pleural space include, but may not be limited to, pneumothorax, re-expansion pulmonary edema, hypotension, circulatory collapse and infection.

**Abdomen:** Potential complications of draining fluid from the abdomen include, but may not be limited to, hypotension, circulatory collapse, electrolyte imbalance, protein depletion, ascites leakage, peritonitis, wound infection, intraperitoneal adhesion, and loculations of the peritoneal space.

### Contact the patient's physician if:

- The patient develops fever (body temperature above 100.5°F (38°C)), or you notice any redness, swelling, oozing or pain at the exit site. These may be signs of infection that may require treatment.
- Shortness of breath is not relieved after draining 1000 ml from the chest or 2000 ml from the abdomen at one time.
- The patient continues to experience symptoms, but little or no fluid drains from the catheter.
- Less than 50 ml drainage in three (3) drainage procedures in a row.
- The appearance (color, thickness, etc.) of the fluid changes significantly between drainages.

### Sterility

This product has been sterilized and is for single use only. Do not re-sterilize. CareFusion will not be responsible for any product that is re-sterilized, nor accept for credit or exchange any product that has been opened but not used.

Do not use if package is damaged.

### Drainage Line Kit Supplies

Blue Wrap

(Wrap around the following Items):

- 1 - Drainage Line with Lockable Access Tip
- 1 - 5-in-1 Drainage Line Adapter
- 1 - Valve Cap
- 1 - Blue **Emergency** Clamp
- 4 - Alcohol Pads
- 1 - Pair Gloves
- 1 - Foam Catheter Pad
- 4 - Gauze Pads, 4" x 4" (10.2 cm x 10.2 cm)
- 1 - Self-Adhesive Dressing

**Note:** -60 cm H<sub>2</sub>O = -1.7 in Hg = -44 mm Hg = -0.8 psi

## General Guidelines

1. These instructions are provided as a guide to the clinician to perform a drainage procedure and are intended to supplement any hospital developed protocols, clinical drainage protocols or appropriate drainage device instructions for vacuum sources other than the PleurX™ bottle. Ensure that you are familiar with these instructions prior to beginning the procedure.
2. When using the Drainage Line, it may be used to drain at a suction level of -60 cm H<sub>2</sub>O or less for as long as it takes to drain 1000 ml of fluid from the chest or 2000 ml of fluid from the abdomen per day. The volume of fluid removed should be based on the individual patient's status and the risks for over draining, including re-expansion pulmonary edema in the chest or hypotension for ascites patients.

## How to Drain Using the Lockable Drainage Line Kit

The drainage and self-adhesive dressing procedures should be performed using aseptic technique. The patient should be drained as directed by their doctor, usually every one to two days. Do not change frequency or drain more fluid than their doctor has recommended without first consulting their doctor.

Make sure you are familiar with the suction method prior to draining.

### Prepare to Drain

**Warning:** Do not use scissors or other sharp objects near the catheter.

**Warning:** The patient should not be left unattended during drainage.

**Note:** The blue **emergency** slide clamp should be used if you accidentally cut the catheter or damage the valve. **See Warnings. (Figure 2)**

1. Set up a clean, clear workspace on a table or counter.
2. Thoroughly wash your hands with soap and water for at least 1 minute.
3. Remove the self-adhesive dressing from over the catheter. Be sure not to tug on the catheter: support the skin while removing the dressing, gently grasp one edge and slowly peel the dressing from the skin **(Figure 3a)**. **Note:** Avoid skin trauma by peeling the dressing back, rather than pulling it from the skin. **(Figure 3b)**.

**Caution:** If the patient develops a fever (body temperature above 100.5°F (38°C)), or you see any redness, swelling or fluid around the catheter, finish this drainage procedure and contact the patient's doctor or nurse.

4. Thoroughly wash your hands again with soap and water for at least (1) one minute.
5. Open the Lockable Drainage Line Kit.

6. Set packet with blue wrap on your workspace with the flap side up. Carefully unfold the blue wrap by pulling on the outside of the wrap. **(Figure 4)** Leave the enclosed items on the wrap. The items and the inside of the wrap are sterile. Do not touch them with your ungloved hands or other non-sterile items.
7. Pick up gloves by the folded cuff and put on as shown. Both gloves fit either hand. Be careful not to let the outside of the gloves touch anything non-sterile, such as your skin or clothing. Put on gloves as shown. **(Figure 5)**
8. Tear open three alcohol pads, but do not remove the pads from their pouches. Place them on the blue wrap.

## Connect the Lockable Drainage Line to Wall/Portable Suction

**Warning:** Keep the valve on the catheter and the access tip on the drainage line clean. Keep them away from other objects to help avoid contamination.

**Warning:** Do not put anything except the access tip of the drainage line into the catheter valve since this could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

**Warning:** If wall suction or portable suction is used it must be regulated to -60 cm H<sub>2</sub>O or less.

**Warning:** Some portable suction units are not capable of regulating to -60 cm H<sub>2</sub>O or less. Read the portable suction unit instructions for use prior to draining.

1. Close the roller clamp completely by rolling the wheel on the roller clamp toward the suction source. **(Figure 6)**

**Caution:** The roller clamp must be fully closed to occlude the drainage line. When not connected to a suction source, make sure the roller clamp is fully closed; otherwise the drainage line may allow air into the body or let fluid leak out.

2. Attach the 5-in-1 adapter to the Luer fitting on the drainage line. **(Figure 7)**
3. Connect the 5-in-1 adapter to the vacuum/suction source.
4. Remove the protective cover by twisting it and pulling gently. Take care to avoid contaminating the access tip. **(Figure 8)**
5. Hold the base of the catheter valve and remove the cap by twisting it counterclockwise and pulling gently. Discard the cap. Take care to avoid contaminating the valve. **(Figure 9)**
6. Clean around the valve opening with a new alcohol pad. **(Figure 10)**

7. Insert the access tip into the catheter valve and advance it completely into the valve. You will feel and hear a click when the access tip and valve are securely connected. **(Figure 11)**
8. If desired, lock the access tip to the catheter valve by twisting the access tip until you feel and hear a second click. **(Figure 12)**

**Caution:** Make sure that the valve and the access tip are fully connected when draining. If they are accidentally separated, they may become contaminated. If this occurs, clean the valve with an alcohol pad and use a new drainage line to avoid potential contamination.

**Warning:** Precautions should be taken to ensure the drainage line is not tugged or pulled.

### **Connect the Lockable Drainage Line to Glass Vacuum Bottle(s)**

**Warning:** Keep the valve on the catheter and the access tip on the drainage line clean. Keep them away from other objects to help avoid contamination.

**Warning:** Do not put anything except the access tip of the drainage line into the catheter valve since this could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

**Warning:** When draining with glass vacuum bottles, do not use a needle larger than 17 G.

1. Close the roller clamp completely by rolling the wheel on the roller clamp toward the glass vacuum bottle. **(Figure 6)**

**Caution:** Before connecting to a glass vacuum bottle, close the roller clamp completely by rolling the wheel on the roller clamp towards the glass vacuum bottle. Otherwise, it is possible for some or all of the vacuum in the bottle to be lost.

**Caution:** The roller clamp must be fully closed to occlude the drainage line. When not connected to a glass vacuum bottle, make sure the roller clamp is fully closed; otherwise the drainage line may allow air into the body or let fluid leak out.

2. Attach a 17 G needle to the Luer fitting on the drainage line. **(Figure 7)**
3. Connect the 17 G needle to the glass vacuum bottle.
4. Remove the protective cover by twisting it and pulling gently. **(Figure 8)** Discard the cover. Set the access tip back on the blue wrap.

5. Hold the base of the catheter valve and remove the cap by twisting it counterclockwise and pulling gently. Discard the cap. **(Figure 9)**
6. Clean around the valve opening with a new alcohol pad. **(Figure 10)**
7. Insert the access tip into the catheter valve and advance it completely into the valve. You will feel and hear a click when the access tip and valve are securely connected. **(Figure 11)**
8. If desired, lock the access tip to the catheter valve by twisting the access tip until you feel and hear a second click. **(Figure 12)**

**Caution:** Make sure that the valve and the access tip are fully connected when draining. If they are accidentally separated, they may become contaminated. If this occurs, clean the valve with an alcohol pad and use a new drainage line to avoid potential contamination.

**Warning:** Precautions should be taken to ensure the drainage line is not tugged or pulled.

### **Drain Fluid**

**Warning:** It is normal to feel some discomfort or pain when draining fluid. If discomfort or pain is experienced when draining, roll the wheel on the roller clamp toward the suction source to slow or stop the flow of fluid for a few minutes. If severe pain or discomfort persists, contact the physician. Pain may be an indication of infection.

**Warning:** Do not drain more than 1000 ml of fluid from the chest or 2000 ml of fluid from the abdomen at any one time.

1. Roll the wheel on the roller clamp away from the suction source to begin drainage. **(Figure 13)**  
When fluid begins to flow into the drainage line, you may partially close the roller clamp to slow the flow of fluid by rolling the wheel on the roller clamp toward the suction source.
2. If you need to change to a new glass vacuum bottle/canister/water seal device for any reason, close the roller clamp completely by rolling the wheel on the roller clamp toward the suction source. Remove the drainage line from the vacuum/suction source and connect to a new glass vacuum bottle/canister/water seal device. Roll the wheel on the roller clamp away from the suction source to resume draining.
3. When flow stops or the desired amount of fluid has been removed, close the roller clamp completely by rolling the wheel on the roller clamp toward the suction source. **(Figure 6)**

### Finish Drainage

1. If locked, twist the access tip to unlock it from the catheter valve. **(Figure 14)**
2. Pull the access tip out of the valve in a firm, smooth motion. Set the used drainage line down. **(Figure 15)**
3. Clean the catheter valve with a new alcohol pad. Discard the alcohol pad.
4. Place the new cap over the catheter valve and twist it clockwise until it clicks into its locked position. Cap might feel loose until it is locked into place. **(Figure 16)**
5. If required, record the drainage volume.
6. Disconnect the suction source and dispose of the used drainage line and glass vacuum bottle/canister/water seal device in accordance with applicable local, state, and federal regulations.

**Caution:** If fluid spills, use soap and water to clean skin and use appropriate cleaning agent for all other surfaces.

### Place a New Self-Adhesive Dressing

1. Clean around the catheter exit site with a new alcohol pad. **(Figure 17)**
- Note:** Make sure the exit site and skin around the catheter are dry before completing the self-adhesive dressing procedure.
2. Place the foam catheter pad around the catheter. **(Figure 18)**
  3. Wind the catheter into loops and place it over the foam pad. **(Figure 19)**
  4. Cover the catheter with up to four (4) gauze pads. **(Figure 20)**
  5. Remove gloves from both hands.
  6. The self-adhesive dressing has three (3) layers:
    - a. printed liner
    - b. clear wound dressing
    - c. center panel and frame backing
  7. Peel the printed liner from the self-adhesive dressing, exposing the adhesive surface. **(Figure 21)**
  8. Center the self-adhesive dressing over the gauze pads and press it down. **(Figure 22)** **Note:** Do not stretch the self-adhesive dressing during application.
  9. Remove and discard the center panel from the backing of the self-adhesive dressing. **(Figure 23)**
  10. Slowly remove the frame while smoothing down the dressing edges. **(Figure 24)**
  11. Smooth the entire dressing from the center toward the edges using firm pressure to enhance adhesion.

**Note:** Not made with natural rubber latex.

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Becton Dickinson İth. İhr. Ltd.