**REF 50-7280A**

# Catheter Access Kit

**For use with the PleurX™ or PeritX™ Catheter Systems**

- en** Catheter Access Kit  
For use with the PleurX™ or PeritX™ Catheter Systems
- fr** Kit d'accès au cathéter  
À utiliser avec les systèmes de cathéter PleurX™ ou PeritX™
- de** Katheterzugangskit  
Zur Verwendung mit den Kathetersystemen PleurX™ oder PeritX™
- it** Kit di accesso del catetere  
Per l'uso con cateteri PleurX™ o PeritX™
- es** Kit de Acceso para Catéter  
Para su uso con los sistemas de catéteres PleurX™ o PeritX™
- pt** Kit de acesso do cateter  
Para utilização com os sistemas de cateter PleurX™ ou PeritX™
- nl** Kathetertoegangskit  
Voor gebruik met PleurX™ of PeritX™ kathetersystemen
- sv** Kit för kateteråtkomst  
För användning med katetersystemen PleurX™ och PeritX™
- fi** Katetrin liitinsarja  
Käytettäväksi PleurX™ tai PeritX™ katetrijärjestelmien kanssa
- no** Katetertilgangssett  
For bruk sammen med PleurX™ eller PeritX™ katetersystemene
- da** Kateteradgangssæt  
Til brug sammen med PleurX™ eller PeritX™ katetersystemer
- el** Κιτ πρόσβασης καθετήρα  
Για χρήση με τα συστήματα Καθετήρα PleurX™ ή Καθετήρα PeritX™
- tr** Kateter Giriş Kiti  
PleurX™ veya PeritX™ Kateter Sistemleri ile kullanılmak üzere tasarlanmıştır

## CareFusion

75 North Fairway Drive  
Vernon Hills, IL 60061 USA

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0123

© 2020 CareFusion Corporation or one of its affiliates.  
All rights reserved.

For product inquiries: [bd.com/drainage](http://bd.com/drainage)  
PleurX, PeritX, CareFusion and the CareFusion logo are  
trademarks or registered trademarks of CareFusion  
Corporation, or one of its affiliates.

**Made in Dominican Republic by  
CareFusion DR 203 Ltd.**

**Becton Dickinson Pty Ltd.**  
Level 5, 66 Waterloo Road  
Macquarie Park, NSW 2113, Australia

**Becton Dickinson Limited**  
14B George Bourke Drive  
Mt. Wellington, Auckland, 1060 New Zealand  
362-24429 • 2020-11

**USA Rx Only**

Do Not  
Resterilize

Not made with  
natural rubber latex.

For Single  
Use Only

Do not use if  
package is damaged.









**DEHP**  
Contains  
phthalates

# CareFusion

Figure, Abb, Figura, Afbeelding, Figur, Kuva, Εικόνα, Şekil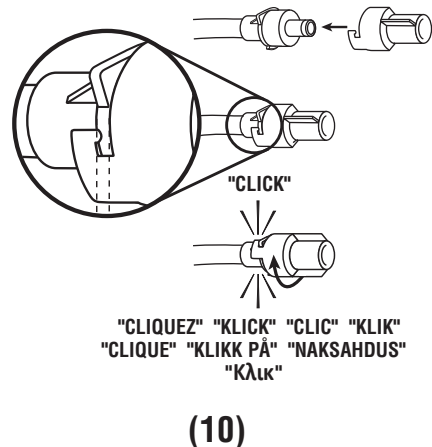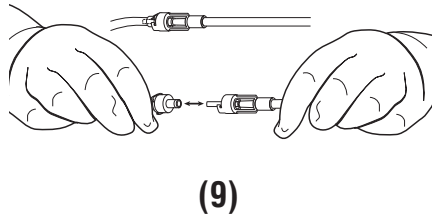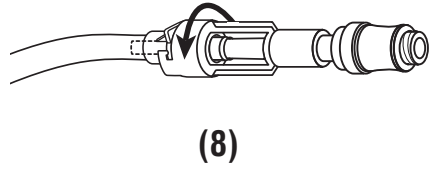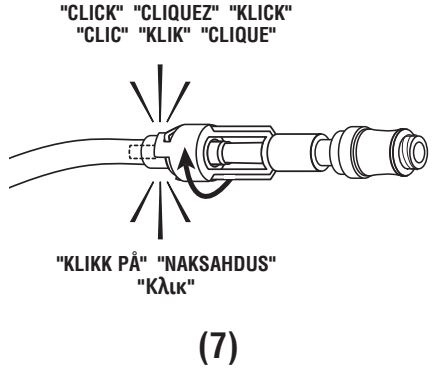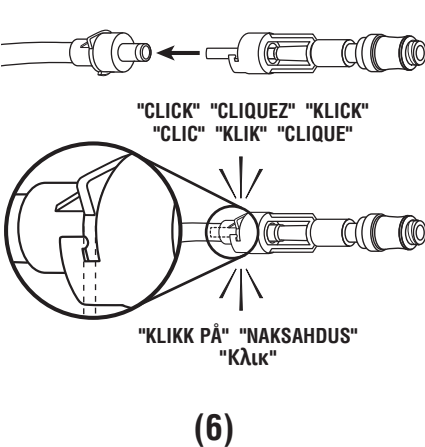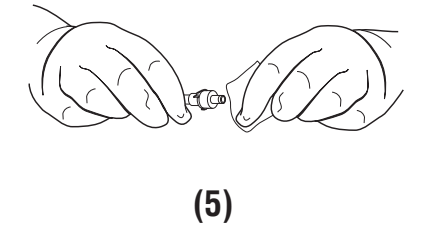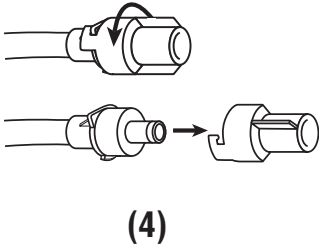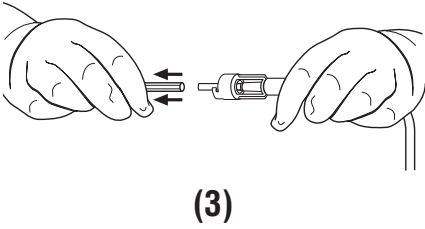

### Instructions for Use

#### Kit Contents

Blue wrap wrapped around the following components:

- 1 Lockable Access Tip with Needleless Access Valve
- 2 Alcohol Pads
- 1 Valve Cap
- 1 **Emergency** Slide Clamp
- 1 10 ml Syringe

#### Product Description

The Catheter Access Kit is designed to be used exclusively with the PleurX™ Catheter or PeritX™ Catheter. It is intended to provide intermittent access to the PleurX™ Catheter or PeritX™ Catheter to aspirate a sample, administer sclerosing agents (pleural use only), or perform catheter maintenance (saline flushing, or dec clotting).

#### For Use with PeritX™ Peritoneal Catheter

##### Indications for Use


The Catheter Access Kit is intended to provide access to the PeritX™ Catheter for aspiration and catheter maintenance.

#### For Use with PleurX™ Pleural Catheter

##### Indications for Use

The Catheter Access Kit is intended to provide access to the PleurX™ Catheter to inject and withdraw fluids.

#### Catheter Access Kit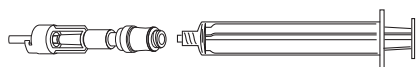

Lockable Access Tip With Needleless Access Valve

Syringe

(Figure 1)

#### Warnings

Keep the valve on the catheter and the access tip clean. Keep them away from other objects to help avoid contamination.


Do not use scissors or other sharp objects near the catheter.

Do not put anything except the access tip into the catheter valve since this could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

If the catheter is resistant to flushing and aspiration, it may be partially or completely occluded. Do not flush against resistance.

#### If you accidentally cut the catheter or damage the valve, follow these steps:

- a. Pinch the catheter closed between your fingers.
- b. Slip the blue **emergency** slide clamp over the catheter and push the catheter completely into the small end of the clamp. This will close the catheter. **(Figure 2)**
- c. Notify the doctor immediately.

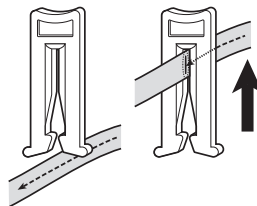(Figure 2)

#### Cautions

##### Rx Only

For single use only. Re-use may result in a nonfunctional product or contribute to cross contamination.

Make sure that the valve and the access tip are fully locked together when accessing the catheter. If they are accidentally separated, they may become contaminated. If this occurs, clean the valve with a new alcohol pad and use a new access tip to avoid potential contamination.

The alcohol pads are flammable. Do not expose the pads to an open flame.

Contains Phthalates. The benefit of treatment outweighs the remote possibility of exposure to phthalates.

**Chest:** Potential complications of draining the pleural space include, but may not be limited to, pneumothorax, re-expansion pulmonary edema, hypotension, circulatory collapse and infection.

**Abdomen:** Potential complications of draining fluid from the abdomen include, but may not be limited to, hypotension, circulatory collapse, electrolyte imbalance, protein depletion, ascites leakage, peritonitis, wound infection, intraperitoneal adhesion, and loculations of the peritoneal space.

Disconnect the locking access tip with needleless access valve from the catheter valve after use.

## Sterility

This product has been sterilized and is for single use only. Do not re-sterilize. CareFusion will not be responsible for any product that is re-sterilized, nor accept for credit or exchange any product that has been opened but not used.

Do not use if package is damaged.

## General Guidelines

These instructions are provided as a guide to the clinician to perform catheter maintenance (aspiration, saline flushing, or dec clotting) and pleurodesis procedures and are intended to supplement hospital developed protocols. The clinician should have a thorough understanding of risks and patient specific concerns related to catheter maintenance. Follow aseptic technique.

## Note

Priming Volume is 0.3 ml\*

Volume to first fenestration:

- Pleural: 2.6 ml\*
- Peritoneal: 3 ml\*

Volume to end of catheter:

- Pleural: 4.2 ml\*
- Peritoneal: 4.4 ml\*

\*Volumes include lockable access tip with needleless access valve.

## Catheter Access Suggested Procedure

**Warning:** Do not use scissors or other sharp objects near the catheter.

**Warning:** Keep the valve on the catheter and the access tip clean. Keep them away from other objects to help avoid contamination.

**Note:** The blue **emergency** slide clamp should be used if you accidentally cut the catheter or damage the valve. See **Warnings. (Figure 2)**

**Note:** The catheter access procedure should be performed using aseptic technique.

**Note:** Dispose of all used product in accordance with applicable local, state, and federal regulations. Used product may present a potential biohazard.

1. Remove the dressing from over the catheter. Be sure not to tug on the catheter. If the patient develops a fever (body temperature above 100.5° F [38° C]), or you see any redness, swelling or fluid around the catheter, finish this drainage procedure and contact the physician.
2. Open the Catheter Access Kit and unfold the blue wrap.
3. Hold the lockable access tip with needleless access valve and remove the protective cover by twisting it and pulling gently. Take care to avoid contaminating the access tip. **(Figure 3)**

4. Hold the base of the catheter valve and remove the cap by twisting it counterclockwise and pulling gently. Discard the cap. Take care to avoid contaminating the valve. **(Figure 4)**
5. While holding the base of the catheter valve in one hand, clean around the valve opening with a new alcohol pad. **(Figure 5)**

**Warning:** Do not put anything except the access tip into the catheter valve since this could damage the valve. A damaged valve may allow air into the body or let fluid leak out through the valve when not draining.

6. Continue holding the catheter near the valve. Carefully align the access tip and valve as shown in **Figure 6** and insert the access tip into the catheter valve and advance it completely into the valve. You will feel and hear a click when the access tip and valve are securely connected.
7. If desired, lock the access tip to the catheter valve by twisting the access tip until you feel and hear a second click. **(Figure 7)**
8. To aspirate a sample, attach the syringe to the needleless access valve.

**Warning:** If the catheter is resistant to flushing and aspiration, it may be partially or completely occluded. Do not flush against resistance.

9. Pull back on the syringe plunger to draw fluid.
  10. If the syringe will not aspirate, the catheter may be flushed with saline per institution protocol.  
**Note:** Do not use a syringe smaller than 10 ml to flush the catheter.
  11. If the catheter cannot be flushed or aspirated due to occlusion, a dec clotting procedure may be followed per institution protocol.
  12. To perform a pleurodesis procedure using a sclerosing agent, follow the sclerosing agent's instructions for use and your institution protocol.
  13. When catheter maintenance is complete, remove the access tip from the catheter. If locked, twist the access tip to unlock it from the catheter valve. **(Figure 8)**
  14. With the access tip in one hand and the catheter valve in the other hand, pull the access tip out of the valve in a firm, smooth motion. **(Figure 9)**
  15. Clean the catheter valve with a new alcohol pad.
  16. Place the new cap over the catheter valve and twist it clockwise until it clicks into its locked position. Cap might feel loose until it is locked into place. **(Figure 10)**
  17. Redress the catheter per hospital protocol.
- Note:** Not made with natural rubber latex.

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