**REF 50-7290A**

# Procedure Pack

**For use with the PleurX™ or PeritX™ Catheter Systems**

- en** Procedure Pack  
For use with the PleurX™ or PeritX™ Catheter Systems
- fr** Trousse de procédure  
À utiliser avec les systèmes de cathéter PleurX™ ou PeritX™
- de** Verfahrensset  
Zur Verwendung mit den Kathetersystemen PleurX™ oder PeritX™
- it** Pacco procedurale  
Per l'uso con i sistemi di cateteri PleurX™ o PeritX™
- es** Paquete para procedimientos  
Para su uso con los sistemas de catéteres PleurX™ o PeritX™
- pt** Pacote de procedimento  
Para utilização com os sistemas de cateter PleurX™ ou PeritX™
- nl** Procedurepak  
Voor gebruik met PleurX™ of PeritX™ kathetersystemen
- sv** Procedurpaket  
För användning med katetersystemen PleurX™ och PeritX™
- fi** Toimenpidepakkaus  
Käytettäväksi PleurX™ tai PeritX™ katetrijärjestelmien kanssa
- no** Prosedyrepakke  
For bruk sammen med PleurX™ eller PeritX™ katetersystemene
- da** Procedurepakke  
Til brug sammen med PleurX™ eller PeritX™ katetersystemer
- el** Πακέτο εξαρτημάτων διαδικασίας  
Για χρήση με τα συστήματα Καθετήρα PleurX™ ή Καθετήρα PeritX™
- tr** Prosedür Paketi  
PleurX™ veya PeritX™ Kateter Sistemleri ile kullanılmak üzere tasarlanmıştır

## CareFusion

75 North Fairway Drive  
Vernon Hills, IL 60061 USA

**CareFusion France 309 S.A.S.**

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0123

## Becton Dickinson Pty Ltd.

Level 5, 66 Waterloo Road  
Macquarie Park, NSW 2113, Australia

## Becton Dickinson Limited

14B George Bourke Drive  
Mt. Wellington, Auckland, 1060 New Zealand  
362-24430 • 2020-11

© 2020 CareFusion Corporation or one of its affiliates.  
All rights reserved.

For product inquiries: [bd.com/drainage](http://bd.com/drainage)  
PleurX, PeritX, CareFusion and the CareFusion logo are  
trademarks or registered trademarks of CareFusion  
Corporation, or one of its affiliates.

**STERILE EO**

Not made  
with natural  
rubber latex.

**USA Rx Only**

For Single  
Use Only

Do not use if  
package is damaged.

**DEHP**  
Contains  
phthalates

# CareFusion

en

### Table of Contents

Product Information .....	1
How to use the Procedure Pack .....	1

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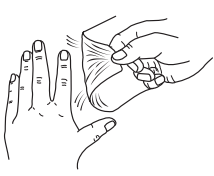
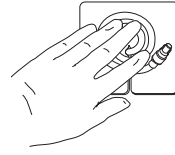
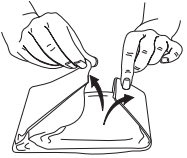
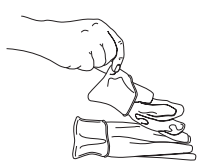
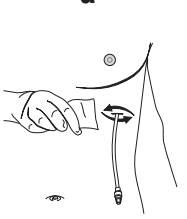
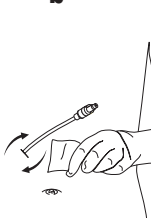
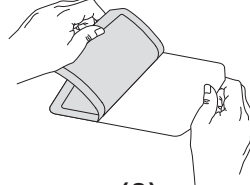
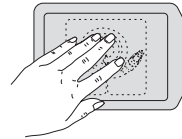
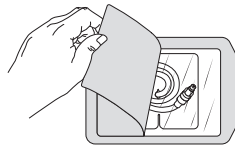
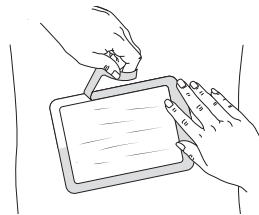
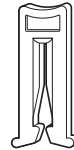
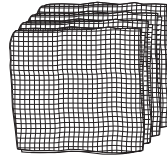
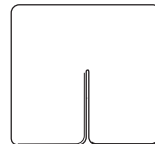
## Procedure Pack

### Product Information

The Procedure Pack contains the necessary supplies for dressing of the PleurX™ Catheter or PeritX™ Catheter after performing a drainage procedure.

You should have been trained by a doctor or nurse in the correct way to use the items in this Procedure Pack. If you have any problems or questions about dressing the catheter and exit site, contact your doctor or nurse. Read this entire pamphlet carefully before dressing the catheter and exit site.

### Indications for Use

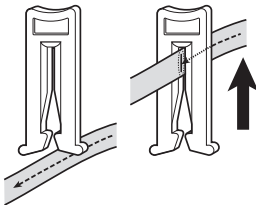
The Procedure Pack is indicated for dressing of the PleurX™ Pleural Catheter and PeritX™ Peritoneal Catheter and exit site.

### Warnings

Keep the valve on the catheter and the access tip on the drainage line clean. Keep them away from other objects to help avoid contamination.

If you accidentally cut the catheter or damage the valve, follow these steps:

- Pinch the catheter closed between your fingers.
- Slip the blue **emergency** slide clamp over the catheter and push the catheter completely into the small end of the clamp. This will close the catheter. (**Figure 1**)
- Notify your doctor immediately.

(Figure 1)

### Cautions

For single use only. Re-use may result in a non-functional product or contribute to cross contamination.

If fluid spills, use soap and water to clean skin and use appropriate cleaning agent for all other surfaces.

Contains Phthalates. The benefit of treatment outweighs the remote possibility of exposure to phthalates.

### Sterility

The Procedure Pack has been sterilized. This product is for single use only. Do not re-sterilize.

Do not use if package is damaged.

CareFusion will not be responsible for any product that is re-sterilized, nor accept for credit or exchange any product that has been opened but not used.

### Procedure Pack Supplies

Self-Adhesive Dressing (**A**)

Blue Wrap (**B**)

(Wrapped around the following Items):

3 Alcohol Pads (**C**)

1 Pair of Gloves (**D**)

1 Valve Cap (**E**)

1 Blue **Emergency** Slide Clamp (**F**)

4 Gauze Pads 4" x 4" (10 cm x 10 cm) (**G**)

1 Foam Catheter Pad (**H**)

### How to use the Procedure Pack

If you are performing a drainage procedure refer to the PleurX™ Drainage Procedure Instructions for Use.

If you are not performing a drainage procedure at this time and only dressing the catheter exit site:

**Note:** The valve cap will not be used.

**Warning:** Do not use scissors or other sharp objects near the catheter.

- Set up a clean clear workspace on a table or counter.
- Thoroughly wash your hands with soap and water for at least 1 minute.
- Remove the self-adhesive dressing from over the catheter. Be sure not to tug on the catheter: support the skin while removing the self-adhesive dressing, gently grasp one edge and slowly peel the self-adhesive dressing from the skin. (**Figure 2a**) **Note:** Avoid skin trauma by peeling the self-adhesive dressing back, rather than pulling it from the skin. (**Figure 2b**) If you see any swelling or fluid around the catheter, finish this drainage procedure and contact your doctor or nurse.

**Caution:** If you develop a fever (body temperature above 100.5 °F [38°C]), see any redness, swelling, or fluid around the catheter, finish this dressing procedure and contact your doctor or nurse.

4. Thoroughly wash your hands again with soap and water for at least 1 minute.
  5. Open the Procedure Pack.
  6. Set the packet with blue wrap on your workspace with the flap side up. Carefully unfold the blue wrap by pulling on the outside of the wrap. **(Figure 3)** Leave the enclosed items on the wrap. The items and the inside of the wrap are sterile. Do not touch them with your ungloved hands or other non-sterile items.
  7. Pick up gloves by the folded cuff and put on as shown. Both gloves fit either hand. Be careful not to let the outside of the gloves touch anything non-sterile, such as your skin or clothing. Put on gloves as shown. **(Figure 4)**
  8. Clean around the catheter site with a new alcohol pad. Discard the alcohol pad. **(Figure 5a and Figure 5b)**
- Caution:** The alcohol pads are flammable. Do not expose the pads to an open flame.
- Note:** Make sure the exit site and skin around the catheter are dry before completing the self-adhesive dressing procedure.
9. Place the foam catheter pad around the catheter. **(Figure 6)**
  10. Wind the catheter into loops and place it over the foam pad. **(Figure 7)**
  11. Cover the catheter with up to four (4) gauze pads. **(Figure 8)**
  12. Remove gloves from both hands.
  13. The self-adhesive dressing has three (3) layers:
    - a. printed liner
    - b. clear wound dressing
    - c. center panel and frame backing
  14. Peel the printed liner from the self-adhesive dressing, exposing the adhesive surface. **(Figure 9)**
  15. Center the self-adhesive dressing over the gauze pads and press it down. **(Figure 10)** **Note:** Do not stretch the self-adhesive dressing during application.
  16. Remove and discard the center panel from the backing of the self-adhesive dressing. **(Figure 11)**
  17. Slowly remove the frame while smoothing down the self-adhesive dressing edges. **(Figure 12)**
  18. Smooth the entire self-adhesive dressing from the center toward the edges using firm pressure to enhance adhesion.

**Note:** Not made with natural rubber latex.

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Becton Dickinson İth. İhr. Ltd.