

BD Integra™ Syringe

with retracting BD PrecisionGlide™ Needle

Important usage guidelines



1. Attachment

Attach the BD Integra™ Needle onto the BD Integra™ Syringe with a push and a clockwise twist. If preattached, tighten the needle onto the syringe prior to use.



2. Aspiration

Pull the shield straight off the needle. Draw up medication in accordance with your institution's protocol.



3. Injection

Administer medication in accordance with established protocol, making sure that all medication has been dispensed.



4. Activation

Push plunger rod to activate the safety mechanism. Depress until you sense two “clicks.”

Activation can occur before or after needle is withdrawn from the patient.



5. Needle retraction

The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete.

Discard immediately into an approved sharps collector after single use.

Luer needle



Luer lock design

BD Integra™ Needle



Tru-Lok™ Technology

Remember

- If activation is performed **before** needle is withdrawn from patient, do not advance needle while depressing plunger rod.
- If activation is performed **after** needle is withdrawn from patient, minimal splatter may occur. When activating the device outside of the patient, hold the device downward at arm's length and activate while pointing away from self and others. Keep device within sight at all times to avoid accidental needlestick injury.
- BD Integra™ Needle and Syringe **do not** have a standard luer connection. Conventional luer lock needles cannot be used with the BD Integra™ Syringe.
- **Do not** use this product to draw blood or for venipuncture.
- When delivering highly viscous medications, use a 22 G or 21 G BD Integra™ Retracting Needle, and inject at a controlled rate.