
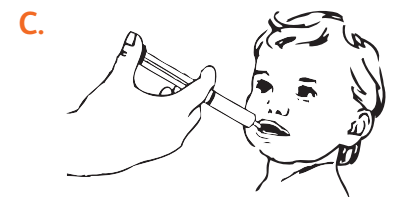
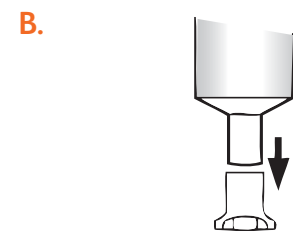


BD™ Oral Dispensing Syringe with Tip Cap

Usage guidelines

Please refer to package insert for detailed Instructions for Use.

- 1 Accurately measure prescribed dosage using the top edge of the stopper. **(A)**
Syringe scales are indicated in milliliters (*mL*).
- 2 After filling syringe, attach tip cap that is provided separately in small bag.
- 3 Before dispensing, **REMOVE TIP CAP (B)** and position syringe tip in the pocket between the cheek and the back teeth (*molars*). **(C)**
- 4 To avoid choking, slowly dispense the medication by pushing gently on the plunger rod. **(C)**
- 5 Discard syringe properly after use.

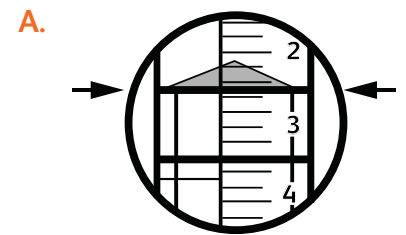

NOTE: Non-sterile • Single-use • For oral use only

This product is **not** made with natural rubber latex.

CAUTION: Reuse may lead to infection or other illness / injury.

Oral Dispensing Syringe will **not** accept a hypodermic needle.

Oral Syringe with tip cap*



Hypodermic Syringe



*Tip caps sold separately.

BD, 1 Becton Drive, Franklin Lakes, NJ 07417, U.S.

bd.com

BD and the BD Logo are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2020 BD. All rights reserved. BD-2708 (10/20)

