

Site-Scrub® IPA Device

Instructions for use

Hub cleaning

For catheters with clamps, ensure clamp is closed.

Remove the existing injection port/needle-free connector, administration tubing or add-on device.

Peel off foil cover of the Site-Scrub® IPA Cleaning Device. Do not touch the interior foam pad. Use device immediately after removal of lid.

Center the Site-Scrub® IPA Cleaning Device onto the hub.

Press down on the Site-Scrub® IPA Cleaning Device until it cannot be pushed any further.

Twist the Site-Scrub® IPA Cleaning Device back and forth for a minimum of eight repetitions of the back and forth scrub over a minimum of 10 seconds.

Remove the Site-Scrub® IPA Cleaning Device and discard as per facility protocol, after cleaning one hub.

Allow hub to dry for a minimum of five seconds. Place new sterile device onto hub.

For catheters with clamps, the clamp may be opened at this time.

1

2

3

4

5

6

7

8

9



Injection port/needle-free connector cleaning

1 Peel off foil cover of the Site-Scrub® IPA Cleaning Device. Do not touch the interior foam pad. Use device immediately after removal of lid.

2 Center the Site-Scrub® IPA Cleaning Device onto the injection port/needle-free connector.

3 Press down on the Site-Scrub® IPA Cleaning Device until it cannot be pushed any further.

4 Twist the Site-Scrub® IPA Cleaning Device back and forth for a minimum of eight repetitions of the back and forth scrub over a minimum of 10 seconds.

5 Remove the Site-Scrub® IPA Cleaning Device and discard as per facility protocol, after cleaning one injection port/needle-free connector.

6 Allow injection port/needle-free connector to dry for a minimum of five seconds.

Site-Scrub® IPA Device





Ordering information

Material number	Packaging	Units per box	Units per case
3882025	IV pole boxes	25 units	1,000 (40 boxes)
3882100	Singles	100 units	500 (5 boxes)

Indications For Use

- The Site-Scrub® IPA Device is intended for use on injection ports and female luer hubs as a disinfecting cleaner.
- The Site-Scrub® IPA Device contains 70% isopropyl alcohol.

Warnings

-  Flammable, keep away from fire or flame.
-  Do not use on devices which access the nervous system.
-  Do not leave the Site-Scrub® Device on an injection port or female luer hub after disinfecting.
-  Keep this and all drugs out of reach of children. In case of accidental ingestion, seek professional assistance or contact a Poison Control Center



Do not reuse



Do not resterilize



Non-pyrogenic



Do not use if package is damaged



Upper limit of temperature

STERILE R

This product and packaging do not contain natural rubber latex

Rx only

Federal (U.S.A) law restricts this device to sale by or on the order of a physician

Assembled in Mexico