



BD PosiFlush™ Saline Syringe

Quick reference card

Prior to flushing: Depress the plunger with the tip cap on to prepare the syringe for smooth fluid administration. Depress the plunger rod slightly forward until you feel a “pop” or see the solution move.

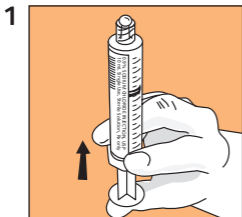


- Remove the syringe cap by twisting it off. If using a needleless system that requires a plastic cannula, attach it by twisting it firmly onto the syringe. Holding the syringe upright, pointing away from you, expel air from the syringe until you see a drop of fluid at the tip of the syringe.

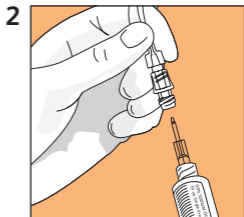


- Attach the syringe to the injection site or valve. If using a plastic cannula, push the syringe tip through the center of the injection site. If using a valve, push and twist the syringe clockwise onto the valve.
- If applicable, open the slide clamp on the IV extension tubing.

turn over >

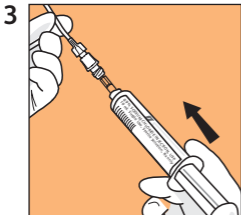

- Administer the saline flush solution following your facility's protocol.
- If applicable, close the slide clamp on the IV extension tubing after the saline flush is complete.

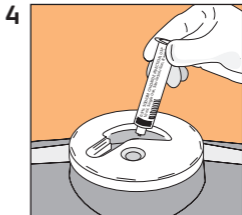

- Discard the syringe, including any unused solution, into a sharps collector.

Things to remember

- Be sure to break loose the plunger rod prior to use.
- Aseptic technique and proper preparation of the site are essential.

Do not reuse any syringe

Do not drop on sterile field

BD, Franklin Lakes, NJ, 07417, U.S.

bd.com

BD and the BD Logo are trademarks of Becton, Dickinson and Company. All other trademarks are the property of their respective owners. © 2019 BD and its subsidiaries. All rights reserved. BD-9104 (02/19)

