

Proper connection technique:

BD PureHub™ disinfecting cap to SmartSite™ needle-free connector

Align. Push. Twist.

Ensuring proper threading is as easy as **1, 2, 3.**



- 1 Align** the BD PureHub™ disinfecting cap threads to SmartSite™ needle-free connector threads.



- 2 Push** the BD PureHub™ disinfecting cap onto the SmartSite™ needle-free threads to ensure it is able to grip the threads.



- 3 Twist** the BD PureHub™ disinfecting cap while pushing in towards SmartSite™ needle-free until fully threaded.