



# BD PureHub™ Disinfecting Cap

## Usage guidelines



**1**  
Peel foil from the single-packaged BD PureHub™ Disinfecting Cap or remove the BD PureHub Disinfecting Cap from the IV pole strip.

**2**  
Apply the cap to the needle-free luer connector by **pushing and twisting**.

**3**  
The BD PureHub Disinfecting Cap must remain on the needle-free luer connector for a minimum of sixty (60) seconds and may remain on for up to seven (7) days.

## Ordering information

Material number	Description	Packaging	Units per box	Units per case
306596	BD PureHub Disinfecting Cap	Singles	300	3,000 (10 boxes)
Coming soon 306597	BD PureHub Disinfecting Cap	IV pole strips (10 count)	30 strips	4,500 (15 boxes)

**Provides a >4 log (99.99%) reduction in bacteria within 1 minute of application.\* Discard cap after removal. Single use only.**

\* Demonstrated reduction on *Staphylococcus aureus*, *Staphylococcus epidermidis*, *Pseudomonas aeruginosa*, *Escherichia coli*, *Candida glabrata* and *Candida albicans* as tested in a laboratory.

Threaded plastic cap is not made with natural rubber latex or plasticizer diethylhexyl phthalate. Active ingredient: 70% isopropyl alcohol.

Please read labeling at all times.

Always perform disinfection techniques as prescribed by hospital protocol.

BD, Franklin Lakes, NJ 07417, U.S.

[bd.com](http://bd.com)

© 2018 BD. BD, the BD Logo and BD PureHub are trademarks of Becton, Dickinson and Company. BD-3667 (06/18)

