


BD Nexiva™ Closed IV Catheter System—Single Port

Tips for success

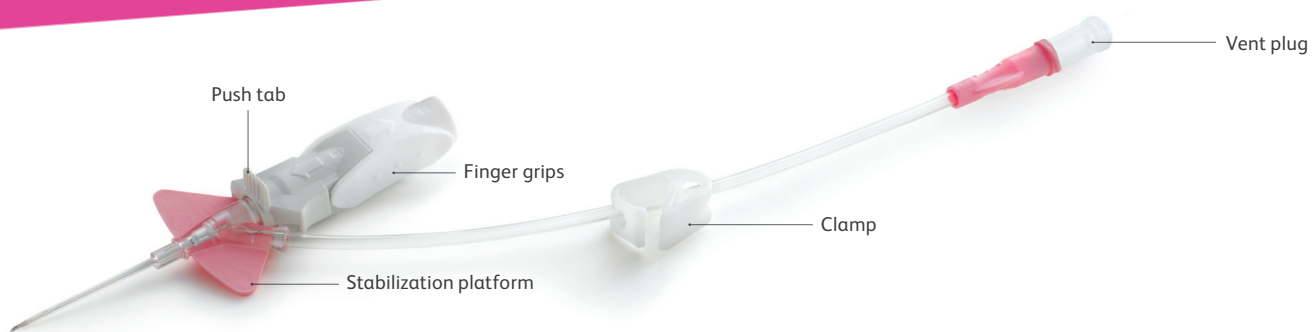

Sharp insertion

- Keep the stabilization platform and finger grips pushed together.
- **Do not** insert holding only onto the stabilization platform as this will push the needle back into the catheter.
- Avoid holding in a manner that pushes forward against the stabilization platform.

Easy vessel access

- Look for initial blood return along the catheter near the insertion site.
- Lower and slightly advance the entire system to ensure the catheter tip is within the vessel.
- Look for continuous blood flow along the extension tubing to confirm catheter is still in the vessel.

Smooth catheter advancement

- Push the catheter forward off the needle rather than pulling the needle back.
- Use the pad of your index finger, not your fingernail, behind the push tab.
- Push down and forward against the push tab.

Simple needle removal

- Pushing down gently on the stabilization platform, pull back on the finger grips until the needle shield releases.
- Do not touch the push tab during needle removal.

Blood containment

- Secure the vent plug before insertion.
- Clamp the extension tubing before replacing the vent plug with a needleless connector.

Consult product insert for complete instructions, warnings and cautions.
For more information visit bd.com/Nexiva

BD, Sandy, UT, 84070, U.S.

bd.com

BD, the BD Logo and Nexiva are trademarks of Becton, Dickinson and Company or its affiliates.
© 2019 BD. All rights reserved. BD-0979 (08/19) 1961

