


BD Nexiva™


Closed IV Catheter System—Dual Port

Points to practice—Priming

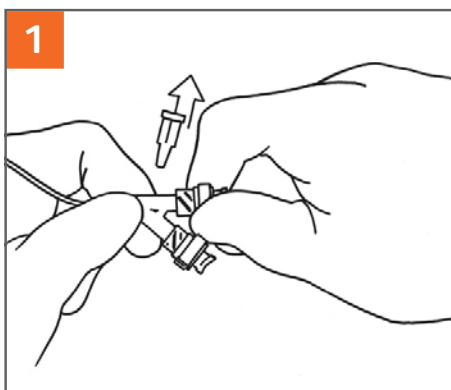

(fig. 1)

- Remove the vent plug and attach the additional BD Q-Syte™ luer access split septum (fig. 1).

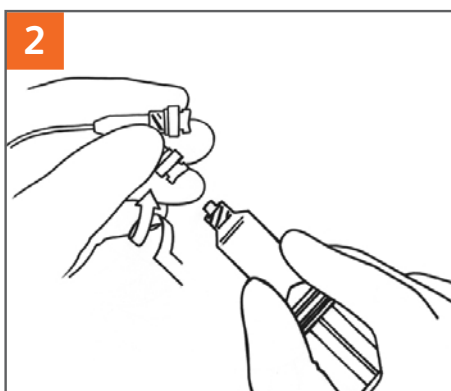

(fig. 2)

- Disinfect the BD Q-Syte device with an appropriate antiseptic.
- Attach flush syringe using a straight-on approach and flush through both BD Q-Syte devices until all air is removed (fig. 2).
- Leave syringe attached during catheter insertion to enhance blood flashback.
- Clamp extension set after insertion and before removal of the syringe.

Consult product insert for complete instructions, warnings and cautions.

BD, Franklin Lakes, NJ, 07417, USA

bd.com

© 2017 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company. BD-3277 (11/17).

