



# BD Vacutainer® Safety-Lok™ Blood Collection Set

## Instructions for Activation


## One-Handed Technique

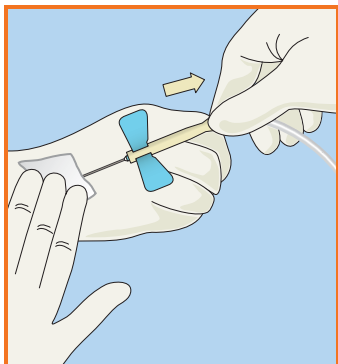


- 1.** After completing blood draw and the final tube has filled, remove the Safety-Lok Blood Collection Set by...

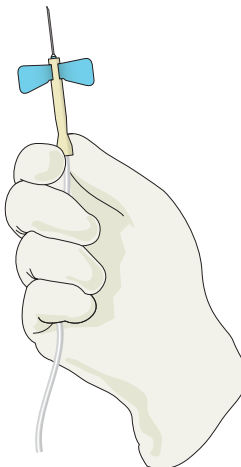

- 2.** ...grasping the translucent yellow safety shield grip area with the thumb and index finger while at the same time grasping the tubing securely with the other 3 fingers.

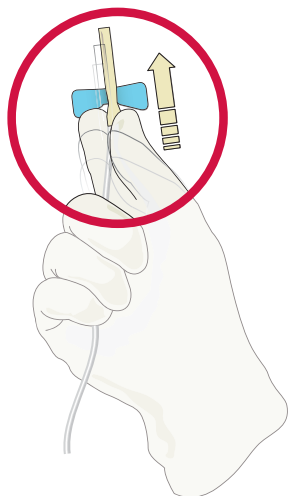

- 3.** Advance translucent yellow safety shield forward with thumb and index finger until the needle is completely covered and a click is heard, indicating that the safety shield is locked in place over the needle tip.

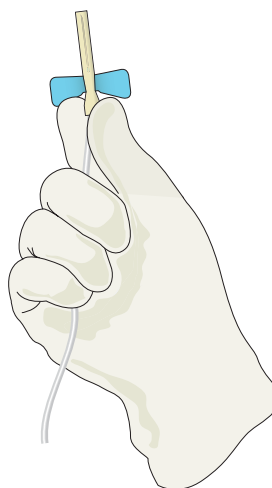

- 4.** Once the safety shield is completely advanced, dispose of the device in an approved sharps container.


For more information about this and other specimen collection products, please contact us at:

**BD Global Technical Services:** 1.800.631.0174

**vacutainer\_techservices@bd.com**

**BD Customer Service:** 1.888.237.2762

**www.bd.com/vacutainer**

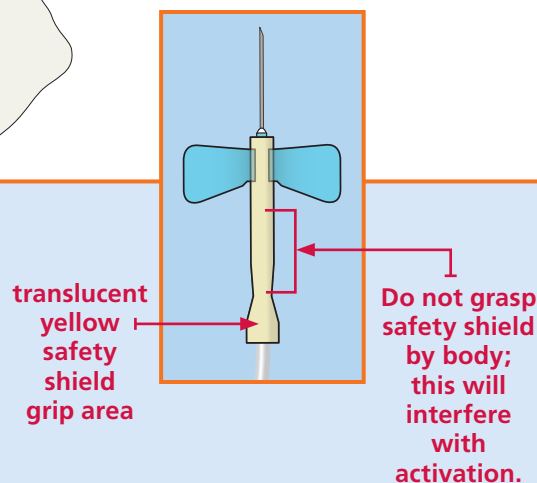

### CAUTION:

Handle all biologic samples and blood collection "sharps" (lancets, needles, luer adapters, and blood collection sets) in accordance with the policies and procedures of your facility. Obtain appropriate medical attention in the event of any exposure to biologic samples (e.g., through a puncture injury) since samples may transmit viral hepatitis, HIV (AIDS), or other infectious diseases. Utilize any safety-engineered feature if the blood collection device provides one. Discard all blood collection "sharps" into biohazard containers approved for their disposal.



**BD Diagnostics**

Preanalytical Systems

1 Becton Drive

Franklin Lakes, NJ 07417

[www.bd.com/vacutainer](http://www.bd.com/vacutainer)