


BD Vacutainer® Eclipse™ Blood Collection Needle


with BD Vacutainer® Pronto™ Quick Release Needle Holder

Usage of Product

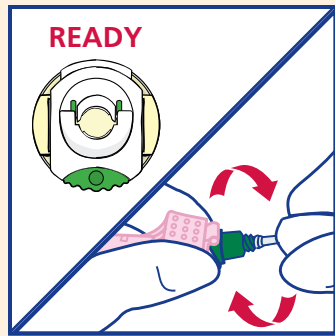


1a. Ensure BD Vacutainer® Pronto™ Holder is ready to accept new collection device. If not, press white reset button on holder.

1b. Holding both pink safety shield and colored needle cap, twist and remove white needle cap.

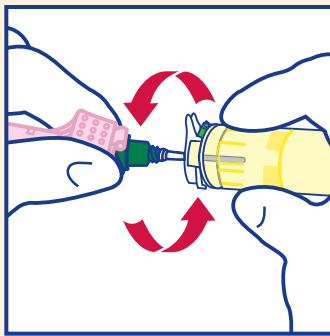

2. While holding the pink safety shield and the colored needle cap firmly, screw holder onto needle by pushing needle into holder while turning hands in opposite directions 1/4 turn until needle is fully seated onto holder.


DO NOT OVERTORQUE

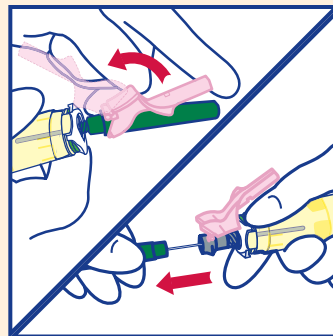


3a. Gently position pink safety shield straight back toward the holder.

3b. Twist and pull colored needle cap straight off.


Note: The needle bevel is always in position for venipuncture when the pink safety shield is facing up. **DO NOT** twist or rotate the pink safety shield.

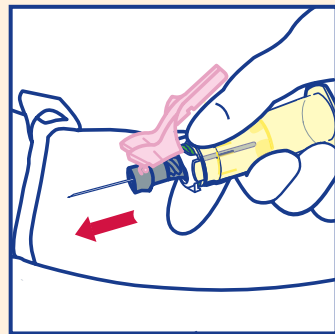

4. Perform venipuncture according to your facility's established procedures.

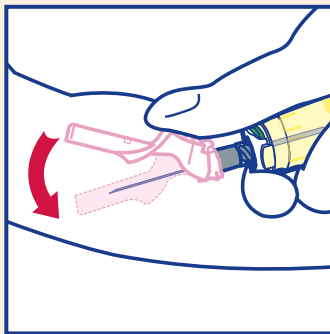

5. Immediately after removing needle from vein and with the needle pointed down and away from yourself and the patient, position thumb squarely on pink safety shield thumb pad and push pink safety shield forward to cover needle. An audible click may be heard. Lock shield into place and inspect. **DO NOT** attempt to engage safety shield by pressing against a hard surface.

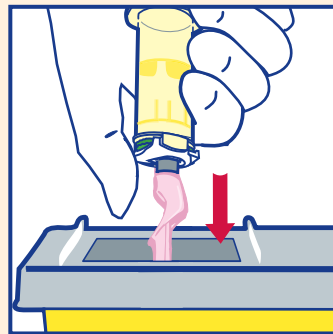


6. Dispose of collection device into nearest approved sharps container:

Hold device in a downward position over container opening, push green button on holder to release collection device into container. Ensure white reset button is not obstructed by other fingers.

To reuse BD Vacutainer® Pronto™ Holder, push white reset button.

BD Vacutainer® Eclipse™ Needle and Pronto™ Quick Release Needle Holder

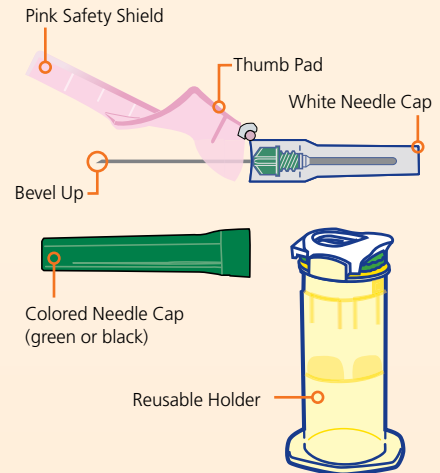

Ordering Information

Reference Numbers:

BD Vacutainer® Pronto™ Holder



368871

BD Vacutainer® Eclipse™ Blood Collection Needles

368607 21 G x 1 1/4"

368608 22 G x 1 1/4"

Product Storage

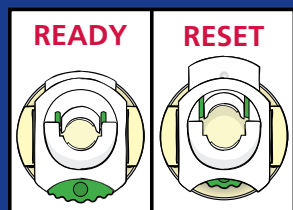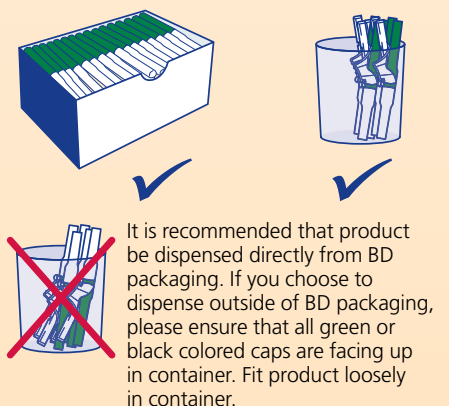

Note
BD Vacutainer® Pronto™ Holder can be used for 100 uses*

Procedure for Cleaning

- Soak in 1:10 dilution household bleach solution for 5 minutes**
- Agitate in the same solution using ultrasonic bath for 5 minutes
- Rinse with clean water
- Drain/Air Dry

Precaution

- Precaution recommends that reusable holders be cleaned after each use. BD in-house testing has verified that the BD Pronto™ Holder can be used for 10 cleaning cycles.† It is recommended that the hospital validates its own procedures for cleaning; the above procedure may provide the basis for such procedures.
- If the holder is visibly contaminated with blood and/or there is failure of product integrity, discard the holder.
- Handle all biologic samples and blood collection "sharps" (lancets, needles, luer adapters, and blood collection sets) in accordance with the policies and procedures of your facility. Obtain appropriate medical attention in the event of any

exposure to biologic samples (e.g., through a puncture injury) since samples may transmit viral hepatitis, HIV (AIDS), or other infectious diseases. Utilize any safety-engineered feature if the blood collection device provides one. Discard all blood collection "sharps" in biohazard containers approved for their disposal.

Reference

- BD has conducted an internal study and has determined that product performance up to 10 cleaning cycles is not impacted when the holder is cleaned after each use using the above indicated cleaning procedure. Autoclaving or the use of 100% bleach solution may impact the performance of this device.

BD Technical Services

1.800.268.5430

BD Customer Service

1.866.979.9408

www.bd.com/ca



2771 Bristol Circle
Oakville, ON L6H 6R5
www.bd.com/ca

* Data on file at BD. Cleaning between uses was not considered in the determination of 100 uses and is recommended.

** CLSI Guideline M29-A3.

† BD has conducted an internal study.

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. © 2009 BD
Franklin Lakes, NJ 07417 1/09 BDPAS0110-1-CE