



September 12, 2014

To whom it may concern:

In reference to the **Alaris® System Cleaning Product Guidelines**, CareFusion is issuing this letter to clarify which cleaning solutions and/or active chemical agents are appropriate for use on the Alaris® System. This document can be found on the Alaris System cleaning products website, [www.carefusion.com/alarissystemcleaning](http://www.carefusion.com/alarissystemcleaning).

- If a cleaning solution is not displayed under the "Recommended" list, the warranty with CareFusion **will be** honored so long as the solution does **not** contain any of the chemicals listed under the "DO NOT use" section.
- Although the PDI® Super Sani-Cloth germicidal disposable wipes contain a formulation of Quaternary Ammonium Chloride ("Quats"), which is a chemical listed under the "DO NOT use" section, this particular cleaning solution has been tested and **is** safe for use with the Alaris® System.
- While not on the "Recommended" list, Hydrogen Peroxide does **not** cause physical damage to the plastics associated with the Alaris® System. However, it can cause discoloration of the plastic (a cosmetic issue), which is **not** covered under the warranty.
- The current "Recommended" list represents brand names as presently marketed by respective manufacturers.
- CareFusion recommends that customers speak directly to the manufacturer about a product's antimicrobial efficacy, and then cross-reference their recommendation with the cleaning product guidelines.

We hope this letter helped to clarify the Alaris® System cleaning protocol. If you have any questions or concerns, please contact us at: [US-AlarisSystemCleaning@carefusion.com](mailto:US-AlarisSystemCleaning@carefusion.com).

Sincerely,

A handwritten signature in black ink, appearing to read "Igor Nesterenko", written over a light blue horizontal line.

Igor Nesterenko  
Director of Marketing