

# Module attachment and removal

## BD Alaris™ System



IUI connector



Module attachment



Module removal

## Module attachment

1. Inspect IUI connectors on each BD Alaris™ PC Unit and on each module before each use to reduce potential IUI connector failure. If surface contaminants, or blue or green deposits are visible, replace the IUI connector. **(A)**
2. Position the module at a 45° angle to the PC unit or pre-attached module.
3. On top of the module, align the IUI connectors.
4. Angle the module down until the latch snaps into place. **(B)**

## Module removal

1. Ensure the module is powered OFF.
2. Push in the module release latch, and angle the module away from the PC unit or attached module to disengage interconnecting components. **(C)**

For product support, contact Customer Advocacy at **888.812.3266** or [customerfeedback@bd.com](mailto:customerfeedback@bd.com)

For technical support, contact Instrument Technical Support at **866.488.1408**.

For product orders, contact Customer Order Management at **800.482.4822**.

⚠ See reverse side for applicable warnings and cautions.

📖 For complete instructions, refer to the BD Alaris™ System User Manual at [bd.com](http://bd.com)



# Module attachment and removal

## Warnings and cautions

**WARNING:** Inserting a finger or other object into the IUI connector, when the module is attached to the PC unit, could result in electrical shock.

**WARNING:** When properly secured/snapped, the release latch provides a very secure connection between modules. If not properly latched, a module can be dislodged during operation.

**WARNING:** Failure to perform these operations can result in improper instrument operations.

**WARNING:** Failure to follow these instructions may result in potential hazards associated with damaged IUI connectors.