

Alaris™ Syringe module: compatible syringes and flow rate ranges

Alaris™ products

Medications delivered on the Alaris Syringe module must be prepared in a syringe that accommodates the desired flow rate. The following chart shows the flow rate ranges for each syringe size. In addition, the compatible syringe sizes with reorder numbers are displayed. To decrease potential start-up delays, delivery inaccuracies and delayed generation of occlusion alarms, it is recommended to use the smallest syringe necessary to deliver the fluid or medication, especially when delivering high-risk or life-sustaining medications at low infusion rates (e.g., if infusing 2.3 mL of fluid, use a 3 mL syringe).

Syringe size	Brand and model numbers				Flow rate range
	BD	Monoject®	Terumo®	IVAC®	
1 mL	309628	N/A	N/A	N/A	0.01–30 mL/h
3 mL	309657	Soft pack only: 1180300777	SS-03L	N/A	0.01–100 mL/h
5 mL	309646	N/A	SS-05L (6 mL total)	N/A	0.1–150 mL/h
6 mL	N/A	Soft pack: 1180600777 Rigid pack: 8881516937	SS-05L (6 mL total)	N/A	0.1–150 mL/h
10 mL	300912	N/A	SS-10L (12 mL total)	N/A	0.1–250 mL/h
12 mL	N/A	Soft pack: 1181200777 Soft pack: 1181200777T Rigid pack: 8881512878	SS-10L (12 mL total)	N/A	0.1–250 mL/h
20 mL	302830	Soft pack: 1182000777 Rigid pack: 8881520657	SS-20L SS-20L2	N/A	0.1–500 mL/h
30 mL	302832	N/A	SS-30L (35 mL total)	N/A	0.1–650 mL/h
35 mL	N/A	Soft pack: 1183500777 Rigid pack: 8881535762	SS-30L (35 mL total)	N/A	0.1–650 mL/h
50 mL	N/A	N/A	N/A	30602N	0.1–999 mL/h
60 mL	309653	Soft pack: 1186000777 Soft pack: 1186000777T Rigid pack: 8881560125	SS-60L	N/A	0.1–999 mL/h

For product support, contact Customer Advocacy at **888.812.3266** or customerfeedback@bd.com.

For technical support, contact Instrument Technical Support at **866.488.1408**.

For product orders, contact Customer Order Management at **800.482.4822**.

BD, San Diego, CA, 92130, U.S.

bd.com



Trademarks are property of their respective owners.

© 2016 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company. IF7064 (1216/1000)

DME 10000274292-01 P/N P00000210