



# BD Microtainer™ Tubes with Microgard™ Closure Tube Guide and Order of Draw

Catalog #/Closure Color	Additive	Mix by Inverting	Laboratory Use
-------------------------	----------	------------------	----------------


 365974 Lavender	K <sub>2</sub> EDTA	10x	For whole blood hematology determinations. Tube inversions prevent clotting.
--	---------------------	-----	--

 365965 Green	Lithium Heparin	10x	For plasma determinations in chemistry. Tube inversions prevent clotting.
---	-----------------	-----	---

 365985 Mint Green	Lithium Heparin and Gel for plasma separation	10x	For plasma determinations in chemistry. Tube inversions prevent clotting.
 365987 Mint Green			

 365992 Grey	NaF/Na <sub>2</sub> EDTA	10x	For glucose determinations. Tube inversions ensure proper mixing of additive and blood.
--	--------------------------	-----	---

 365967 Gold	Clot Activator and Gel for serum separation	5x	For serum determinations in chemistry.
 365978 Gold			

 365963 Red	No additive	0x	For serum determinations in chemistry, serology and blood banking.
---	-------------	----	--



365976  
Tube Extender

