

# BD SafetyGlide™ Needle

## Quick reference guide



### 1. Attachment

1a. Attach the BD SafetyGlide™ needle to any standard luer lock or luer slip hypodermic syringe. Twist until firmly seated to connect to a luer lock syringe.

1b. Pull shield straight off needle to avoid damaging needle point.



### 2. Aspiration

Draw up medication in accordance with your institution's protocol.



### 3. Injection

Administer medication in accordance with your institution's protocol, making sure that all the medication has been dispensed. For user convenience, the needle's "bevel up" position is oriented to the lever arm.



### 4. Activation

Activate away from self and others immediately after removal from the patient by using a one-handed technique and pushing the lever arm completely forward until the needle tip is completely covered.

Visually confirm that the lever arm has fully advanced and the needle tip is covered. After activation or if unable to activate, immediately discard into an approved sharps container. This device is designed for a single use only.

### Remember

- Activation of the protective mechanism may cause minimal splatter of any fluid that is remaining on the needle after injection.
- To connect the BD SafetyGlide™ needle to a luer slip syringe, push firmly.
- Follow your institution's protocol for aseptic technique and skin preparation.
- Follow your institution's policy for safe disposal of all medical waste.