

Nutrition beyond boundaries:



Adapting treatments for independent lifestyles

Malnutrition is a major clinical condition, caused by, among other things, gastrointestinal dysfunction, surgery, or cancer. As such, patients with wide ranging medical conditions often rely on the use of total parenteral nutrition. However, depending on the flexibility and size of the infusion solution, patients are often confronted with not only a new treatment, but also an impact on their independence.


At BD, we see the patient beyond the process. Our TPN solution helps ensure that their freedom is not compromised. The BD BodyGuard™, as well as its associated accessories helps patients achieve their desired independence. We invite you to take a journey through this guide, to discover how your patients' travel dreams can still be a reality with the BD BodyGuard™ solution.



Flexible accessories, limited restrictions

BD provides a wide range of accessories that are applicable to both adult and paediatric TPN use.

The transportation of the nutrition bag is at the core of ensuring flexibility when undergoing TPN therapy. Our BD BodyGuard™ backpacks were carefully designed to allow patients to be mobile with 2L or 3L nutrition bags.

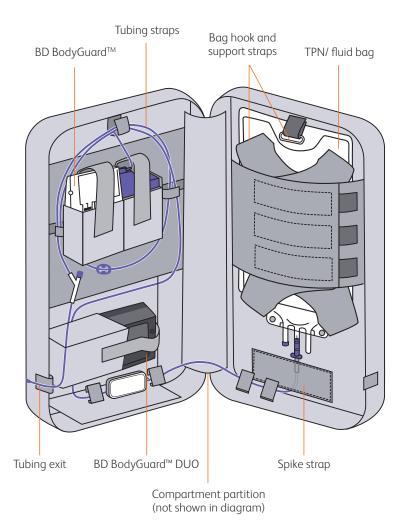



Carry case, infusion kit Ref. CASE00002



Trolley with wheels Ref. CNC00001



Bag, 3 litre Ref. BAG00008



BAG, 2 litre Ref. 160-002XG Ref. 160-002XB Ref. 160-002XP (grey, black and pink)



500ml carrying bag Ref. 150-001X

Keep going on the move

In addition to ensuring that your patients' transportation needs are covered, providing a substantial battery life is key to ensure that they can remain unplugged for as long as possible.

The BD BodyGuard™ pump's long battery life on a single charge, can give your patients the confidence and freedom they need to carry out their day to day activities.



Rechargeable battery Ref. 130-050XV (1800 mAh)



Rechargeable battery Ref. 130-051XV (3600 mAh)



Wall charger Ref. 999-WC 8.2V

Battery specifications

Rechargeable Li-Polymer 7.4 V Battery 1800 mAh.

Life at 25 and 125 ml/h 15h Life at 1200 ml/h 8h

3600 mAh. Life at 125 ml/h (optional) up to 30 h

Our comprehensive range of accessories can gear your patients for travel:




By car



By train



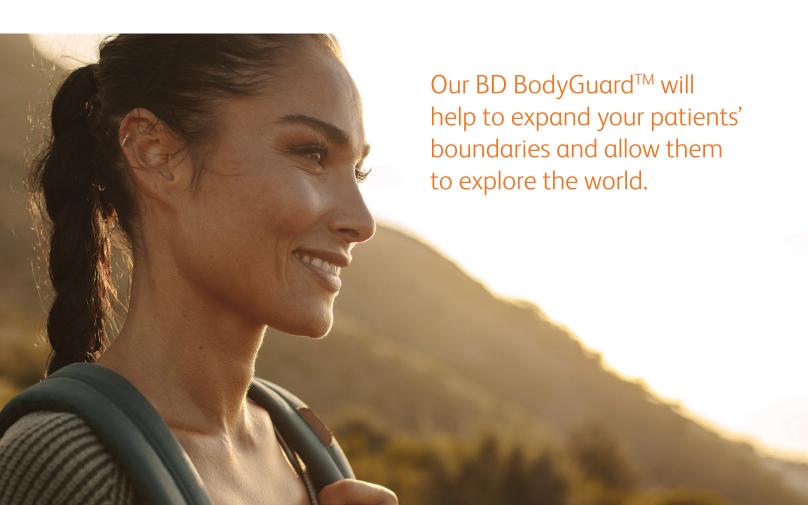


Helping you with things to consider when choosing your holiday destination

When you are planning a holiday, you will have a lot of things to consider. We have prepared a simple check list for you to help.

What you need to start your TPN infusion

The BD BodyGuard™ Infusion System

The DD Body Cadra Thruston System	
1. The BD BodyGuard™ infusion pump	
2. Power cord or battery charging cable	
3. Rechargeable battery pack 1800mAH (inside the pump)	-
4. Directions for Use (in print out format)	-
	-
Itching to travel? Aligning with your clinical team is	How to pack your nutrition supplies
key:	
Seek advice from your clinical/hospital support	Prepare packaging that will ensure appropriate safety during transport
Consult your doctor for an appropriate medical certificate	Ensure that your compounding bags are stored in the appropriate temperature range
Double check your health insurance	Plan a possible delivery of preparations to your destination address
Tips for choosing your destination	Filtra base 2 Province de de Cille Con-
	Flying by air? Remember to do the following:
Select a location in a temperature climate	
without high humidity	Contact customer service to inform them of
Make sure you have easy assess to a refrigerator	your flight plans and medical situation
Make sure you have easy access to a refrigerator	Complete an excess baggaga waiver form
with constant temperature	Complete an excess baggage waiver form
Select a location with high hygiene standards	Request a medical baggage waiver letter (if available)
Identify the nearest nutrition center	•
Travel with a companion	

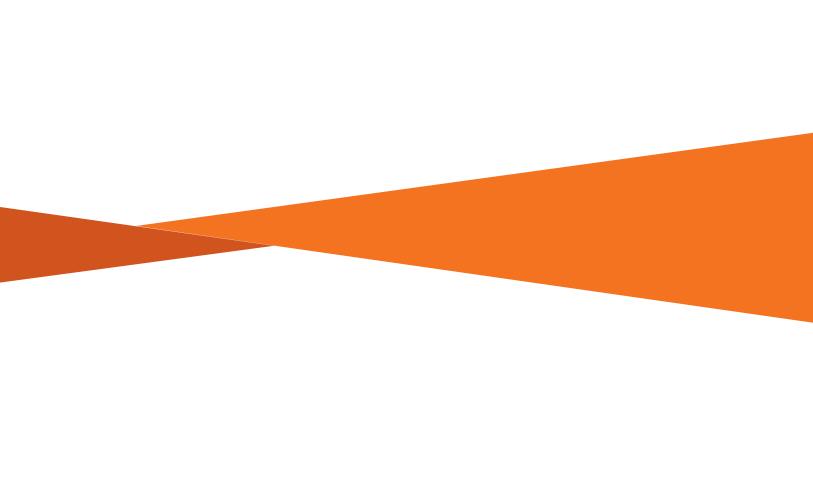

BD

Please contact your BD sales representative for further information

BD - Europe, Terre Bonne Park – A4, Route de Crassier 17, 1262 Eysins, Switzerland

BD, the BD Logo and BD BodyGuard are trademarks of Becton, Dickinson and Company or its affiliates. © 2020 BD. All rights reserved. 0000CF04926 Issue 1

on our comprehensive TPN solution.

bd.com