

Low pressure power injector use with the SmartSite[®] extension set 20041E


1. Prior to first use and before every subsequent access, swab the top of the needle-free valve with 70% isopropyl alcohol and allow to dry (approximately 30 seconds)
2. Flush per hospital protocol to check catheter placement and patency
3. Connect the low pressure power injector set securely to the needle-free valve by fully turning the luer lock clockwise

Precaution

- Connect the set from the power injector to the needle-free valve on the T-port closest to the catheter
 - Clamp the T-connector extension set tubing
4. Administer power injection
 5. Disconnect low pressure power injector set by turning luer lock counter-clockwise
 6. Flush the catheter and clamp per hospital protocol

Precaution:

- These products must be secured to other luer lock devices that are pressure rated for low pressure power injector use up to 325 psi
- Always refer to the *Directions for Use* insert to verify if the set is labeled for use with low pressure power injectors up to 325 psi and maximum flow rate of 10 mL/second
- Refer to the device *Directions for Use* for full directions and precautions for use

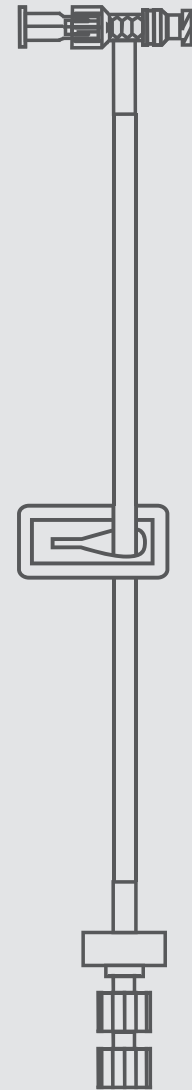

CareFusion
San Diego, CA

carefusion.com

