

# Bag Spike with SmartSite® Needle-Free Valve and Check Valve

## User Information

### FOUR EASY STEPS:

#### 1. PREPARE BAG

- \* Remove protective tab from the bag.



#### 2. INSERT SPIKE

- \* Remove protective cap and insert bag spike into bag.



#### 3. PREPARE VALVE

- \* Swab the top of the SmartSite® Needle-Free Valve port with preferred antiseptic prior to accessing.



#### 4. ATTACH SYRINGE

- \* Connect the syringe to the valve and withdraw the desired amount of solution.



Model 2309E

### HELPFUL HINTS:

*Note: The check valve prevents medication additions to the bag.*

- Do not use bag spike to access more than one bag.
- Always swab the top of the SmartSite® Valve port prior to accessing.
- For set replacement interval, refer to facility protocol and/or government standards (such as CDC Guidelines in the United States).

CareFusion Australia  
Ph: 1800 833 372

CareFusion New Zealand  
Ph: 0508 422 734



CareFusion