

Chloraprep® 26mL Applicator

Patient Preoperative Skin Preparation
2% chlorhexidine gluconate (CHG) & 70% isopropyl alcohol (IPA)



Application instructions



Pinch

- Hold the applicator as shown; do not touch sponge
- Pinch lever **once** to activate applicator and release antiseptic



Apply

- Allow solution to partially load in sponge; gently press applicator against treatment area to evenly distribute solution throughout sponge
- Once solution is visible on the skin, use gentle back-and-forth strokes to prep the incision site(s) for at least 30 seconds, then continue back-and-forth strokes, progressing from the incision site to the periphery of the surgical site:
 - **For dry sites** (e.g., abdomen or arm): Use repeated back-and-forth strokes for 30 seconds
 - **For moist sites** (e.g., inguinal fold or axilla): Use repeated back-and-forth strokes for 2 minutes



Dry

- Allow area to air dry for approximately 3 minutes for dry and moist sites
- Do not blot or wipe solution away
- Solution must be dry for optimal drape/dressing adhesion

Before using the Chloraprep® 26mL applicator, please read the instructions on the package. Use in accordance with the policies and procedures of your hospital.



Catalog numbers:

- 260800 Clear
- 260815 Hi-Lite Orange®
- 260825 Scrub Teal®

Approximate coverage area: 13.2" x 13.2"
25 applicators per case

Safety points for products containing alcohol:

- Do not drape or use ignition source until solution is completely dry (minimum of 3 minutes on hairless skin)
- Hairy areas may take up to 1 hour to dry
- Do not allow the solution to pool
- Remove wet materials from prep area

The Chloraprep® tint

- The *Hi-Lite Orange*® tint in Chloraprep is Food, Drug and Cosmetic (FD&C) Yellow #6 dye and the *Scrub Teal*® tint is FD&C Green #3. Both are "Generally Recognized As Safe" (GRAS) by the Food and Drug Administration (FDA). The tint allows for easy visualization on a variety of skin tones.
- Post procedure, the tint will slowly fade from the skin
 - If tint removal is desired, saline, soap and water, alcohol or a Chloraprep clear applicator may be used

To provide continued antimicrobial activity, it is recommended that Chloraprep remain on the skin, especially at the incision site, post procedure

Additional information:

- Discard the applicator after a single use
- Latex-free
- For external use only
- Do not use on children less than 2 months of age
- Do not use 26mL applicator for head and neck surgery
- Do not use on patients with known allergies to CHG/IPA
- Keep solution out of eyes, ears and mouth
- Store between 15-30°C (59-86°F)
- Avoid freezing and excessive heat above 40°C (104°F)
Store within recommended conditions to maintain appearance of tint and efficacy

Chloraprep representative: _____

In-service dates: _____

Time: _____

REORDER #: _____

Location of Chloraprep applicators at this facility: _____

chloraprep.com | 800.523.0502

Chloraprep®

Patient Preoperative Skin Preparation
2% chlorhexidine gluconate (CHG) & 70% isopropyl alcohol (IPA)

© 2011 CareFusion Corporation or one of its subsidiaries. All rights reserved.
Chloraprep, Hi-Lite Orange and Scrub Teal are registered trademarks of CareFusion Corporation or one of its subsidiaries. MRK-11042

