

Cat. 4420

# Wet PVP-I Small Procedure Preoperative Skin Prep Tray

Non-sterile Solution • Applicator is sterile if package is intact



59 mL Bottle of Povidone Iodine Cleansing Solution (Scrub) USP 7.5% and 59 mL  
Bottle of Povidone Iodine Topical Solution (Paint) USP 10% with Powder-Free Gloves

For Single Use Only

STERILE EO



## Contents

### Sterile

- Two Cotton-Tipped Swabs
- Three Foam-Winged Sponge Applicators
- Two Sponge Stick Applicators
- Two Blotting Towels
- Two Towels/Drapes
- One Pair Synthetic, Powder-Free Gloves in Wallet

### Overwrap

### Non-Sterile

- One Bottle Povidone Iodine Cleansing Solution USP 7.5% 2 oz. (Scrub)
- One Bottle Povidone Iodine Topical Solution USP 10% 2 oz. (Paint)

## Directions for Use

1. Open plastic bag, remove contents, and unfold overwrap.
2. Put on gloves.
3. Scrub area, per labeled directions, using cleansing solution.
4. Blot with sterile towel.
5. Paint area, per labeled directions, using topical solution.
6. Remove excess or pooled solution and soaked materials.
7. Discard all materials after use.

Store at room temperature. Avoid excessive heat (not to exceed 104°F/40°C). Protect from freezing.

Lot No.

Exp.

U.S. components packaged in Mexico except as indicated:

Gloves and Cotton-Tipped Swabs  
Made in China

Foam-Winged Sponge Applicators and  
Sponge Stick Applicators  
Made in Mexico

## CareFusion

Vernon Hills, IL 60061 USA

36-10842



(01)10885403142185