

Cat. 4469A

Wet PVP-I Vaginal Preoperative Skin Prep Tray

Non-sterile Solution • Applicator is sterile if package is intact



4 oz. Bottle of Povidone Iodine Cleansing Solution (Scrub) USP 7.5% and 4 oz.
Bottle of Povidone Iodine Topical Solution (Paint) USP 10% with Powder-Free Gloves
For Single Use Only



STERILE EO

Contents

Sterile

Two Cotton-Tipped Swabs
Six Foam-Winged Sponge Applicators
Three Sponge Stick Applicators
Three Contoured Sponge Stick Applicators
Two Blotting Towels
Two Towels/Drapes
One Removable Basin Prep
One Pair Synthetic, Powder-Free Gloves in
Wallet
Overwrap

Non-sterile

One Bottle Povidone Iodine Cleansing Solution
USP 7.5% 4 oz. (Scrub)
One Bottle Povidone Iodine Topical Solution
USP 10% 4 oz. (Paint)

Directions for Use

1. Open plastic bag, remove contents, and unfold overwrap.
2. Put on gloves.
3. Scrub area, per labeled directions, using cleansing solution.
4. Blot with sterile towel.
5. Paint area, per labeled directions, using topical solution.
6. Remove excess or pooled solution and soaked materials.
7. Discard all materials after use.

Store at room temperature. Avoid excessive heat (not to exceed 104°F/40°C). Protect from freezing.

Lot No.

Exp.

U.S. components packaged in Mexico
except as indicated:

Gloves and Cotton-Tipped Swabs
Made in China

Foam-Winged Sponge Applicators, Sponge Stick
Applicators and Contoured Sponge Stick Applicators
Made in Mexico

CareFusion

Vernon Hills, IL 60061 USA

36-10844



(01)10885403184482