

Cat. 4489

# Wet Exidine<sup>®</sup>-4 (CHG 4%) Preoperative Skin Prep Tray

Non-sterile Solution • Applicator is sterile if package is intact



With Powder-Free Gloves, Foam Sponges and  
4 oz. Bottle of Chlorhexidine Gluconate 4% Solution

For Single Use Only

STERILE EO



## Contents

### Sterile

Two Cotton-Tipped Swabs

Six Urethane Foam-Winged Sponge Applicators

Two Blotting Towels

Two Patient Underdrapes

One Pair Synthetic, Powder-Free Gloves in  
Wallet

Overwrap

### Non-sterile

One Bottle Exidine<sup>®</sup>-4 Scrub Solution Contains  
4% w/w Chlorhexidine Gluconate Solution

## Directions for Use

1. Open plastic bag and remove wrapped tray.
2. Unfold overwrap to form sterile field.
3. Put on sterile gloves provided.
4. Position two underdrapes where desired.
5. Open bottle of Exidine<sup>®</sup>-4 and pour solution into basin.
6. Apply Exidine<sup>®</sup>-4 liberally to surgical site and swab for at least 2 minutes. Dry with a sterile towel. Repeat procedure for an additional 2 minutes and dry with a sterile towel.

Store at room temperature. Avoid excessive heat (not to exceed 104°F/40°C). Protect from freezing.

Lot No.

Exp.

U.S. components packaged in Mexico except as indicated:

Gloves and Cotton-Tipped Swabs  
Made in China

Foam-Winged Sponge Applicators  
Made in Mexico

### CareFusion

Vernon Hills, IL 60061 USA

36-10806



(01)10885403142178