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
Revisions

BALTSO0191 Version 10.0 Template 4
Inserts

Rev from	Rev to	JOB #
02	03	8916-16

NOTES:

- BD Catalog Number: 440928
- Blank (Sheet) Size: Length: 8 1/2" Width: 33"
- Number of Pages: 12 Number of Sheets: 1
- Page Size: Length: 8 1/2" Width: 5 1/2" Final Folded Size: 4 1/4" x 5 1/2"
- Ink Colors: No. of Colors: 1 PMS#: Standard Black
- Printed two sides: Yes ☒ No ☐
- Style (see illustrations below): # 4

- Vendor Printed ☒ Online/In House Printed ☐ Web ☐
- See specification control no. 8012543 for material information.
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Sonia Thompson at 11:48 am, Dec 06, 2016	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 BD Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	PROOFING APPROVED BY By Natalie Morio at 2:13 pm, Dec 07, 2016		
Checked By	THIRD EYE BY By Sarah Henderson at 2:35 pm, Dec 07, 2016		
Part Number:	8012543	Category and Description Package Insert, BD ProbeTec™ Urine Preservative Transport Kit	Sheet: 1 of 13 Scale: N/A A

BD ProbeTec™ Urine Preservative Transport Kit

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INTENDED USE

The **BD ProbeTec™ Urine Preservative Transport Kit** with **NAP Guard™** technology is designed to preserve and transport *Chlamydia trachomatis* and *Neisseria gonorrhoeae* in male and female urine specimens from symptomatic and asymptomatic individuals prior to processing for analysis with the **BD ProbeTec ET C. trachomatis** (CT) and *N. gonorrhoeae* (GC) Amplified DNA Assays.

SUMMARY AND EXPLANATION

Nucleic acid amplification methods offer advantages over non-amplified methods including increased sensitivity and testing from a variety of sample types. Testing for *C. trachomatis* and *N. gonorrhoeae* in screening programs and diagnostic situations is not always possible immediately after specimen collection. The **BD ProbeTec Urine Preservative Transport (UPT)** with **NAP Guard** technology allows for an extended range of time and temperature conditions for specimen storage and transport.

PRINCIPLES OF THE PROCEDURE

The **BD ProbeTec Urine Preservative Transport (UPT)** with **NAP Guard** technology allows for an extended range of time and temperature conditions for specimen storage and transport when testing for the presence of *Chlamydia trachomatis* and *Neisseria gonorrhoeae* in urine specimens using the **BD ProbeTec ET CT** and GC Amplified DNA Assays.

REAGENTS

Materials Provided: Each **BD ProbeTec Urine Preservative Transport Kit** contains 100 UPT tubes and 100 transfer pipettes. Each UPT contains 50 µL of **NAP Guard** (≥742.5 mM K₂EDTA).

Materials Required But Not Provided: Sterile, plastic, preservative-free, urine collection cups and gloves.

Storage Instructions: Store kit at 2–33 °C. Do not use past the expiration date.

Warnings and Precautions:

For *in vitro* Diagnostic Use.

1. Pathogenic microorganisms, including hepatitis viruses and Human Immunodeficiency Virus, may be present in clinical specimens. "Standard Precautions"¹⁻⁴ and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids.
2. The Urine Preservative Transport Kit is for use with male and female urine specimens to be tested using only the **BD ProbeTec ET C. trachomatis** (CT) and *N. gonorrhoeae* (GC) Amplified DNA Assays. Do not use the UPT kit with other amplified DNA assays.
3. Optimal performance of the **BD ProbeTec ET C. trachomatis** (CT) and *N. gonorrhoeae* (GC) Amplified DNA Assays requires proper specimen collection, handling and transport.

SPECIMEN COLLECTION AND TRANSPORT

Urine Specimen Collection

1. The patient should not have urinated for at least 1 h prior to specimen collection.
2. Collect the specimen in a sterile, plastic, preservative-free specimen collection cup.
3. The patient should collect the first 15–60 mL of voided urine (the first part of the stream – NOT midstream) into a urine collection cup.
4. Label with patient identification and date/time collected.

Urine transfer to the UPT

NOTES:

Urine should be transferred from the collection cup to the UPT within 8 h of collection provided the urine has been stored at 2–30 °C. Urine can be held for up to 24 h prior to transfer to the UPT provided that the urine has been stored at 2–8 °C. Wear clean gloves when handling the UPT and the urine specimen. If gloves come in contact with the specimen, immediately change gloves to prevent contamination of other specimens.

1. Remove the UPT and the transfer pipette from their packaging. Label the UPT with the patient identification and date/time collected.
2. Hold the UPT upright and firmly tap the bottom of the tube on a flat surface to dislodge any large drops from inside the cap. Repeat if necessary.
3. Uncap the UPT and use the transfer pipette to transfer urine into the tube. The correct volume of urine has been added when the fluid level is between the black lines on the fill window located on the UPT label. This volume corresponds to approximately 2.5–3.45 mL of urine. DO NOT overflow or under fill the tube.
4. Discard the transfer pipette. NOTE: The transfer pipette is intended for use with a single specimen.
5. Tighten the cap securely on the UPT.
6. Invert the UPT 3–4 times to ensure that the specimen and reagent are well mixed.

UPT Storage and Transport

Store and transport urine specimens in UPT at 2–30 °C and process within 30 days of collection. Specimens may also be stored at -20 °C for up to two months.

For domestic and international shipments, specimens should be packaged and labeled in compliance with applicable state, federal, and international regulations covering the transport of clinical specimens and etiologic agents/infectious substances. Time and temperature conditions for storage must be maintained during transport.

Shipping studies have been performed with seeded urine specimens to demonstrate that temporary exposure to extreme temperatures (-20 °C and 60 °C) during transport does not reduce UPT product performance.

REFERENCES

1. Clinical and Laboratory Standards Institute. 2005. Approved Guideline, M29-A3. Protection of laboratory workers from occupationally acquired infections 3rd ed. CLSI, Wayne, PA.
2. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53-80.
3. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories. HHS Publication (CDC), 5th edition. U.S. Government Printing Office, Washington, D.C.
4. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 021-0045.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

BD ProbeTec Urine Preservative Transport Kit

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Benex Limited