BRUKSOMRÅDE

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261510 50 BD Bio-Bag Type C-kamre, 50 sett per kartong / 50 generatorer.

REFERANSER

1. Kellogg, D.S. Jr., Holmes, K., Hill, G.A., 1976. Cumitech 4, Laboratory diagnosis of gonorrhoeae. Coordinating eds., S. Marcus, J.C. Sherris. American Society for Microbiology, Washington, D.C.
2. Morello, J.A., Bohnhoff, M.: *Neisseria* and *Branhamella*, p. 111-130. In E.H. Lennette, A. Balows, W.J. Hausler, J.P. Truant (eds.), Manual of clinical microbiology, 3rd ed. American Society for Microbiology, Washington, D.C., 1980.
3. Bannatyne, R.M., Clausen, C., McCarthy, L.R., 1979. Cumitech 10, Laboratory diagnosis of upper respiratory tract infections. Coordinating ed., I.B.R. Duncan. American Society for Microbiology, Washington, D.C.
4. Kilian, M., *Haemophilus*, p. 330. In E.H. Lennette, A. Balows, W.J. Hausler, J.P. Truant (eds.), Manual of clinical microbiology, 3rd ed., American Society for Microbiology, Washington, D.C., 1980.
5. Carlson, B.L., Haley, M.S., Tisei, N.A., McCormack, W.M., 1980. Evaluation of four methods for isolation of *Neisseria gonorrhoeae*. J. Clin. Microbiol. 12:301-303.
6. Clinical and Laboratory Standards Institute. 2005. Approved guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, Pa.
7. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53-80.
8. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
9. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official journal L262, 17/10/2000, p. 0021-0045

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For US: "For Investigational Use Only"

Lower limit of temperature / Долен лимит на температурата / Dolní hranice teploty / Nedre temperaturgrænse / Temperaturuntergrenze / Κατώτερο όριο θερμοκρασίας / Limite inferior de temperatura / Alumine temperatuuriipiir / Limite inférieure de température / Najniža dozvoljena temperatura / Alsó hőmérsékleti határ / Limite inferiore di temperatura / Температураның төменгі рұқсат шері / 하한 온도 / Žemiausia laikymo temperatūra / Temperatūras zemākā robeža / Laagste temperatuurilimiet / Nedre temperaturgrænse / Dolna granica temperatury / Limite minimo de temperatura / Limitā minimā de temperaturā / Нижний предел температуры / Spodná hranica teploty / Donja granica temperature / Nedre temperaturgräns / Sicaklık alt sınırı / Мінімальна температура / 温度下限

Control / Контролно / Kontrola / Kontrol / Kontrolle / Μάρτυρας / Kontroll / Contrôle / Contollo / Бақылау / 컨트롤 / Kontrolè / Kontrolle / Controle / Controlo / Контроль / kontroll / Контроль / 对照

Positive control / Положительный контроль / Pozitívni kontrola / Positiv kontrol / Positive Kontrolle / Θετικός μάρτυρας / Control positivo / Positiivne kontroll / Contrôle positif / Pozitivna kontrola / Pozitiv kontrol / Controllo positivo / Оң бақылау / 양성 컨트롤 / Teigiama kontrolė / Pozitivná kontrola / Positive controle / Kontrola dodatna / Controllo positivo / Control pozitiv / Положительный контроль / 양성 kontrol / Позитивный контроль / 阳性对照试剂

Negative control / Отрицателен контрол / Negativní kontrola / Negativ kontrol / Negative Kontrolle / Αρνητικός μάρτυρας / Control negativo / Negatiivne kontroll / Contrôle négatif / Negativa kontrola / Negativ kontrol / Controlo negativo / Негативтік бақылау / 음성 컨트롤 / Neigiama kontrolė / Negativa kontrol / Negative controle / Kontrola ujemna / Controlo negativo / Control negativ / Отрицательный контроль / Negatív kontrol / Негативный контроль / 阴性对照试剂

Method of sterilization: ethylene oxide / Μέθοδος στειλίωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseringsmethode: ethyleenoxid / Sterilisationsmethode: Ethylenoxid / Μεθόδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Sterilisierungsmethode: ethylénoxiid / Méthode de stérilisation : oxyde d'éthylène / Metóda sterilizácie: etilen oksíd / Szterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Sterilizacija edici – этилен тотыгы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizācijas metode: etilēnoksīds / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringssysteem: ethyleenoxid / Metóda sterilizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metodă de sterilizare: oxid de etilenă / Metód sterilizacijske: etilénoksid / Metóda sterilizácie: etilén oksíd / Metóda sterilizacije: etilen oksid / Steriliseringsmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Metód sterilizáciú: etilénoksidom / 灭菌方法: 环氧乙烷

Method of sterilization / Metoda na sterilizacija: iradiacija / Způsob sterilizace: záření / Steriliseringmetode: bestråling / Sterilisationsmethode: Bestrahlung / Μέθοδος αποστείρωσης: ακτινοβολία / Método de esterilización: irradiación / Steriliseringmismeetod: kiirgus / Méthode de stérilisation : irradiation / Metoda sterilizacije: zračenje / Sterilizálás módszere: besugárzás / Metodo di sterilizzazione: irradiazione / Стерилизація: опромінення / 소독 방법: 방사 / Sterilizavimo būdas: radiacija / Sterilizēšanas metode: apstarošana / Gesteriliseerd met behulp van bestraling / Steriliseringmetode: bestråling / Metoda sterylizacji: napromienianie / Método de esterilização: irradiação / Metoda de sterilizare: iradiere / Метод стерилизации: облучение / Metóda sterilizácie: ozarovanie / Metoda sterilizacije: ozračevanje / Steriliseringsmetod: strålning / Sterilizasyon yöntemi: ırdıyasyon / Метод стерилизації: опромінення / 灭菌方法: 辐射

Biological Risks / Биологични рискове / Biológická rizika / Biologisk fare / Biogefährdung / Βιολογικοί κίνδυνοι / Riesgos biológicos / Biolooglised riskid / Risques biologiques / Biološki rizik / Biológiai veszélyes / Rischio biologico / Биологиялык тәуекелдер / 생물학적 위험 / Biologinis pavojus / Biologiskie riski / Biologisch risico / Biologisk risiko / Zagrożenia biologiczne / Perigo biológico / Riscuri biologice / Биологическая опасность / Biologické riziko / Biološki rizici / Biologisk risk / Biolojik Riskler / Биологічна небезпека / 生物学风险

Caution, consult accompanying documents / Внимание, направте справка в придружаващите документи / Pozor! Pristudujte si příloženou dokumentaci! / Forsigtig, se ledsagede dokumenter / Achtung, Begleitedokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatus! Lugea kaasvatel dokumentatsiooni / Attention, consulter les documents joints / Upporejenje, koristiti prateću dokumentaciju / Fügelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайланыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dėmesio, žiūrėkite pridėjusiam dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atentie, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, bilirlicte verilen belgeler basyurun / Увага: див. супутню документацию / 小心, 请参阅附带文档。

Upper limit of temperature / Горен лимит на температурата / Horní hranice teploty / Øvre temperaturgrænse / Temperaturobergrenze / Ανώτερο όριο θερμοκρασίας / Limite superior de temperatura / Ülemine temperatuuripiir / Limite supérieure de température / Gornja dozvoljena temperatura / Felső hőmérsékleti határ / Limite superiore di temperatura / Температураның рұқсат етілген жоғарғы шегі / 상한 온도 / Aukščiausia laikymo temperatūra / Augšējā temperatūras robeža / Hoogste temperatuurlimiet / Øvre temperaturgrense / Górná granica temperatury / Limite máximo de temperatura / Limită maximă de temperatură / Верхний предел температуры / Horná hranica teploty / Gornja granica temperature / Øvre temperaturgräns / Sicaklık üst sınırı / Максимальная температура / 温度上限

Keep dry / Пазете сухо / Skladujte v suchém prostředí / Opbevaras tørt / Trocklagern / Φυλάξτε το στεγνό / Mantener seco / Hoida kuivas / Conserver au sec /
Držati na suhom / Száraz helyen tartandó / Tenere all'asciutto / Құрақ күйінде ұста / 건조 상태 유지 / Laikykite sausi / Uzglabāt sausu / Droog houden / Holdes
tørt / Przechowywać w stanie suchym / Manter seco / A se feri de umezeală / Не допускать попадания влаги / Uchovávať v suchu / Držite na suwom mestu /
Förvaras tørt / Kuru bir şekilde muhafaza edin / Беретти күй вологи / 请保持干燥

Collection time / Време на събиране / Čas odběru / Opsamlingstidspunkt / Entnahmehurzeit / Ώρα συλλογής / Hora de recogida / Kogumisaeg / Heure de prélèvement / Sati prikupljanja / Mintavétel időpontja / Ora di raccolta / Жинау уакыты / 수집 시간 / Paėmimo laikas / Savākšanas laiks / Verzameltijd / Tid prøvetaking / Godzina pobrania / Hora de colheita / Ora colectării / Время сбора / Doba odberu / Vreme prikupljanja / Uppsamlingsstid / Toplama zamanı / Час забори / 采集时间

Peel / Obenete / Otevřete zde / Åbn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ўстіңі қабатын алып таста / 벗기 / Plešti čia / Atļimēt / Schillen / Trekk av / Oderwać / Destacar / Se dezlipeste / Отклеить / Odtrhnite / Oluštiti / Dra isår / Avrima / Biskleiti / 撕下

Perforation / Перфорация / Perforace / Perforering / Διήτρηση / Perforación / Perforatsioon / Perforacija / Perforálás / Perforazione / Тесик тесу / 절취선 / Perforacia / Perforácia / Perforatie / Perforacja / Perfuracão / Perforare / Перфорация / Perforácia / Perforasyon / Перфорация / 穿孔

Do not use if package damaged / Не использовать, если упаковка повреждена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhal beschädigter Packung nicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιά. / Nao usar si e paquete está danado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használja, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Eter paketi byzurnyly bonca, paldanalanba / पैकेटिंग को 손상लत त्वा सारु क्मि / Jeśli pakując pazeista, nenaoditi / Nelietoti, ja iepakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Nao usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / Ne ispolzovati pri povredjenju pakovanja / Nepoužívať, ak je obal poškodený / Ne koristite ako je pakovanje oštećeno / Ävänd ej om förpackningen är skadad / Ambalaj hasır görtmüße kullanılmayın / Не використовувати за пошкодженої упаковки / 如果包裝破損，請勿使用

Keep away from heat / Παζετε от топлина / Nevystavujte přilišnému teplu / Må ikke udsættes for varme / Vor Wärme schützen / Κρατήστε το μακριά από τη θερμότητα / Mantener alejado de fuentes de calor / Hoida eemal valgusest / Protégée de la chaleur / Držati dalje od izvora topline / Ojva a melegtől / Tenere lontano dal calore / Салтөн жерде сакта / 열을 피해야 함 / Laikyti atokiau nuo šilumos šaltinių / Sargāt no karstuma / Beschermen tegen warmte / Må ikke utsettes for varme / Przechowywać z dala od źródeł ciepła / Manter ao abrigo do calor / A se feri de căldură / Не награвяте / Uchowávaite mimo zdroja tepla / Držite dalje od

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland

Australian Sponsor:

Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

JEMBEC is a trademark of Miles Scientific.

BD, the BD Logo and Bio-Bag are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2019 BD. All rights reserved.