Part Number:	L009617	BALTSO0191 Version 13.0 Template 4 Inserts	
Category and Description:	Package Insert BD BBL™ Single Polyester Fiber-Tipped Applicator Swab	Rev from: 02	Rev to: 03
		Job Number:	1219-19

Catalog Number: 220690

Blank (Sheet) Size: Length: 25.5" Width: 11"

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
Number of Pages: 6 Number of Sheets: 1

Page Size: Length: 11" Width: 8.5" Final Folded Size: 8.5" X 11"

Ink Colors: Number of Colors: 1 PMS #: Standard Black

Printed Two Sides: Yes: ☒ No: ☐

Style (see illustrations below): # 4

Vendor Printed: ☒ Online / In House Printed: ☐ Web Printed: ☐

Material Specification controlled by OEM Vendor.

Becton, Dickinson and Company
 7 Loveton Circle
 Sparks, MD 21152 USA

Company confidential. This document is the property of Becton, Dickinson and Company and is not to be used outside the company without written permission. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Revised By: **REVISED BY**
 By Paul Meyers at 8:09 am, Jul 10, 2019

Proofing Approved By: **PROOFING APPROVED BY**
 By Sarah Henderson at 1:15 pm, Jul 15, 2019

Third Eye By: **THIRD EYE BY**
 By Paul Meyers at 7:58 am, Jul 17, 2019

BD BBL™ Collection and Transport Systems

Single Polyester Fiber-Tipped Applicator Swabs

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INTENDED USE

The **BD BBL™** applicator swab is for specimen collection and transport. Reports indicate the recovery of certain organisms, especially beta-hemolytic streptococci, may be improved by collection of specimens with the polyester fiber-tipped applicator.¹

SUMMARY AND EXPLANATION

The **BD BBL** Single Polyester Fiber-Tipped Applicator Swab is approximately 15.2 cm (6") in length. Applicators are sterile and individually packaged in Peel-Apart packages.

Warnings and Precautions:

For *in vitro* Diagnostic Use.

Do not use unless immediate package is intact.

⊗ **BD BBL** Applicator Swab is for single use only; reuse may cause a risk of infection and/or inaccurate results.

SPECIMEN COLLECTION AND TRANSPORT

Pathogenic microorganisms, including hepatitis viruses and Human Immunodeficiency Virus, may be present in clinical specimens. "Standard Precautions"²⁻⁵ and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids. Prior to discarding, sterilize specimen containers and other contaminated materials by autoclaving.

A number of general considerations should be observed regarding the collection and handling of clinical specimens. Whenever possible, the laboratory should provide suitable collection containers and instructions for their proper use.

Specimens should be obtained before antimicrobial agents have been administered. If therapy was initiated prior to collection of the specimen, this should be noted on specimen containers or on forms accompanying the specimen. Material should be collected from the site where the suspected organism is most likely to be found, with as little external contamination as possible, and in a clinical situation at the stage of disease most likely to yield a positive culture. Specimens should be of sufficient quantity to permit complete examination. To prevent overgrowth by contaminants and to ensure that pathogens are viable upon receipt, provision must be made for prompt delivery to the laboratory. Specimen containers should be properly labeled with all pertinent information such as name of patient, time specimen was taken, etc. Whenever possible, specimens should be inoculated directly onto appropriate media. When this is not possible, an appropriate transport medium may be used. However, some change in the numbers of viable organisms may occur during transport.

The laboratory must be furnished sufficient clinical information to enable the microbiologist to select the most suitable media and appropriate techniques. If at all possible, specimens should be transported directly to the laboratory. For more detailed information, appropriate references should be consulted.^{6,7}

PROCEDURE

Material Provided: **BD BBL** Single Polyester Fiber-Tipped Applicator Swab.

Materials Required But Not Provided: Culture media, reagents and laboratory equipment as required for the procedures and tests to be performed on the specimens.

Instructions:

Collection of Clinical Specimens

1. Peel open pouch, remove applicator swab and collect specimen.
2. Place swab in a tube for transport to the laboratory. Tubes for transport contain suitable transport or culture media; e.g., Amies, Stuart or Cary-Blair transport media, **BD Port-A-Cul™**, **BD Trypticase™** Soy Broth or other medium of choice. Alternatively, streak plates directly. If the presence of anaerobic organisms is suspected, immediately inoculate **BD BBL** Thioglycollate Medium without Indicator –135C in glass screw-capped tube.
3. When a direct smear is to be prepared, in addition to a culture, it is suggested that two applicators be employed. One is smeared over the marked area on a clean, dry slide for staining; the other is used for culturing the specimen.

Handling of Swabs in the Laboratory:

1. If a swab applicator is received dry, as soon as possible, immerse swab into a suitable sterile solution, i.e., broth, and then inoculate appropriate culture media.
2. If swab applicator is received in a transport medium, as soon as possible, inoculate into or on appropriate culture media.
3. After inoculation, prepare a slide for staining of clinical specimens and then discard applicator. Dispose of used applicators by autoclaving or incinerating.

AVAILABILITY

Cat. No. Description

220690 **BD BBL™** Single Polyester Fiber-Tipped Applicator Swab,
Ctn. of 1000

REFERENCES

1. Ellner, P.D. and C.J. Ellner. 1966. Survival of bacteria on swabs. J. Med. Tech. 91: 905.
2. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, Pa.
3. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. Infect. Control Hospital Epidemiol. 17:53–80.
4. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
5. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). Official Journal L262, 17/10/2000, p. 0021–0045.
6. Isenberg, H.D., F.D. Schoenkecht, and A. von Graevenitz. 1979. Cumitech 9, Collection and processing of bacteriological specimens. Coordinating ed., S.J. Rubin. American Society for Microbiology, Washington, D.C.
7. Miller, J.M., and H.T. Holmes. 1995. Specimen collection, transport, and storage, p. 19–32. In P.R. Murray, E.J. Baron, M.A. Pfaller, F.C. Tenover, and R.H. Tenover (ed.), Manual of clinical microbiology, 6th ed. American Society for Microbiology, Washington, D.C.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Becton Dickinson France S.A.S.
11 Rue Aristide Bergès
38800 Le Pont de Claix, France

In Vitro Diagnostic Medical Device / Медицински уред за диагностика ин витро / Lékařské zařizení určené pro diagnostiku in vitro / In vitro diagnostisk medicinsk anordning / Medizinisches In-vitro-Diagnostikum / In vitro διαγνωστική ιατρική συσκευή / Dispositivo médico para diagnóstico in vitro / In vitro diagnostika meditsiiniaparatuur / Dispositif médical de diagnostic in vitro / Medicinska pomagala za In Vitro Dijagnostiku / In vitro diagnosztikai orvosi eszköz / Dispositivo medicale per diagnostica in vitro / Жасанды жағдайда жүзінетін медициналық диагностика аспабы / In Vitro Diagnostic 의료 기기 / In vitro diagnostikos prietaisais / Medicīnas ierīces, ko lieto in vitro diagnostikā / Medisch hulpmiddel voor in-vitro diagnostiek / In vitro diagnostisk medisinsk utstyr / Urządzenie medyczne do diagnostyki in vitro / Dispositivo médico para diagnóstico in vitro / Dispozitiv medical pentru diagnostic in vitro / Медицинский прибор для диагностики ин витро / Medicinska pomůcka na diagnostiku in vitro / Medicinski uređaj za in vitro dijagnostiku / Medicinteknisk produkt för in vitro-diagnostik / In Vitro Diyagnostik Tibbi Cihaz / Медичний пристрій для діагностики in vitro / 体外诊断医疗设备

Batch Code (Lot) / Код на партидата / Kód (číslo) šarže / Batch-kode (lot) / Batch-Code (Charge) / Κωδικός παρτίδας (παρτίδα) / Código de lote (lote) / Partii kood / Numéro de lot / Lot (kod) / Tétel száma (Lot) / Codice batch (lotto) / Топтама коды / 배치 코드(로트) / Partijos numeris (LOT) / Partijas kods (laidiens) / Lot nummer / Batch-kode (parti) / Kod partii (seria) / Código do lote / Cod de serie (Lot) / Код партии (лот) / Kód série (šarža) / Kod serie / Partinumner (Lot) / Parti Kodu (Lot) / Код партии / 批号 (亚批)

Contains sufficient for <n> tests / Съдържащието е достатъчно за <n> теста / Dostatečné množství pro <n> testů / Indeholder tilstrækkeligt til <n> tests / Ausreichend für <n> Tests / Περιέχει επαρκή ποσότητα για <n> εξετάσεις / Contenido suficiente para <n> pruebas / Küllaldane <n> testide jaoks / Contenu suffisant pour <n> tests / Sadržaj za <n> testova / <n> teszthez elegendő / Contenuto sufficiente per <n> test / <n> тесттері үшін жеткілікті / <n> 테스트가 충분히 포함됨 / Pakankamas kiekis atlikti <n> testų / Satur pietiekami <n> pārbaudēm / Inhoud voldoende voor "n" testen / Innholder tilstrekkelig til <n> tester / Zawiera ilość wystarczającą do <n> testów / Conteúdo suficiente para <n> testes / Conținut suficient pentru <n> teste / Достаточно для <n> тестов(а) / Obsah vystačí na <n> testov / Sadržaj dovoljan za <n> testova / Innehåller tillräckligt för <n> analyser / <n> test için yeterli miktarda içerir / Вистачить для аналізу: <n> / 足够进行 <n> 次检测

Consult Instructions for Use / Направете справка в инструкциите за употреба / Prostudujte pokyny k použití / Se brugsanvisningen / Gebrauchsanweisung beachten / Συμβουλευτείτε τις οδηγίες χρήσης / Consultar las instrucciones de uso / Lugeda kasutusjuhendit / Consulter la notice d'emploi / Koristi upute za upotrebu / Olvassa el a használati utasítást / Consultere le istruzioni per l'uso / Пайдалану нұсқаулығымен танысып алыңыз / 사용 지침 참조 / Skaitkite naudojimo instrukcijas / Skatīt lietošanas pamācību / Raadpleeg de gebruiksaanwijzing / Se i brugsanvisningen / Zobacz instrukcja użytkowania / Consultar as instruções de utilização / Consultați instrucțiunile de utilizare / См. руководство по эксплуатации / Pozri Pokyny na používanie / Pogledajte uputstvo za upotrebu / Se brugsanvisningen / Kullanım Talimatları'na başvurun / Див. інструкції з використання / 请参阅使用说明

Peel / Обелете / Otevfete zde / Åbn / Abziehen / Αποκολλήστε / Desprender / Koorida / Décoller / Otvoriti skini / Húzza le / Staccare / Ўстінгі қабатын алып таста / 벗기 기 / Plēsti čia / Atīmēt / Schillen / Trekk av / Oderwač / Destacar / Se dezlipeste / Отклейти / Odrhňte / Oljuštiti / Dra isär / Ayırma / Відклеїти / 撕下

Method of sterilization: ethylene oxide / Метод на стерилизация: етиленов оксид / Způsob sterilizace: etylenoxid / Steriliseringmetode: ethylenoxid / Sterilisationsmethode: Ethylenoxid / Μέθοδος αποστείρωσης: αιθυλενοξείδιο / Método de esterilización: óxido de etileno / Steriliseerimismetode: etüleenoksiid / Méthode de stérilisation : oxyde d'éthylène / Metoda sterilizacije: etilen oksid / Sterilizálás módszere: etilén-oxid / Metodo di sterilizzazione: ossido di etilene / Стерилизация әдісі – этилен тотығы / 소독 방법: 에틸렌옥사이드 / Sterilizavimo būdas: etileno oksidas / Sterilizēšanas metode: etilēnoksids / Gesteriliseerd met behulp van ethyleenoxide / Steriliseringsmetode: etylenoksids / Metoda sterylizacji: tlenek etylu / Método de esterilização: óxido de etileno / Metoda de sterilizare: oxid de etilenă / Метод стерилизації: етиленоксид / Metoda sterilizácie: etylénoxid / Metoda sterilizacije: etilen oksid / Steriliseringsmetod: etenoxid / Sterilizasyon yöntemi: etilen oksit / Метод стерилизації: етиленоксидом / 灭菌方法: 环氧乙烷

This only applies to US: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / S'applique uniquement aux États-Unis: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Vale solo per gli Stati Uniti: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Gilt nur für die USA: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Sólo se aplica a los EE.UU.: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner."