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
Revisions

BALTSO0191 Version 11.0 Template 4
Inserts

Rev from	Rev to	JOB #
02	03	8633-16

Notes:

- BD Catalog Number: 441358
- Blank (Sheet) Size: Length: 33" Width: 8.5"
- Number of Pages: 16 Number of Sheets: 4
- Page Size: Length: 8.5" Width: 5.5" Final Folded Size: 8.5" x 5.5"
- Ink Colors: No. of Colors: 2 PMS#: Standard Black; 032 Red
- Printed two sides: Yes ☒ No ☐
- Style (see illustrations below): # 4

- Vendor Printed ☒ Online/In House Printed ☐ Web ☐
- See specification control no. D16354RM for material information.
- Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Nancy Carlsen at 10:02 am, Dec 01, 2017	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 BD Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	PROOFING APPROVED BY By Natalie Morio at 8:39 am, Dec 04, 2017		
Checked By	THIRD EYE BY By Nichole Graham at 10:17 am, Dec 05, 2017		
Part Number:	L009851	Category and Description Package Insert, Male Urethral Specimen Kit for the BD ProbeTec™ Qx Amplified DNA Assays	Sheet: 1 of 17 Scale: N/A A

BD Male Urethral Specimen Collection Kit for the BD ProbeTec™ Qx Amplified DNA Assays

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INTENDED USE

The Male Urethral Specimen Collection Kit for the **BD ProbeTec™ Qx Amplified DNA Assays** is used to collect and transport male patient urethral specimens to the laboratory for testing with the **BD ProbeTec Qx Amplified DNA Assays** on the **BD Viper™** System in Extracted Mode.

SUMMARY AND EXPLANATION

The Male Urethral Specimen Collection Kit for the **BD ProbeTec Qx Amplified DNA Assays** consists of a sterile swab and a tube of Qx Swab Diluent for specimen transport. The rayon-tipped collection swab consists of a stainless steel wire in a blue-colored plastic shaft that is scored for breaking. After the specimen is collected, the cap of the diluent tube is removed, the swab is placed into the Qx Swab Diluent tube and the shaft is broken at the score mark. The tube is recapped and transported to the testing facility.

REAGENTS

Materials Provided: Each Male Urethral Specimen Collection Kit for the **BD ProbeTec Qx Amplified DNA Assays** contains one sterile rayon-tipped collection swab and one tube of Qx Swab Diluent. The Swab Diluent contains approximately 2 mL of potassium phosphate/potassium hydroxide buffer with DMSO and preservative.

Materials Required But Not Provided: Gloves

Storage Instructions: Store collection kit at 2–33 °C. Do not use beyond the expiration date. Swab sterility is guaranteed if primary swab container is intact.

Warnings and Precautions:

For *in vitro* Diagnostic Use.

WARNING

H302+H312+H332 Harmful if swallowed, in contact with skin or if inhaled.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray. **P280** Wear protective gloves/protective clothing/eye protection/face protection. **P264** Wash thoroughly after handling. **P270** Do not eat, drink or smoke when using this product. **P271** Use only outdoors or in a well-ventilated area. **P321** Specific treatment (see on this label). **P301+P312** IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. **P304+P340** IF INHALED: Remove person to fresh air and keep comfortable for breathing. **P330** Rinse mouth. **P302+P352** IF ON SKIN: Wash with plenty of soap and water. **P362+P364** Take off contaminated clothing and wash it before reuse. **P501** Dispose of contents/container in accordance with local/regional/national/international regulations.

1. Do not pre-wet the swab with the Qx Swab Diluent prior to collecting specimen.
2. Pathogenic microorganisms including hepatitis viruses and Human Immunodeficiency Virus may be present in clinical specimens. "Standard Precautions"¹⁻⁴ and institutional guidelines should be followed in handling all items contaminated with blood and other body fluids.
3. Optimal performance of the **BD ProbeTec Qx Amplified DNA Assays** requires proper specimen collection, handling and transport.
4. Use only collection and transport systems labeled for use with the **BD ProbeTec Qx Amplified DNA Assays**.
5. Proper labeling should accompany each specimen to the laboratory.

6. The Q^x Swab Diluent tube contains dimethyl sulfoxide (DMSO). DMSO is harmful by inhalation, in contact with skin and if swallowed. Keep container in a well-ventilated place. Avoid contact with eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. After contact with skin, wash immediately with plenty of water. Wear suitable protective clothing and gloves.
7. When breaking the shaft of the swab, take care to avoid splashing of contents.
8. Dispose of unused Q^x Swab Diluent tubes according to the Material Safety Data Sheet.
9. Take care to avoid cross-contamination during the specimen handling steps. Specimens can contain high levels of organisms. Ensure that specimen containers do not contact one another, and discard used materials without passing over open containers. If gloves come in contact with specimen, change gloves to avoid cross-contamination.
10. Specimens must be collected and tested before the expiration date of the Q^x Swab Diluent tube.
11. ⓧ Male Urethral Specimen Collection Kit for the **BD ProbeTec Q^x Amplified DNA Assays** is for single use only; reuse may cause a risk of infection and/or inaccurate results.

SPECIMEN COLLECTION AND TRANSPORT

1. Remove the swab from the packaging.
2. Insert the swab 2–4 cm into the urethra and rotate for 3–5 s.
3. Withdraw the swab.
4. Uncap the Q^x Swab Diluent tube.
5. Fully insert the collection swab into the Q^x Swab Diluent tube.
6. Break the shaft of the swab at the score mark. Use care to avoid splashing of contents.
7. **Tightly** recap the tube.
8. Label the tube with patient information and date/time collected.
9. Transport to the laboratory.

Swab Storage and Transport:

The Q^x Swab Diluent tube with swab must be stored and transported to the laboratory and/or test site at 2–30 °C within 30 days of collection. Alternatively, the Swab Diluent tube may be stored and transported frozen (-20 °C) to the laboratory and/or test site within 180 days of collection.

For domestic and international shipments, specimens should be packaged and labeled in compliance with applicable state, federal, and international regulations covering the transport of clinical specimens and etiologic agents/infectious substances. Time and temperature conditions for storage must be maintained during transport.

REFERENCES

1. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, Pa.
2. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guide for isolation precautions in hospitals. *Infect. Control Hospital Epidemiol.* 17:53-80.
3. U. S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
4. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/31/EEC). *Official Journal L262*, 17/10/2000, p. 0021-0045.

Technical Information: In the United States contact BD Technical Service and Support at 1.800.638.8663 or www.bd.com.

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R_x Only

This only applies to US: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / S'applique uniquement aux États-Unis: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Vale solo per gli Stati Uniti: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Gilt nur für die USA: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner." / Sólo se aplica a los EE.UU.: "Caution: Federal Law restricts this device to sale by or on the order of a licensed practitioner."

Caution, consult accompanying documents / Внимание, направте справка в придружаващите документи / Pozor! Pristudujte si priloženou dokumentaci! / Forsigtig, se ledsagende dokumenter / Achtung, Begleitdokumente beachten / Προσοχή, συμβουλευτείτε τα συνοδευτικά έγγραφα / Precaución, consultar la documentación adjunta / Ettevaatust! Lugeka kaasnevat dokumentatsiooni / Attention, consulter les documents joints / Upozorenje, koristiti prateću dokumentaciju / Figyelem! Olvassa el a mellékelt tájékoztatót / Attenzione: consultare la documentazione allegata / Абайлаңыз, тиісті құжаттармен танысыңыз / 주의, 동봉된 설명서 참조 / Dėmesio, žiūrėkite pridedamus dokumentus / Piesardzība, skatīt pavaddokumentus / Voorzichtig, raadpleeg bijgevoegde documenten / Forsiktig, se vedlagt dokumentasjon / Należy zapoznać się z dołączonymi dokumentami / Cuidado, consulte a documentação fornecida / Atenție, consultați documentele însoțitoare / Внимание: см. прилагаемую документацию / Výstraha, pozri sprievodné dokumenty / Pažnja! Pogledajte priložena dokumenta / Obs! Se medföljande dokumentation / Dikkat, birlikte verilen belgelere başvurun / Увара: див. супутню документацию / 小心, 请参阅附带文档。

Do not use if package damaged / Не използвайте, ако опаковката е повредена / Nepoužívejte, je-li obal poškozený / Må ikke anvendes hvis emballagen er beskadiget / Inhalt beschädigter Packung nicht verwenden / Μη χρησιμοποιείτε εάν η συσκευασία έχει υποστεί ζημιό. / No usar si el paquete está dañado / Mitte kasutada, kui pakend on kahjustatud / Ne pas l'utiliser si l'emballage est endommagé / Ne koristiti ako je oštećeno pakiranje / Ne használja, ha a csomagolás sérült / Non usare se la confezione è danneggiata / Erep paket бузылган болса, пайдаланба / 패키지가 손상된 경우 사용 금지 / Jei pakuotė pažeista, nenaudoti / Nelietoti, ja ierakojums bojāts / Niet gebruiken indien de verpakking beschadigd is / Må ikke brukes hvis pakke er skadet / Nie używać, jeśli opakowanie jest uszkodzone / Não usar se a embalagem estiver danificada / A nu se folosi dacă pachetul este deteriorat / He использовать при повреждении упаковки / Nepoužívejte, ak je obal poškodený / Ne koristite ako je pakovanje oštećeno / Använd ej om förpackningen är skadad / Ambalaj hasar görmüşse kullanmayın / Не використовувати за пошкодженої упаковки / 如果包装破损, 请勿使用

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Becton, Dickinson and Company
7 Loveton Circle
Sparks, MD 21152 USA

Australian Sponsor:

Becton Dickinson Pty Ltd.
4 Research Park Drive
Macquarie University Research Park
North Ryde, NSW 2113
Australia

Benex Limited
Pottery Road, Dun Laoghaire
Co. Dublin, Ireland