
Revisions

BALTSO0191 Version 8.0 Template 4
Inserts

Rev from	Rev to	JOB #
01	02	7160-14

NOTES:

1. BD Catalog Number: 245031, 245032
2. Blank (Sheet) Size: Length: 22" Width: 8.5"
3. Number of Pages: 8 Number of Sheets: 1
4. Page Size: Length: 5.5" Width: 8.5" Final Folded Size: 5.5" x 8.5"
5. Ink Colors: No. of Colors: 1 PMS#: Standard Black
6. Printed two sides: Yes ☒ No ☐
7. Style (see illustrations below): # 4

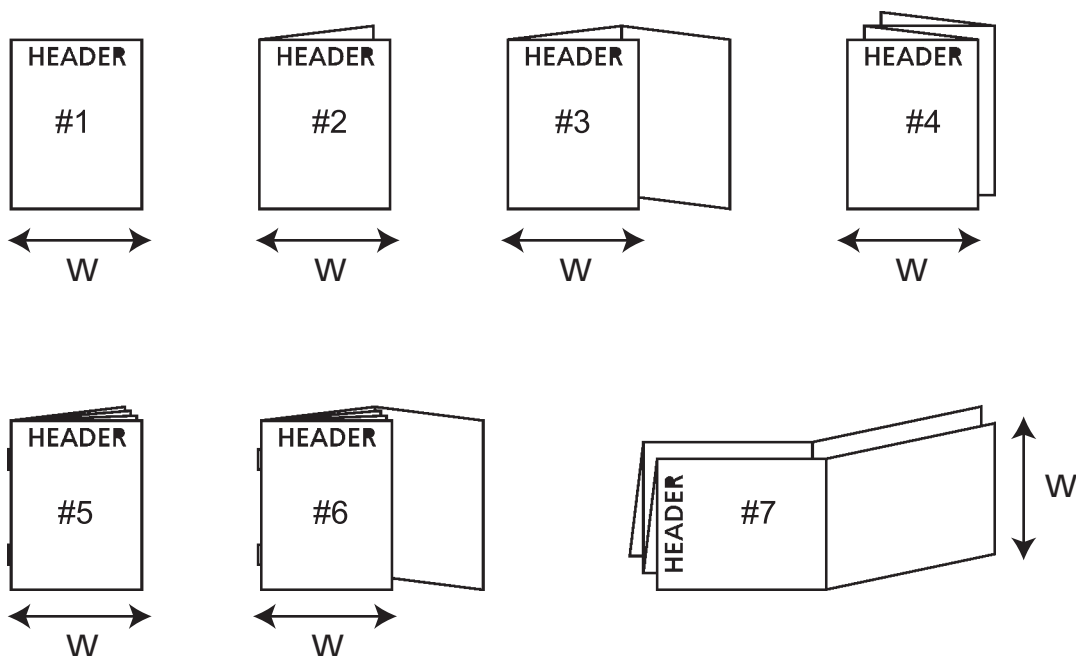8. See specification control no. N/A for material information.
9. Graphics are approved by Becton, Dickinson and Company. Supplier has the responsibility for using the most current approved revision level.

Label Design	REVISED BY By Nancy Carlson at 3:19 pm, Feb 19, 2015	COMPANY CONFIDENTIAL. THIS DOCUMENT IS THE PROPERTY OF BECTON, DICKINSON AND COMPANY AND IS NOT TO BE USED OUTSIDE THE COMPANY WITHOUT WRITTEN PERMISSION.	 BD Becton, Dickinson and Company 7 Loveton Circle Sparks, MD 21152 USA
Proofer	PROOFING APPROVED BY By Helaine Jones at 4:45 pm, Feb 19, 2015		
Checked By	THIRD EYE BY By Kim Stone at 2:01 pm, Feb 20, 2015		
Part Number:	81013301JAA	Category and Description Package Insert, BBL Crystal Panel Viewer Operator Instructions	Sheet: 1 of 9 Scale: N/A
A			

BD BBL™ Crystal™ Panel Viewer Operator's Instructions

English: pages 1 – 2 Italiano: pagine 5 – 6
Français: pages 2 – 3 Español: páginas 6 – 7
Deutsch: Seiten 3 – 4

 81013301JAA(02)
2015-01

Contact your local BD representative for instructions. / Свържете се с местния представител на BD за инструкции. / Pokyny vám poskytne místní zástupce společnosti BD. / Kontakt den lokale BD repræsentant for at få instruktioner. / Επικοινωνήστε με τον τοπικό αντιπρόσωπο της BD για οδηγίες. / Kasutusjuhiste suhtes kontakteeruge oma kohaliku BD esindajaga. / Ota yhteyts lähimpään BD:n edustajaan ohjeiden saamiseksi. / Kontaktiraj lokalnog predstavnika BD za upute. / A használati utasítást kérje a BD helyi képviselőjétől. / Нұсқаулар үшін жергілікті BD өкілімен хабарласыңыз. / Lai saņemtu norādījumus, sazinieties ar vietējo BD pārstāvi. / Naudojimo instrukcijų teiraukitės vietos BD įgaliotojo atstovo. / Neem contact op met uw plaatselijke BD-vertegenwoordiger voor instructies. / Kontakt din lokale BD-representant for mer informasjon. / Aby uzyskać instrukcje użytkowania, skontaktuj się z lokalnym przedstawicielem BD. / Contacte o reprezentante local da BD para instruções. / Pentru instrucțiuni, contactați reprezentantul local BD. / Для получения указания обратитесь к местному представителю компании BD. / Instrukcie ziskate u miestneho zástupcu spoločnosti BD. / Obratite se svom lokalnom predstavniku kompanije BD za uputstva. / Kontakta närmaste BD-representant för anvisningar. / Talimatlar için yerel BD temsilcinizle temasa geçin. / За інструкціями зверніться до місцевого представника компанії BD.

INTENDED USE

The BD BBL™ Crystal™ Panel Viewer is for use with the BD BBL Crystal ID Test Panels.

SUMMARY AND EXPLANATION

The BD BBL Crystal Panel Viewer is designed for use with BD BBL Crystal ID Systems. The Panel Viewer illuminates the BBL Crystal ID System color reactions with either white light or longwave (365 nm) ultraviolet (UV) light depending upon the position of the panel viewer.

PROCEDURES FOR USE

Set the Panel Viewer on a flat dry surface. Insert the plug of the power cord into a three-wire grounded receptacle providing: Domestic (Model No. 245031) 115 volts, 60 Hz AC only; European (Model No. 245032) 230 volts, 50/60 Hz AC only; and Japanese (Model No. 245033) 100 volts, 50/60 Hz. If a two-wire receptacle is encountered, it must be replaced with a properly grounded three-wire receptacle in accordance with the U.S. National Electric Code. Do not remove the grounding plug from the power plug of this unit. When the unit is not in use, the electrical cord may be removed from the unit for storage purposes.

To turn the Panel Viewer on, press the switch located on the right side of the unit to the "I" position. The internal white light will illuminate the lower panel and the ultraviolet light will illuminate the upper chamber. With the slider in the down position, place an inoculated panel on the white light viewing ledge to allow recording of the data. For proper orientation, the lettering across the top of the panel should read A to J and the lettering along the left side should read 4, 2, 1, from top to bottom. Report forms and instructions on scoring the BD BBL Crystal ID System color reactions are included in each BD BBL Crystal ID Systems kit.

To utilize the ultraviolet illumination, slide the panel up into the UV chamber. Viewing through the preformed viewport will allow recording of the UV-induced fluorescence. The BD BBL Crystal Panel Viewer is equipped with a special contrast control filter which blocks out UV light to prevent direct exposure from the longwave UV.

The angle adjustment bracket can be adjusted to raise or lower the angle of the viewer for comfortable viewing of panels. The bracket, located on the back of the unit, can be adjusted by sliding the bracket to the appropriate angle and tightening the knob.

Indoor use, altitude up to 2000 m, temperature 5 to 40 °C, maximum relative humidity of 80% to temperatures up to 31 °C linearly to 50% relative humidity at 40 °C, mains supply voltage not to exceed ± 10 of the nominal voltage. Installation Category II, Pollution degree 2.

Symbol Definition:

 On-Off

 Alternating Current

 Fuse

PREVENTIVE MAINTENANCE

Do not operate the unit when wet. Store unit in a clean, dry area at room temperature (23 to 29 °C). The viewer is painted with a scratch-resistant paint and can be cleaned with commercial cleaners and a damp cloth, WITH THE UNIT UNPLUGGED. To avoid direct UV exposure do not operate the unit when the viewport and filter have been removed for cleaning. Handle the filter glass carefully when cleaning. Power cord is the mains disconnect device.

Warnings and Precautions:

For *in vitro* Diagnostic Use.

Read instructions before attempting to operate this device.

INSTRUCTIONS FOR REPLACEMENT OF TUBES

Turn the Panel Viewer off, disconnect the power cord from the wall receptacle and the unit as instructed in the warning label. Remove back cover of the unit by removing the 11 attachment screws. Remove the defective UV or white light tube from the retaining sockets and replace with a new tube. **Caution: defective tube may be warm to touch.** Reattach the back cover using **ALL** of the screws.

Replacement Tube Part Numbers: White Tube Light – Cat. No. 245036, Longwave (UV) Tube – Cat. No. 245034

SERVICE INFORMATION

The operator should not perform any servicing except replacement of light tubes as described above. For other types of services, contact the following companies for response to questions, or to obtain a Return Goods Authorization (RGA) number to return the unit for repair.

UVP, Inc.
2066 W. 11th Street, Upland, CA 91786, U.S.A.
Toll Free: (800) 452-6788
Outside U.S.A.: (909) 946-3197 • Fax: (909) 946-3597

Ultra Violet Products Limited
Uniti, Trinity Hall Estate, Nuffield Road
Cambridge LB4 1TE U.K.
Telephone: 1223 420022 • Fax: 1223 420561

SPECIFICATIONS:

Domestic Model No. 245031

Voltage	Frequency	Cord	AMPS	Unit Size	Weight	Color	Warranty
115	60 Hz	3-wire grounded Type SV	0.32	12½ x 10½ x 9½ in.	9 lbs.	Black Outer Case	1 yr. from date of purchase

European Model No. 245032

Voltage	Frequency	Cord	AMPS	Unit Size	Weight	Color	Warranty
230	50/60 Hz	3-wire grounded Type SV	0.32	31.75 x 26.67 x 24.13 cm	4.1 kg	Black	1 yr. from date of purchase

Technical Information: In the United States, contact BD Technical Service and Support at 800-638-8663 or www.bd.com/ds.

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Becton Dickinson Pty Ltd.
Benex Limited